



British Medical Auctions Ultrasound Equipment Quality Check (EQC) Preventive Maintenance Service Report (PM Service)

GE / Voluson S8 / VS8804149

Complete

British Medical Auctions

Address: Unit 1, Sterling Centre, Eastern Road, Bracknell, RG12
Tel: +44 1252 299024
Email: enquiries@britishmedicauctions.co.uk
Web: www.britishmedicauctions.co.uk

Department: Service

System has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Safe

System Type: Ultrasound System

System Manufacture: GE

Model: Voluson S8

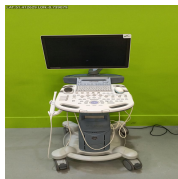


Photo 1

Serial No.: VS8804149



Photo 2

Year of Manufacture: 2020-07

Software Level: BT18

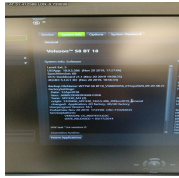


Photo 3



Photo 4

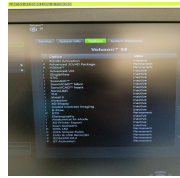


Photo 5

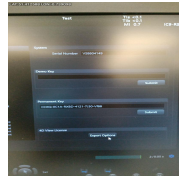


Photo 6

Service Code: 506

Electrical Safety Test Asset ID: 573

Test Equipment No. (Electrical Safety Analyzer):
Rigel 288+
SN.:10M-0263

Calibration Due Date:
05.2024

Test Equipment No. (Ultrasound Phantom):
ATS Labolatories MODEL: 539
Multipurpose Ultrasound Phantom
SN.: 200493148

Date Performed 22.03.2024 11:58 GMT

Performed By PS

1. PRELIMINARY CHECKS

Power Up

Yes

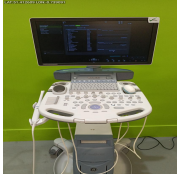


Photo 7

Database Cleared (Including all Patient Data and Previous Owner Details)

Yes

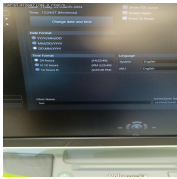


Photo 8

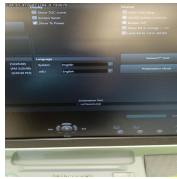


Photo 9



Photo 10

Password Protection

No

Time and Date Check

Yes

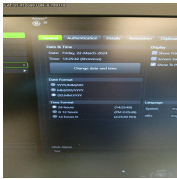


Photo 11

2. PHYSICAL INSPECTION

Input Power

Pass

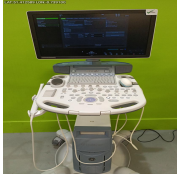


Photo 12

Wheels and Brakes

Pass



Photo 13



Photo 14



Photo 15



Photo 16

Control Panel Movement

Pass



Photo 17



Photo 18



Photo 19

Cables and Connectors

Pass

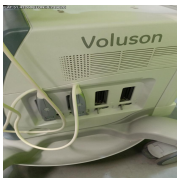


Photo 20



Photo 21

Shielding and Covers

Pass



Photo 22

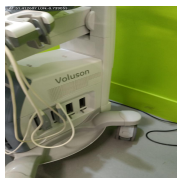


Photo 23



Photo 24



Photo 25



Photo 26



Photo 27

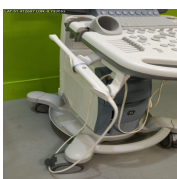


Photo 28

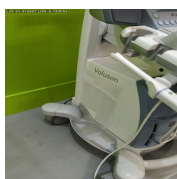


Photo 29



Photo 30

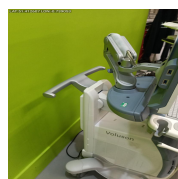


Photo 31

Console

Fail

1. Broken console plastic cover.
2. Broken function buttons x 3

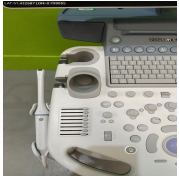


Photo 32



Photo 33

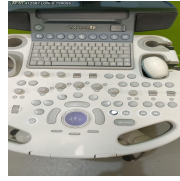


Photo 34



Photo 35

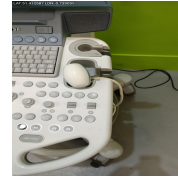


Photo 36



Photo 37

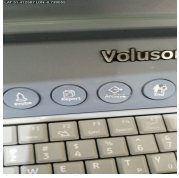


Photo 38



Photo 39

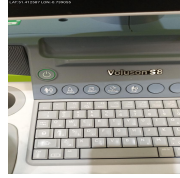


Photo 40

Control Panel

Pass

Monitor Arm

Pass

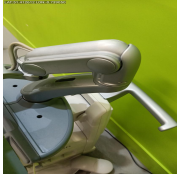


Photo 41



Photo 42

3. CLEANING

Monitor (CRT/LCD)

Yes



Photo 43

Touch screen

N/A

Console

Yes

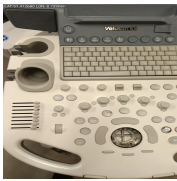


Photo 44

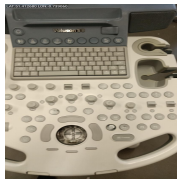


Photo 45



Photo 46

Keyboard

Yes



Photo 47

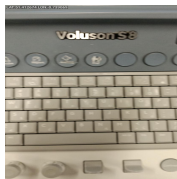


Photo 48



Photo 49



Photo 50

Trackball / Optical mouse

Yes

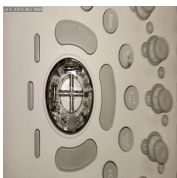


Photo 51

Probe Holder

Yes



Photo 52



Photo 53



Photo 54



Photo 55



Photo 56

Internal Cleaning

Yes



Photo 57



Photo 58



Photo 59

Filter

Yes



Photo 60

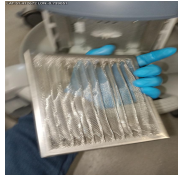


Photo 61

Shielding and Covers

Yes

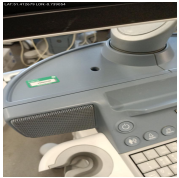


Photo 62



Photo 63



Photo 64



Photo 65



Photo 66



Photo 67

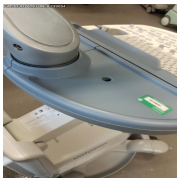


Photo 68

Thermal Printer Head

N/A

4. FUNCTIONAL CHECKS

Monitor

Pass

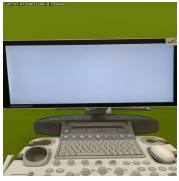


Photo 69



Photo 70



Photo 71



Photo 72



Photo 73

Touch Panel

N/A

Console

Pass



Photo 74

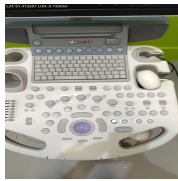


Photo 75

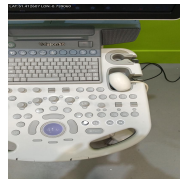


Photo 76

TGC

Pass

1 x broken TGC Button

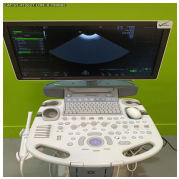


Photo 77

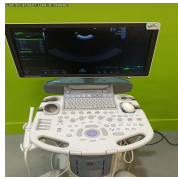


Photo 78



Photo 79

Keyboard

Pass

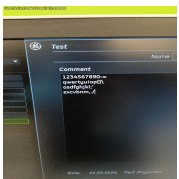


Photo 80

B-Mode

Pass

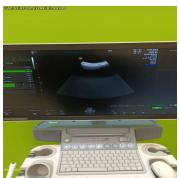


Photo 81

M-Mode

Pass

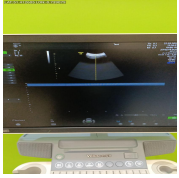


Photo 82

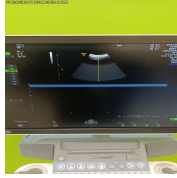


Photo 83

CF-Mode

Pass

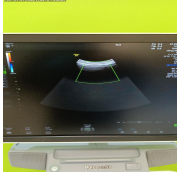


Photo 84

PD-Mode

Pass

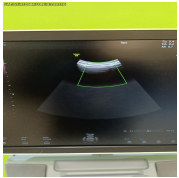


Photo 85

PW-Mode

Pass

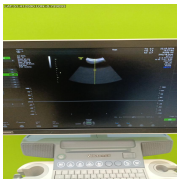


Photo 86

3D Mode

Pass

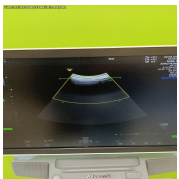


Photo 87



Photo 88



Photo 89

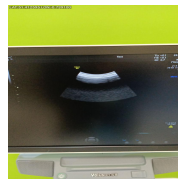


Photo 90



Photo 91

4D Mode

Pass

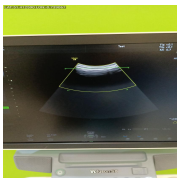


Photo 92



Photo 93

B/W Printer

N/A

Network Connection

N/A

5. PROBE NO.: 1

Probe Make

GE

Probe Model

IC9-RS

Probe Serial Number

877858WX1



Photo 94

Visual inspection

Pass

Cable

Pass



Photo 95

Lens

Pass

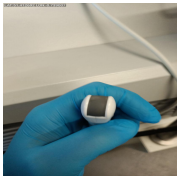


Photo 96



Photo 97

Probe Casing

Pass



Photo 98



Photo 99



Photo 100



Photo 101

Probe Casing Strain Relief

Pass



Photo 102



Photo 103



Photo 104



Photo 105



Photo 106

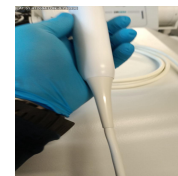


Photo 107

Connector

Pass



Photo 108

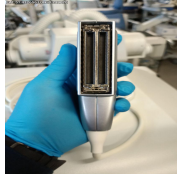


Photo 109



Photo 110



Photo 111

Connector Strain Relief

Pass



Photo 112

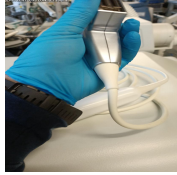


Photo 113



Photo 114



Photo 115

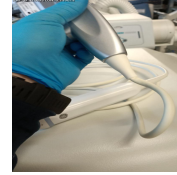


Photo 116

Image benchmark

Pass

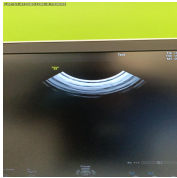


Photo 117

Probe passes inspection & test

Yes

Probe has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Yes

PROBE NO.: 2

Probe Make

GE

Probe Model

RAB6-RS

Probe Serial Number

916259WX5



Photo 118

Visual inspection

Pass

Cable

Pass



Photo 119

Lens

Pass



Photo 120

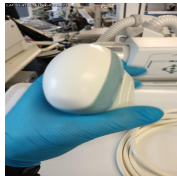


Photo 121



Photo 122



Photo 123

Probe Casing

Pass



Photo 124



Photo 125



Photo 126



Photo 127

Probe Casing Strain Relief

Pass



Photo 128



Photo 129



Photo 130



Photo 131



Photo 132

Connector

Pass



Photo 133



Photo 134



Photo 135



Photo 136

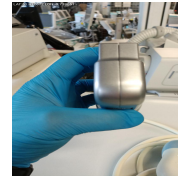


Photo 137

Connector Strain Relief

Pass



Photo 138



Photo 139



Photo 140



Photo 141

Image benchmark

Pass



Photo 142

Probe passes inspection & test

Yes

Probe has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Yes

6. SUMMARY

This service has been completed in accordance with our quality procedure.
If you have any questions or concerns, please contact us on
+44 1252 299024
or email: enquiries@britishmedicalauctions.co.uk to discuss further.

System has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Safe

Date Performed:

22.03.2024 13:37 GMT

Service Engineer Signature



PS
22.03.2024 13:37 GMT

British Medical Auctions
Unit 1, Sterling Centre, Eastern Road, Bracknell, RG12 2PW
Tel: +44 1252 299024
Email: enquiries@britishmedicalauctions.co.uk
Web: www.britishmedicalauctions.co.uk

British Medical Auctions
Ultrasound Equipment Quality Check (EQC)
Preventive Maintenance Service Report (PM Service)

V1.0 05.2022

7. PHOTOGRAPHIC DOCUMENTATION



Photo 143



Photo 144

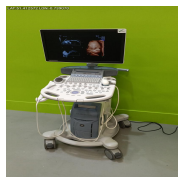


Photo 145



Photo 146

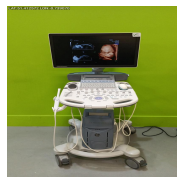


Photo 147

Media summary



Photo 1

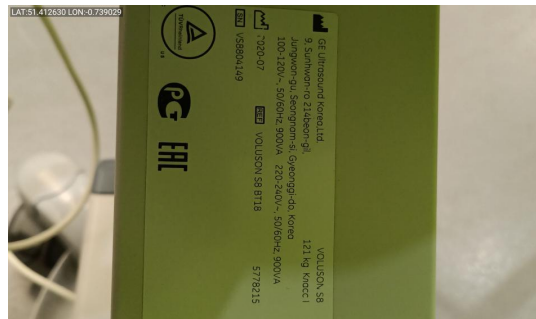


Photo 2

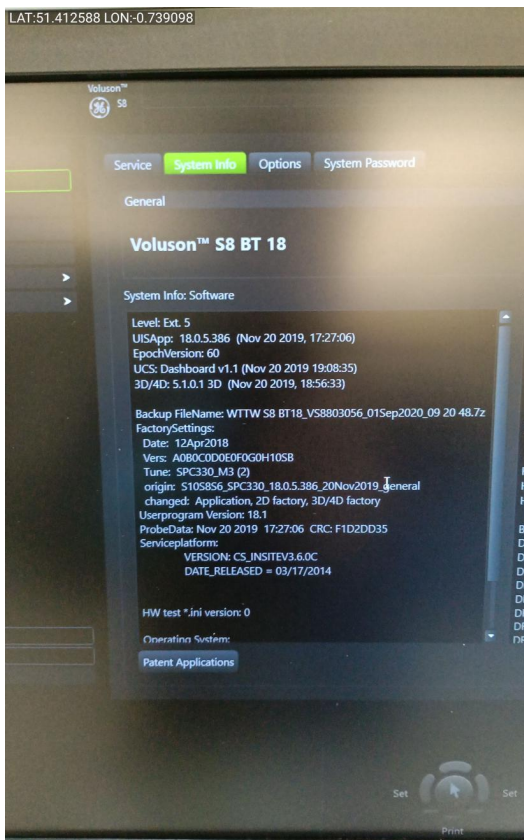


Photo 3

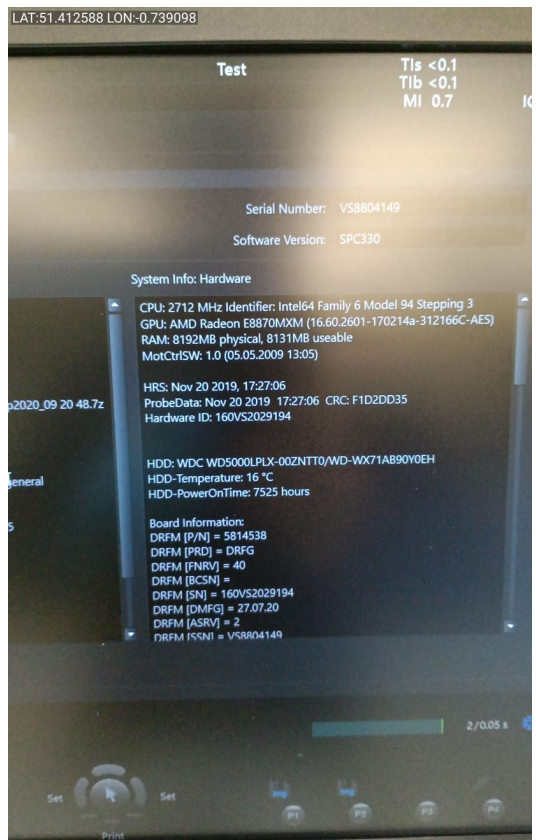


Photo 4

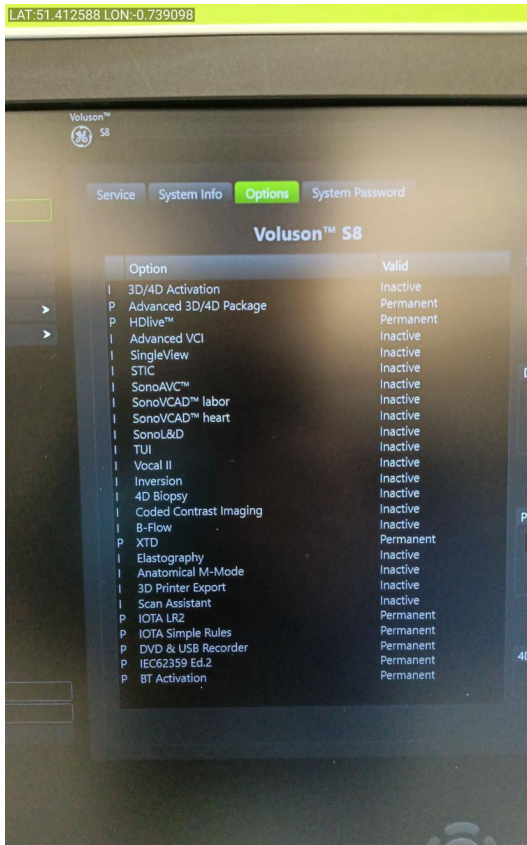


Photo 5

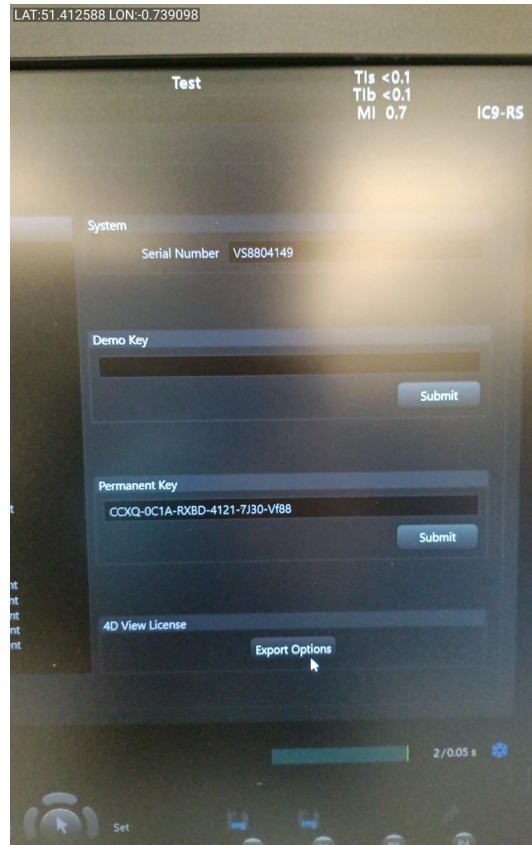


Photo 6



Photo 7

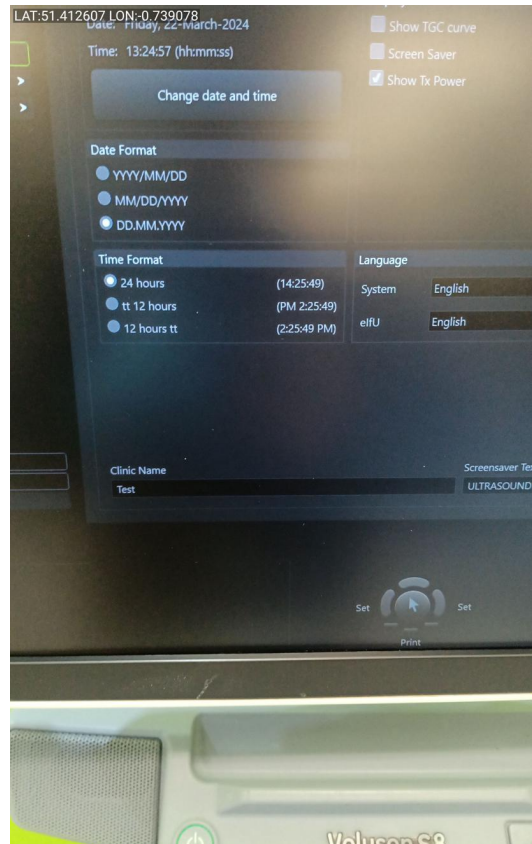


Photo 8

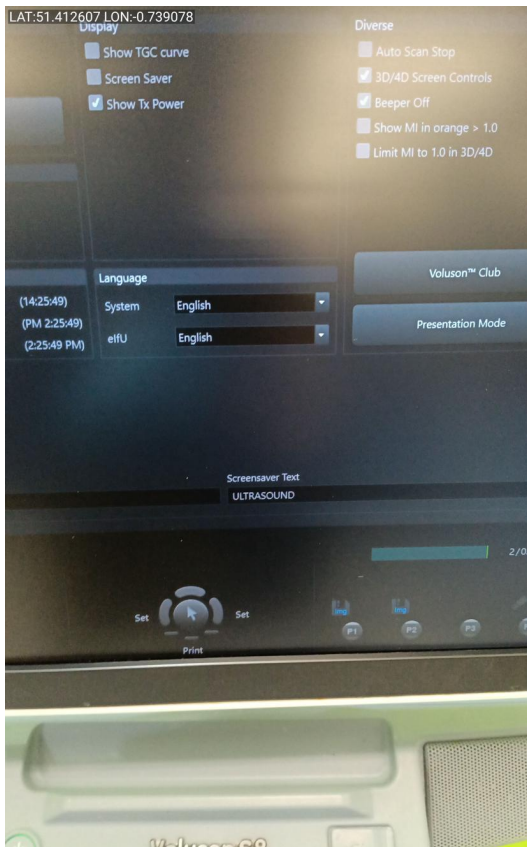


Photo 9

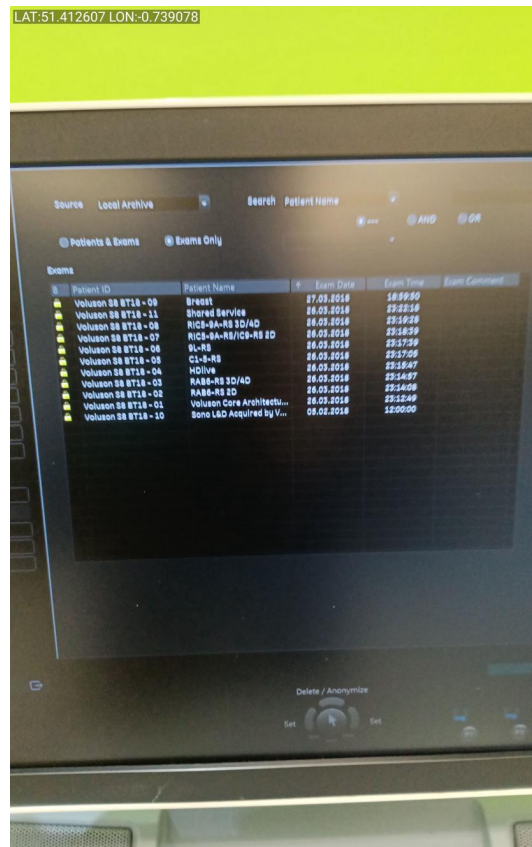


Photo 10

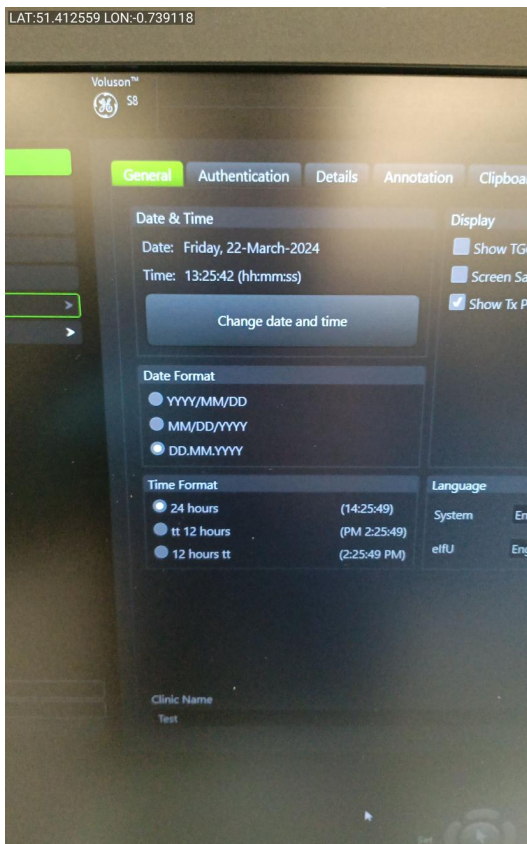


Photo 11

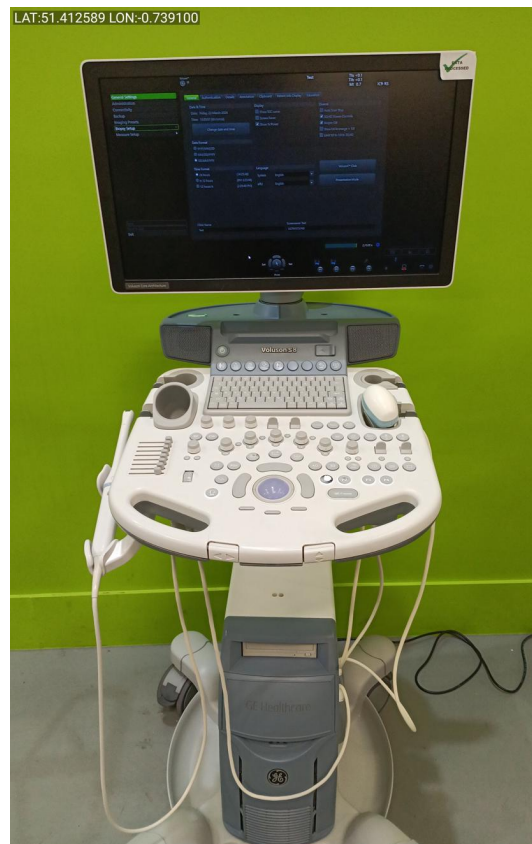


Photo 12



Photo 13



Photo 14



Photo 15

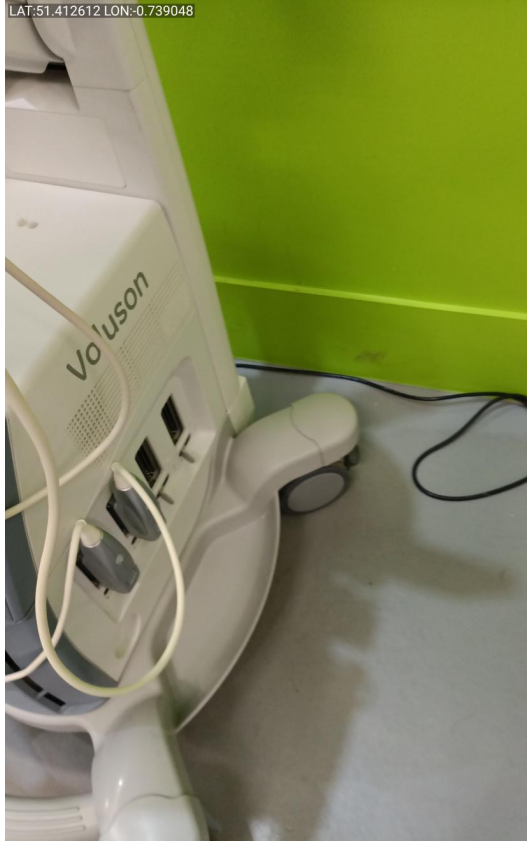


Photo 16

LAT:51.412612 LON:-0.739048



Photo 17

LAT:51.412612 LON:-0.739048



Photo 18

LAT:51.412612 LON:-0.739048



Photo 19

LAT:51.412608 LON:-0.739070

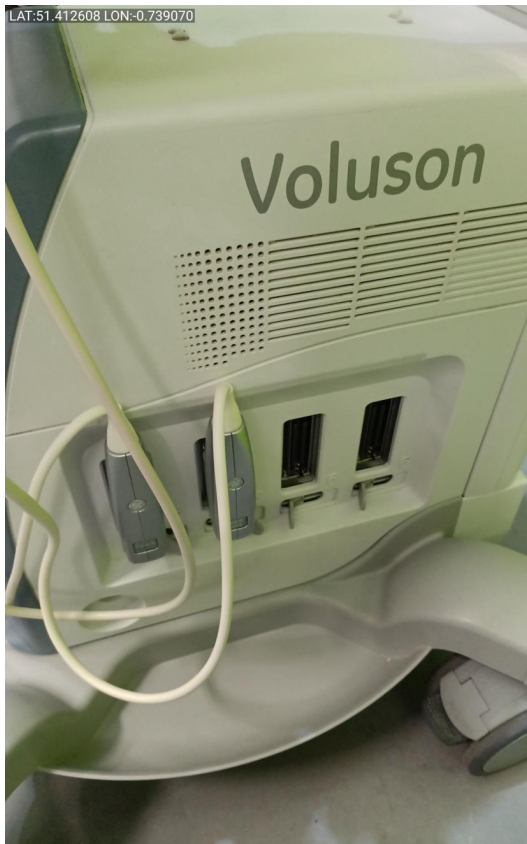


Photo 20



Photo 21

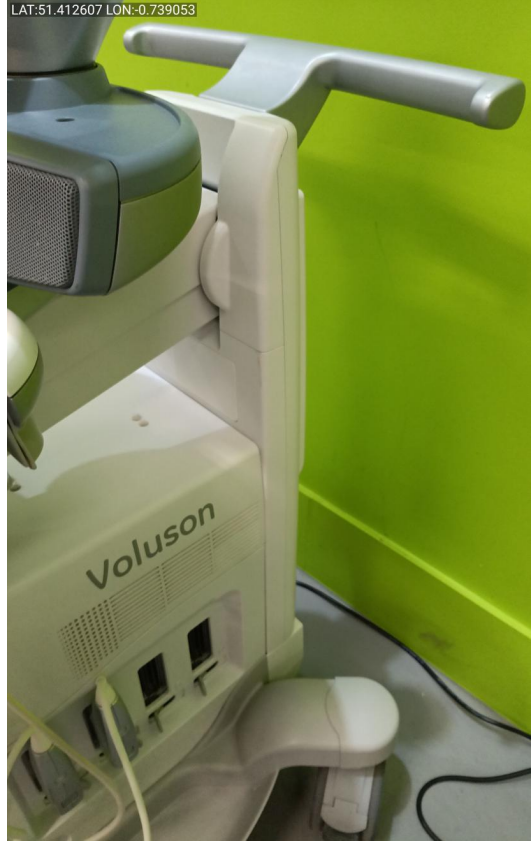


Photo 22

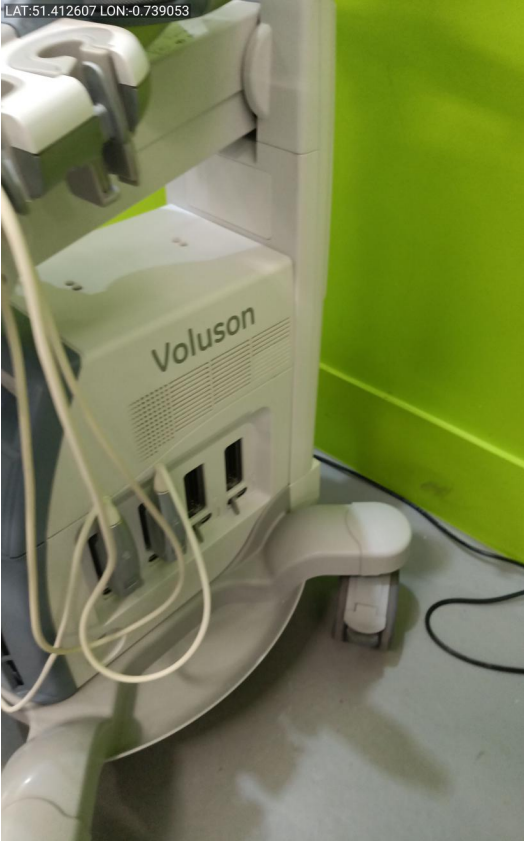


Photo 23

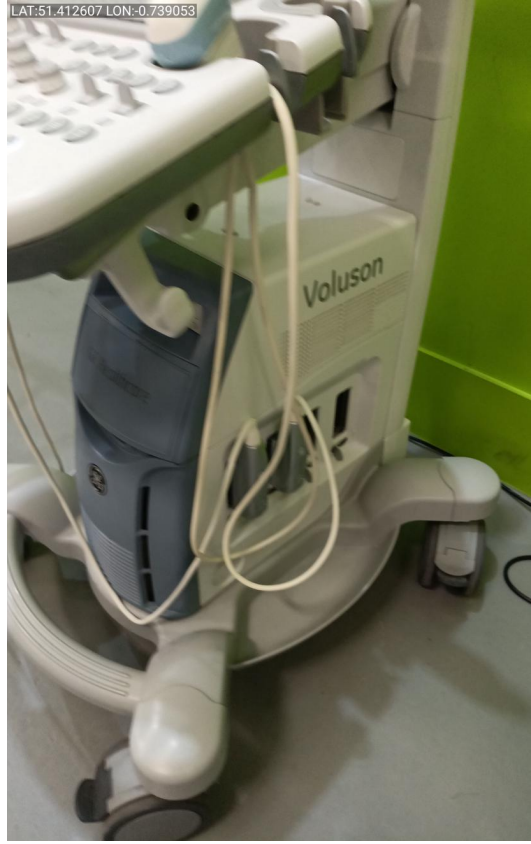


Photo 24



Photo 25



Photo 26



Photo 27

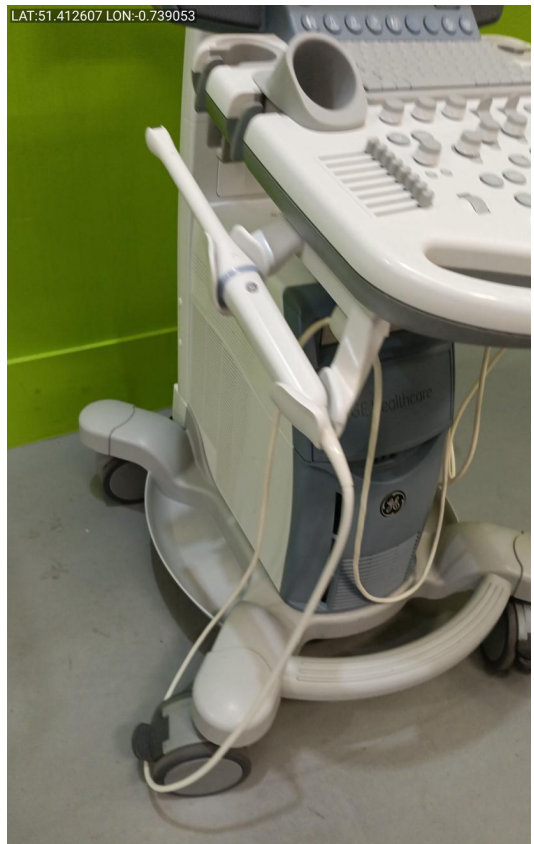


Photo 28



Photo 29



Photo 30



Photo 31

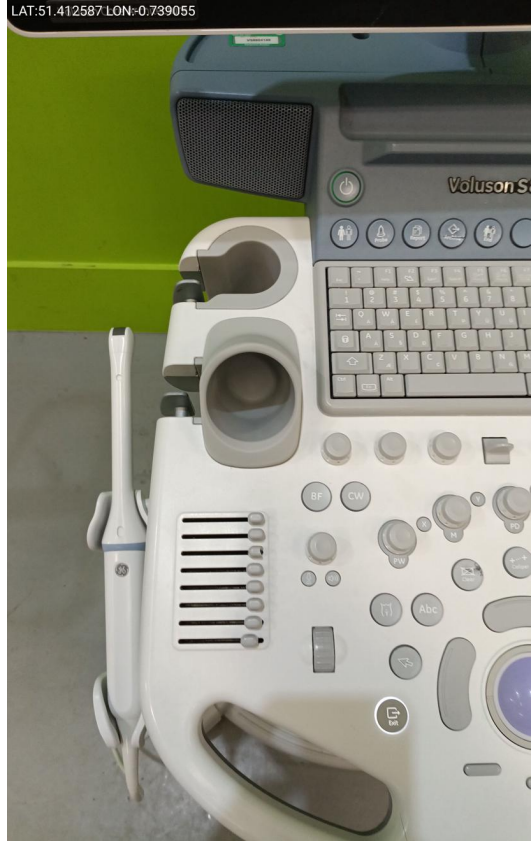


Photo 32



Photo 33



Photo 34

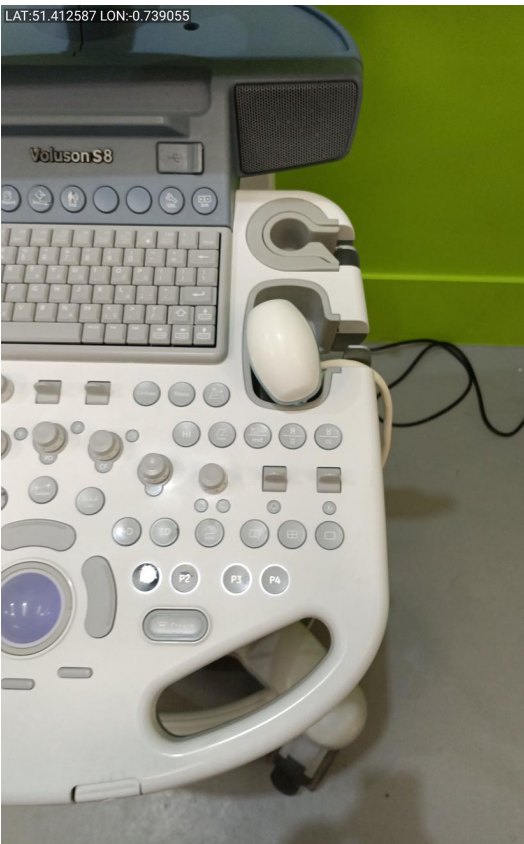


Photo 35



Photo 36

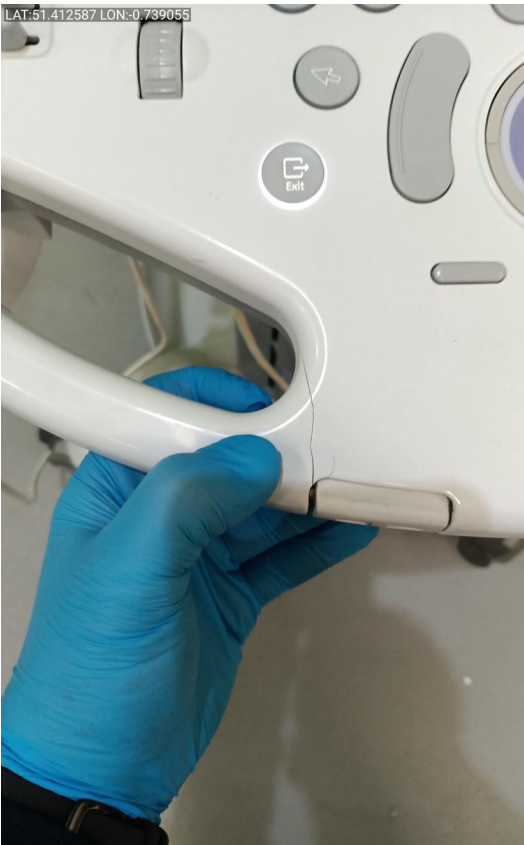


Photo 37

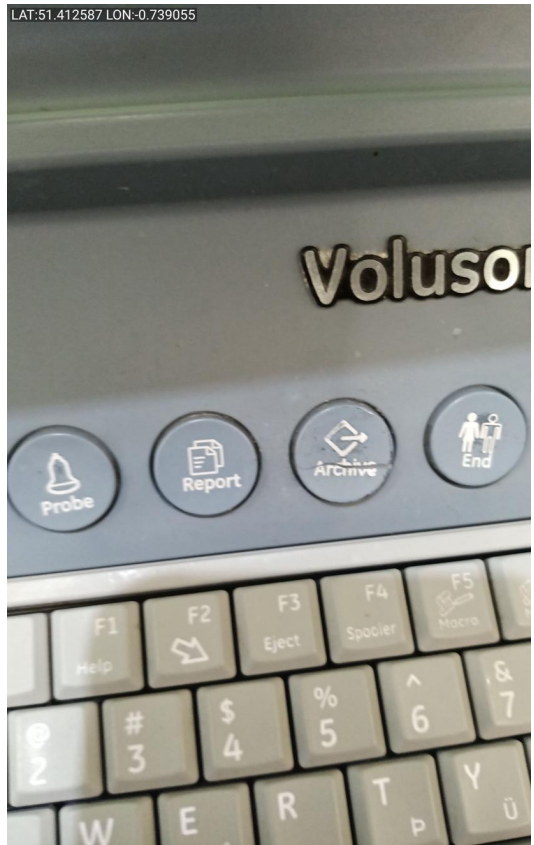


Photo 38

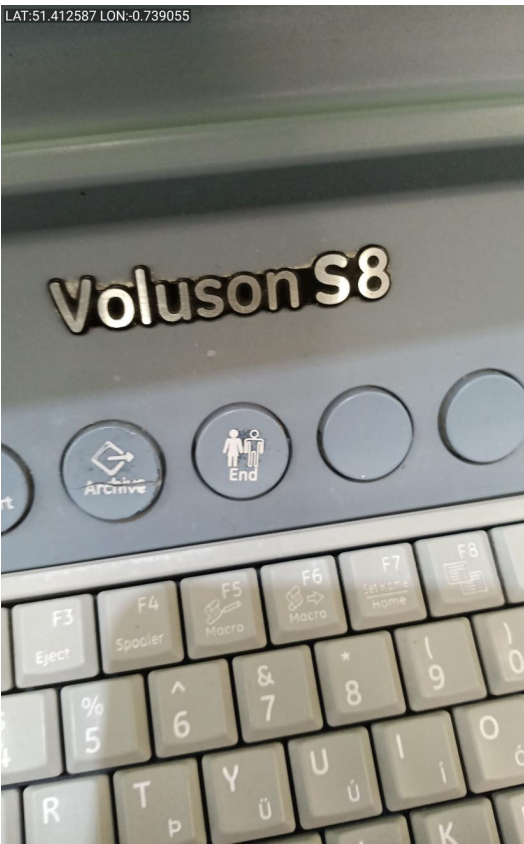


Photo 39

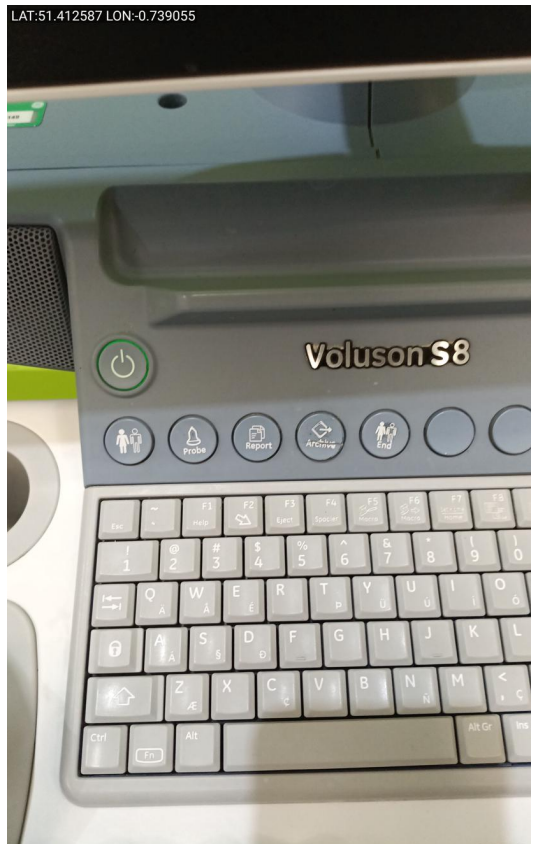


Photo 40



Photo 41

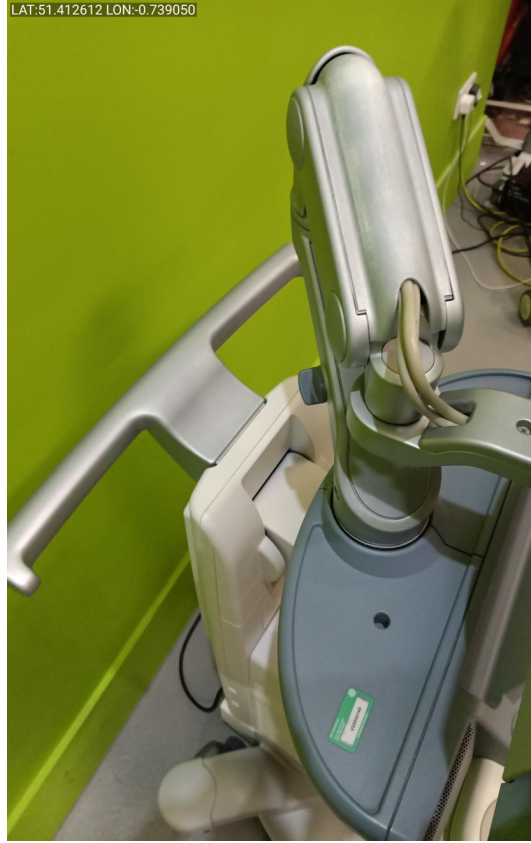


Photo 42



Photo 43



Photo 44



Photo 45



Photo 46

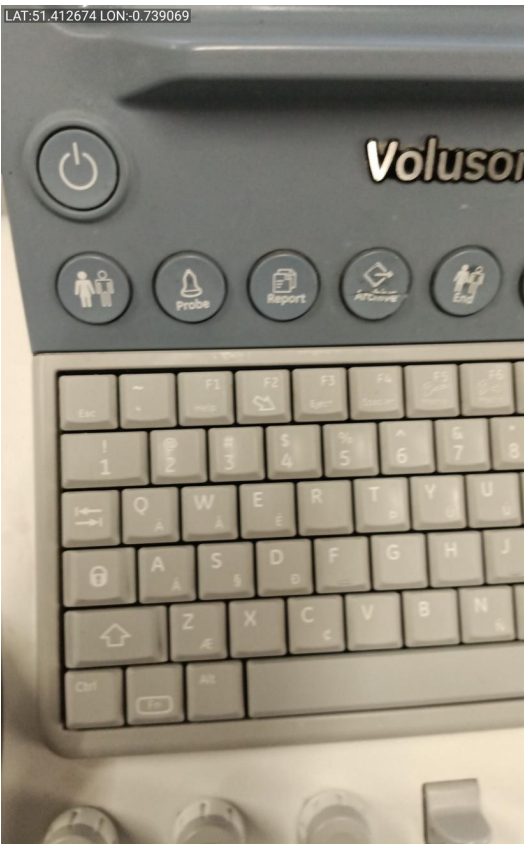


Photo 47

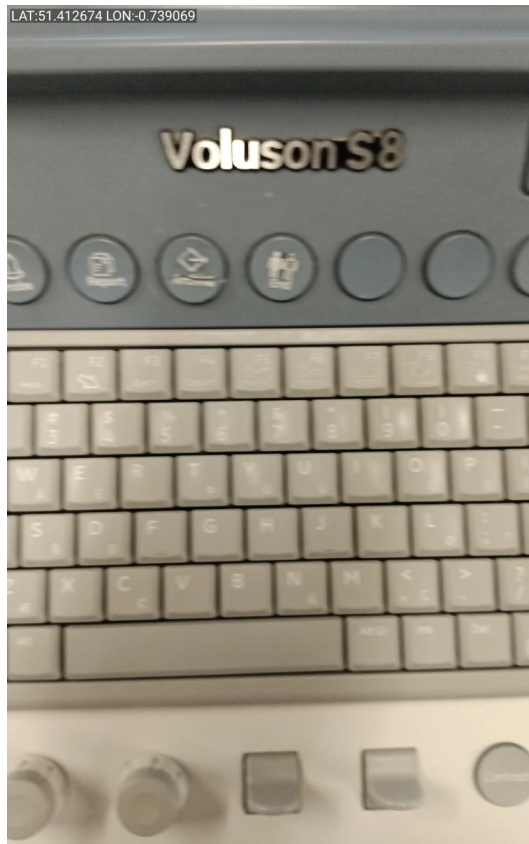


Photo 48

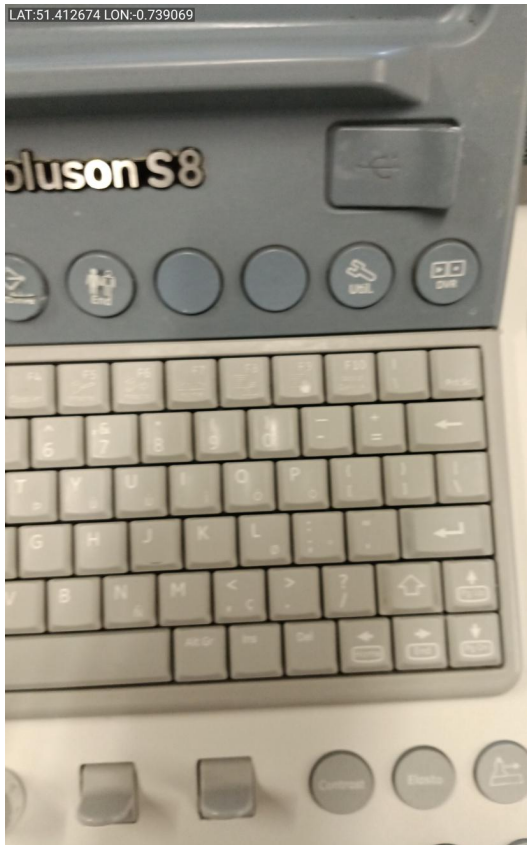


Photo 49



Photo 50



Photo 51



Photo 52



Photo 53



Photo 54



Photo 55

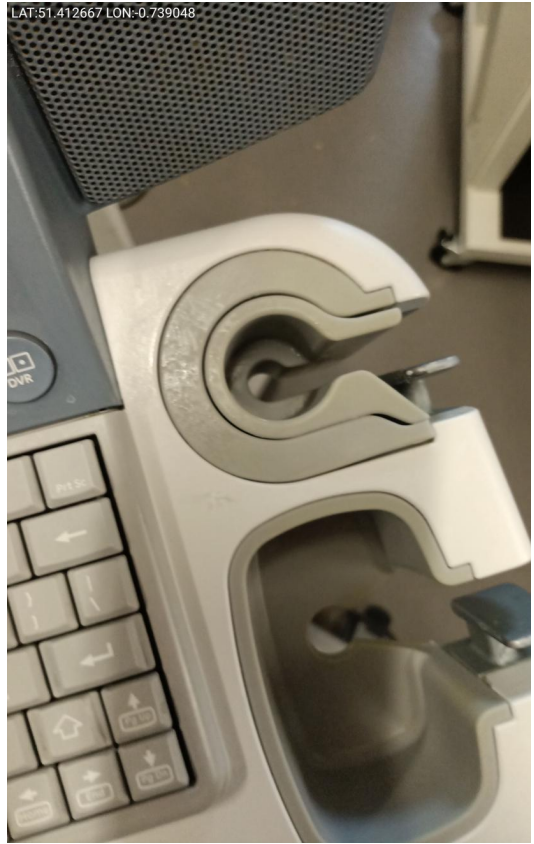


Photo 56



Photo 57



Photo 58



Photo 59



Photo 60



Photo 61



Photo 62



Photo 63



Photo 64



Photo 65



Photo 66



Photo 67



Photo 68

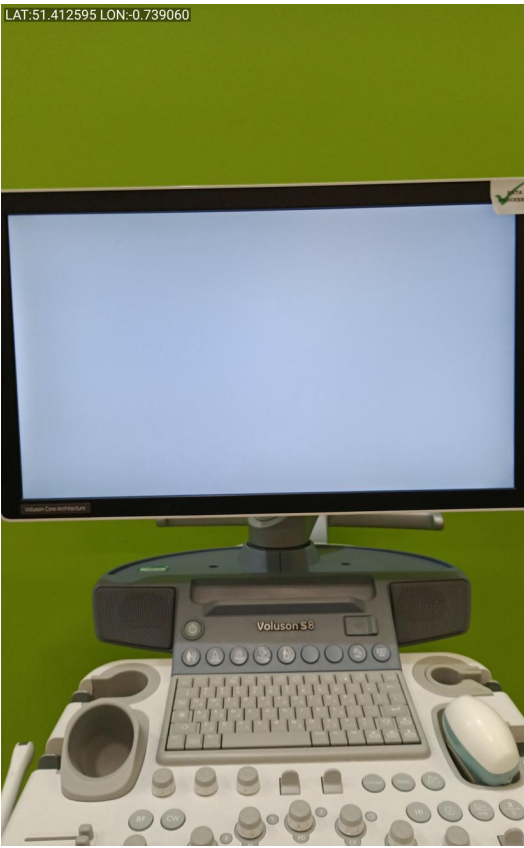


Photo 69

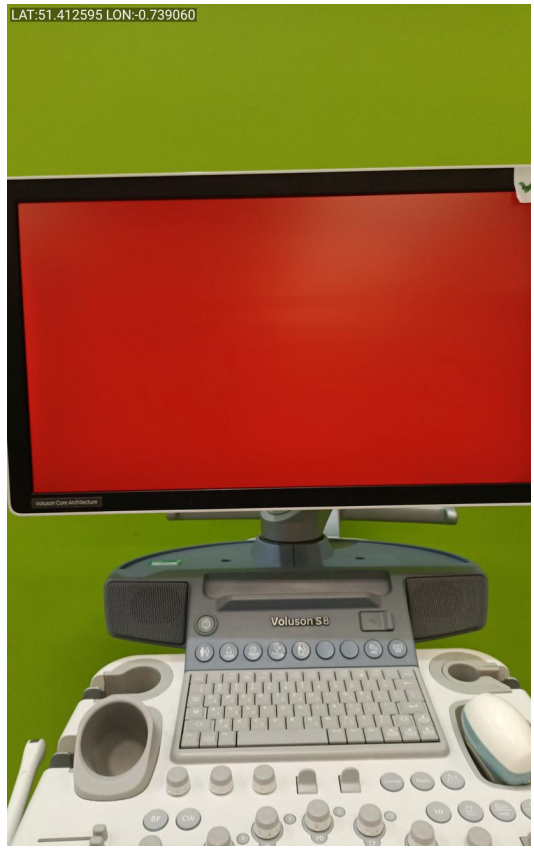


Photo 70

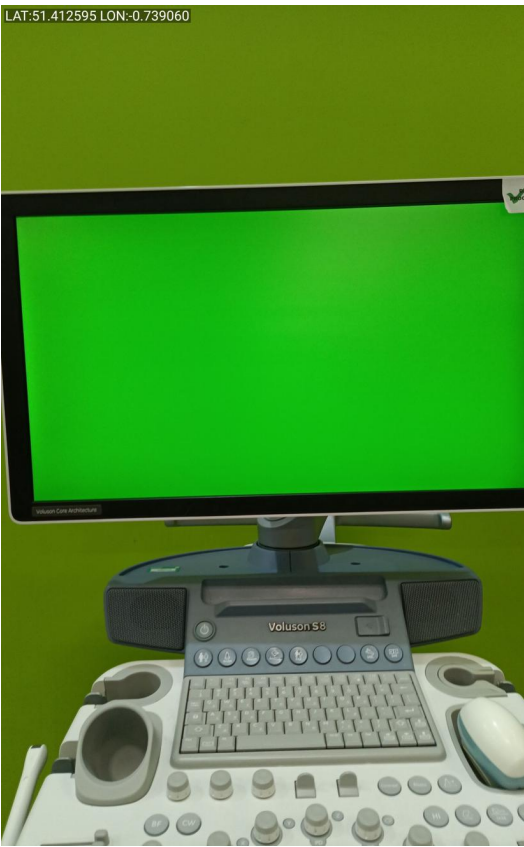


Photo 71

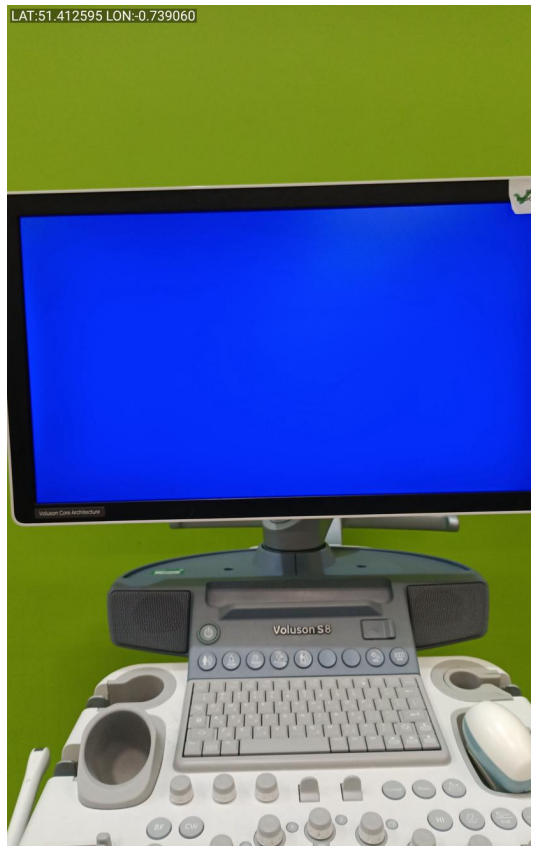


Photo 72

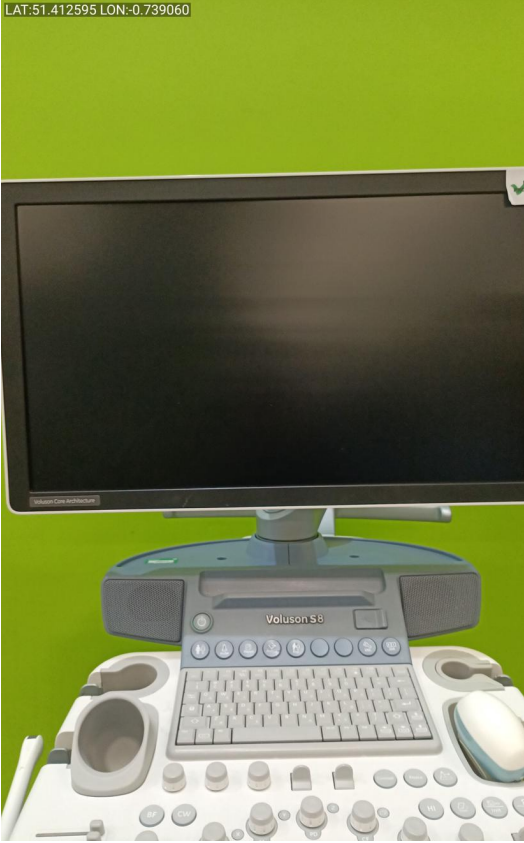


Photo 73



Photo 74

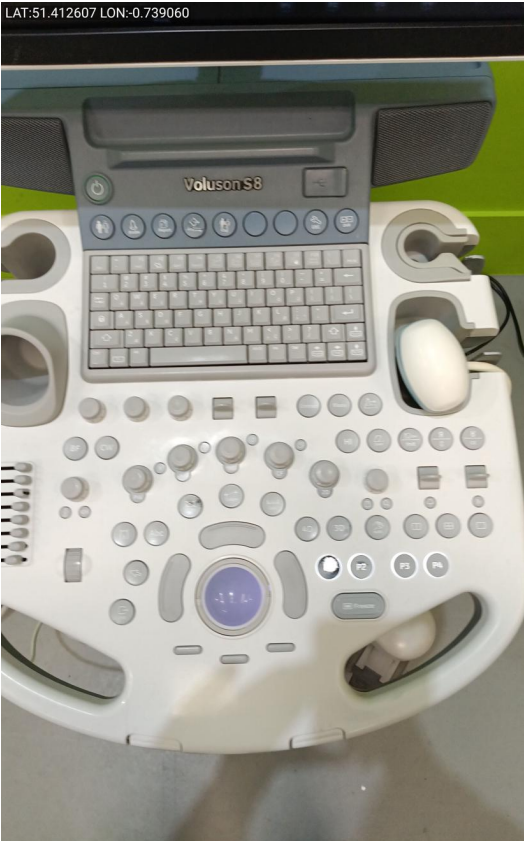


Photo 75

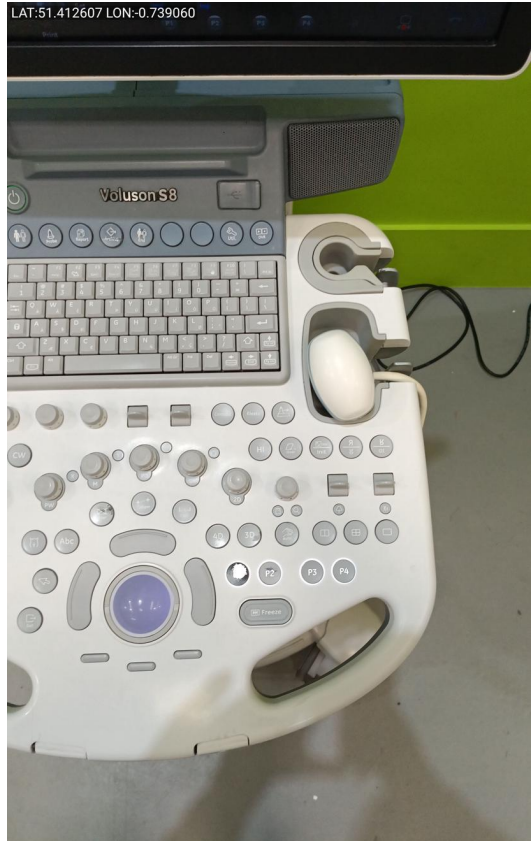


Photo 76



Photo 77



Photo 78



Photo 79

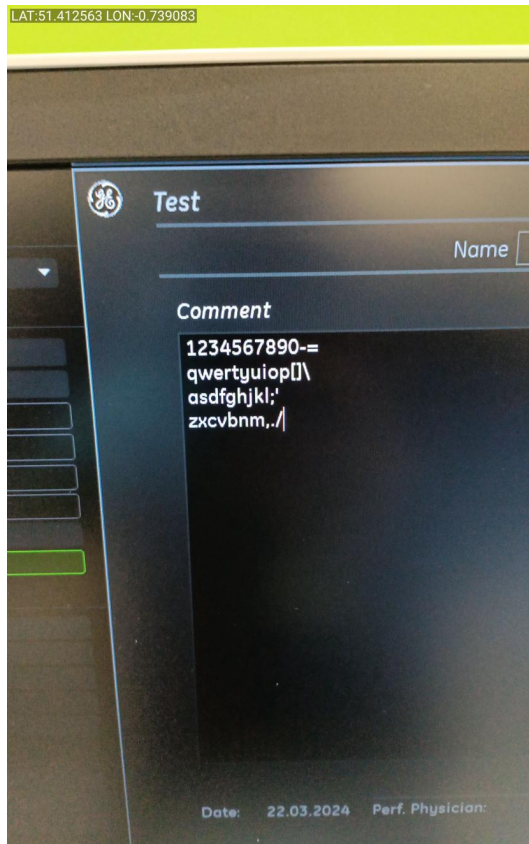


Photo 80



Photo 81

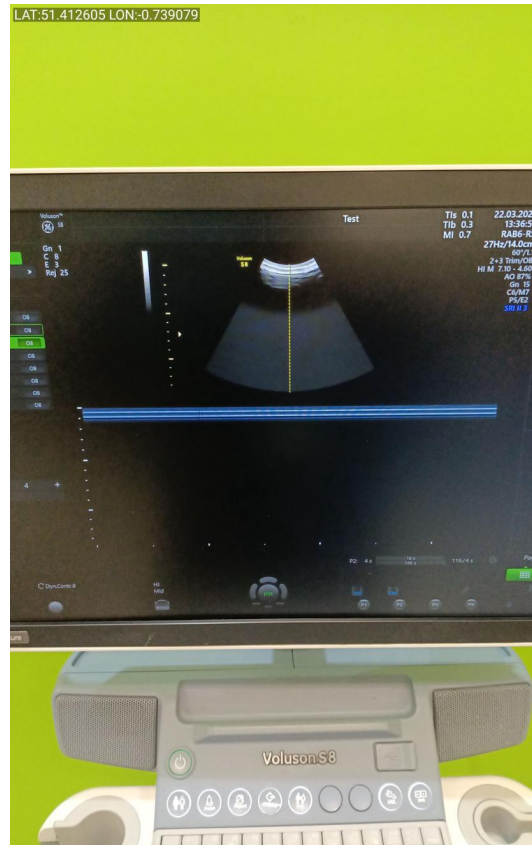


Photo 82



Photo 83

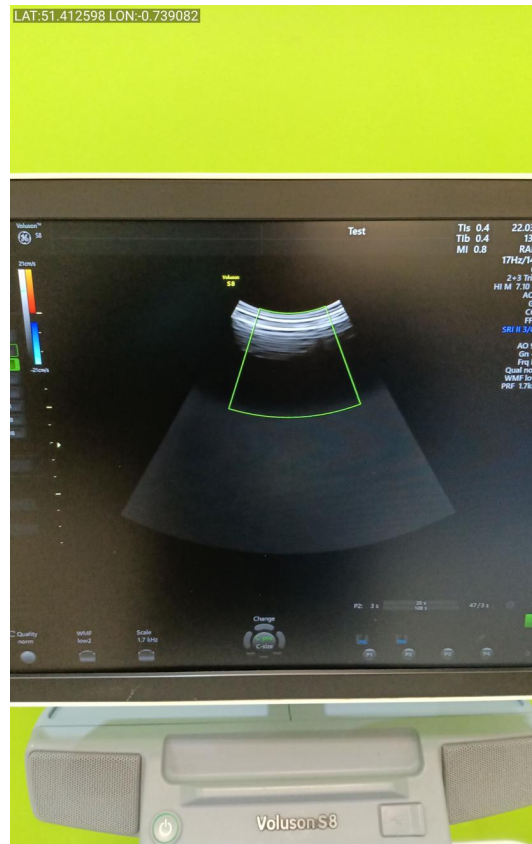


Photo 84

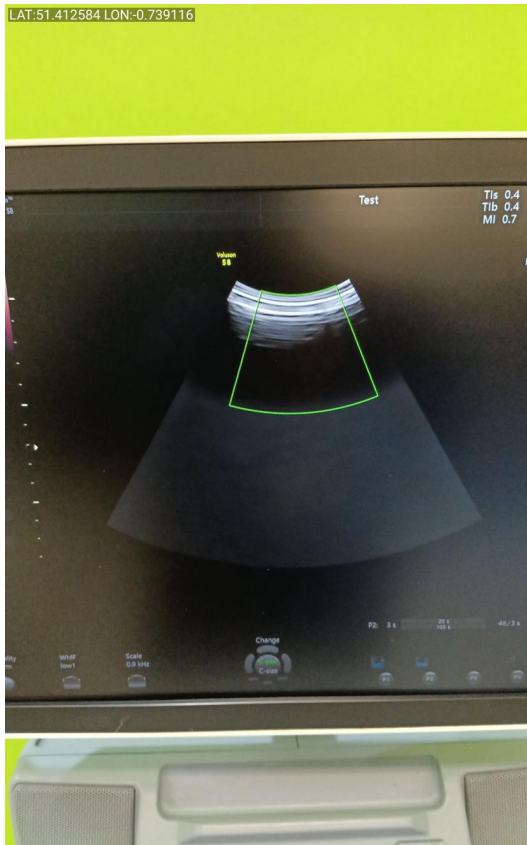


Photo 85



Photo 86

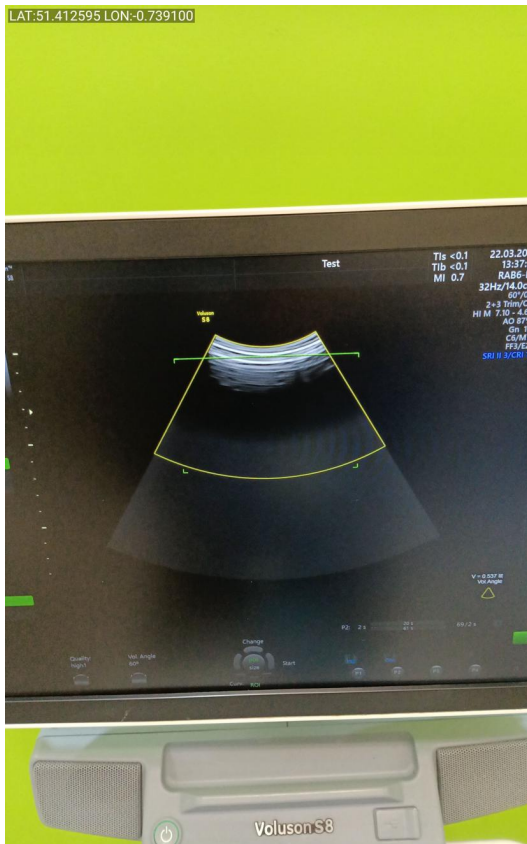


Photo 87



Photo 88



Photo 89



Photo 90



Photo 91

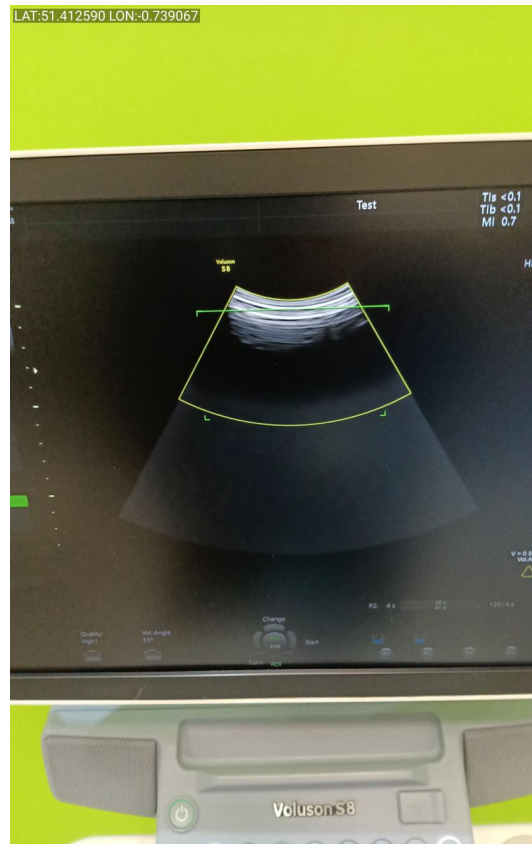


Photo 92

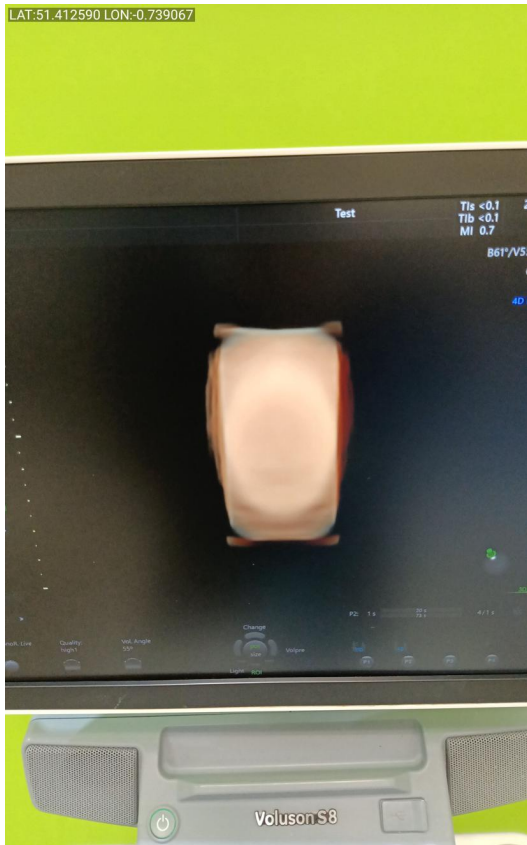


Photo 93



Photo 94



Photo 95



Photo 96



Photo 97



Photo 98



Photo 99



Photo 100



Photo 101



Photo 102



Photo 103



Photo 104



Photo 105



Photo 106



Photo 107



Photo 108

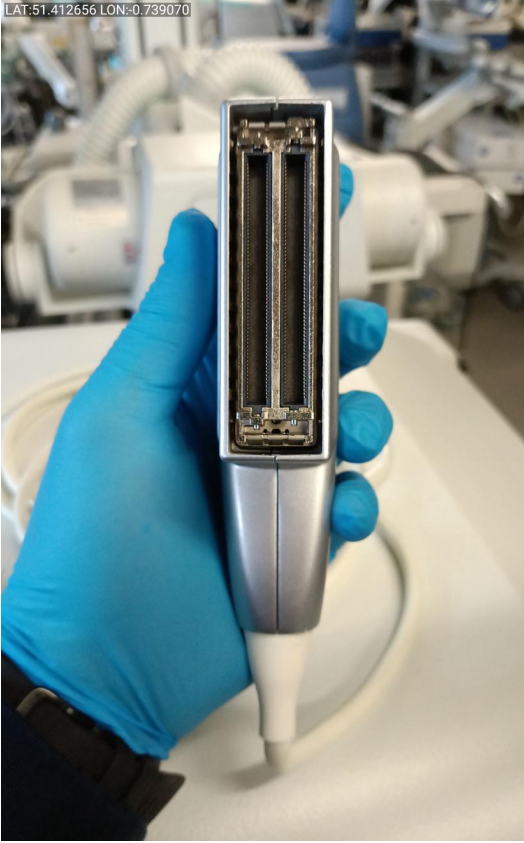


Photo 109

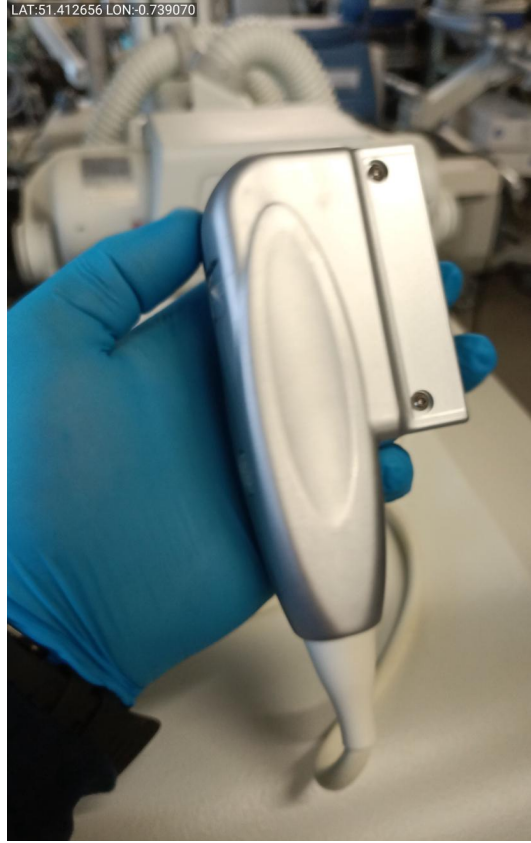


Photo 110

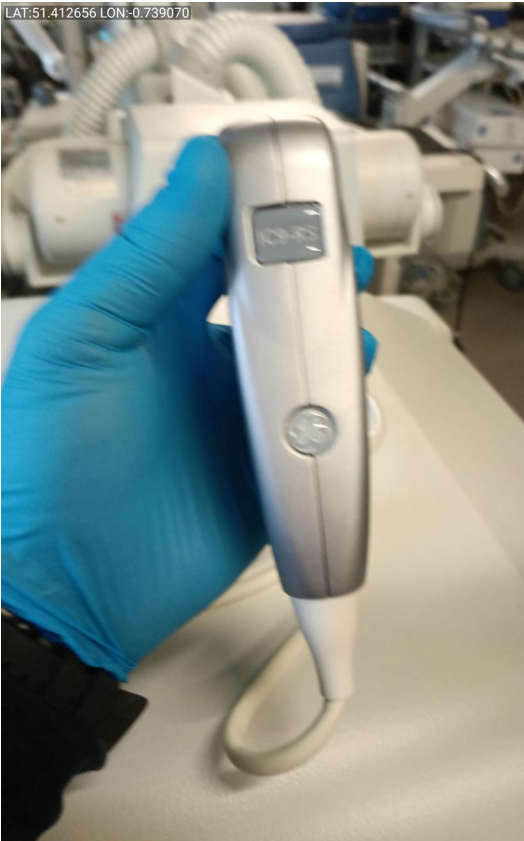


Photo 111



Photo 112



Photo 113



Photo 114

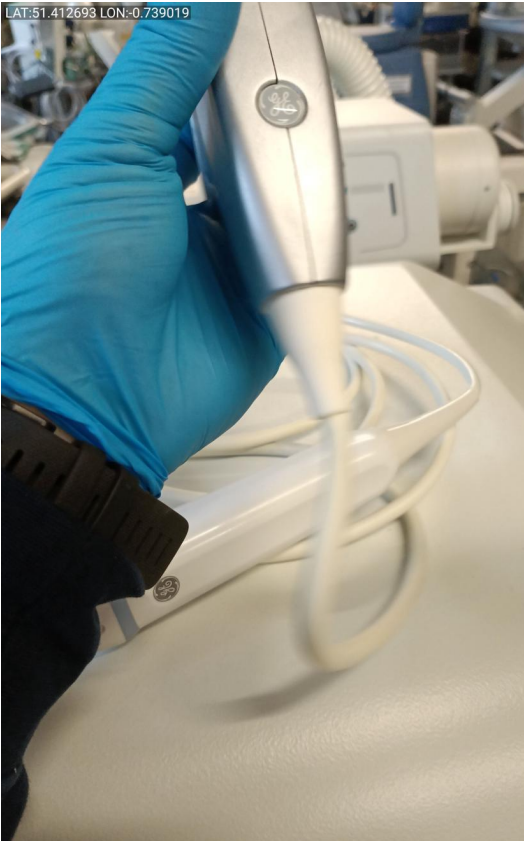


Photo 115

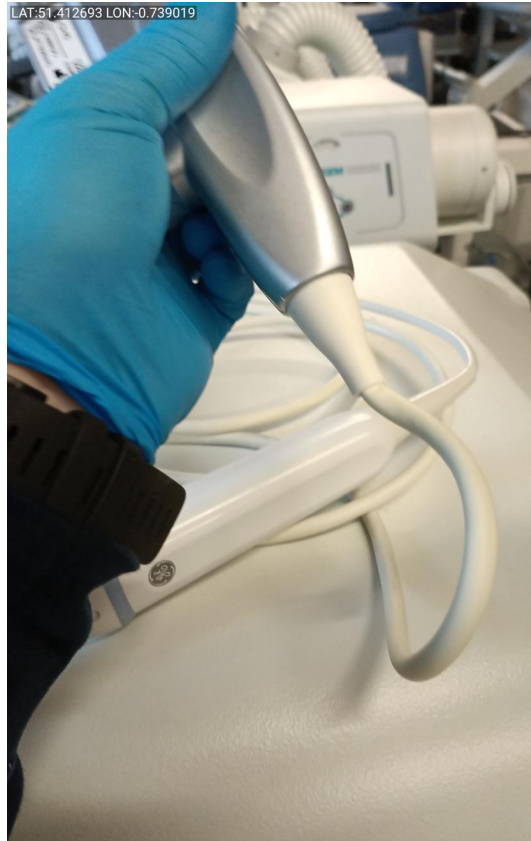


Photo 116

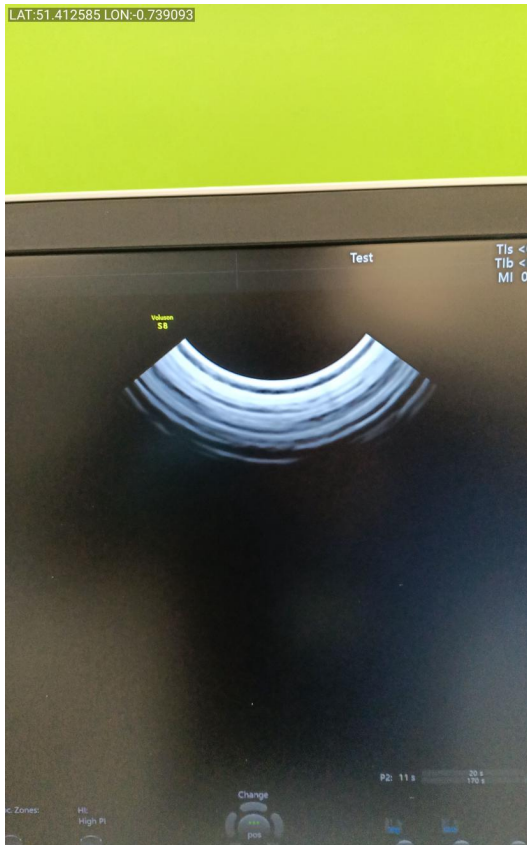


Photo 117



Photo 118



Photo 119



Photo 120



Photo 121

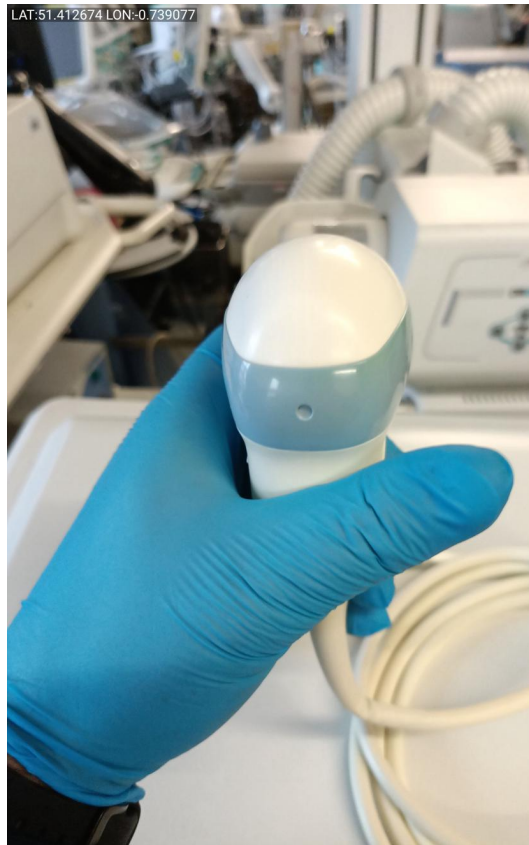


Photo 122

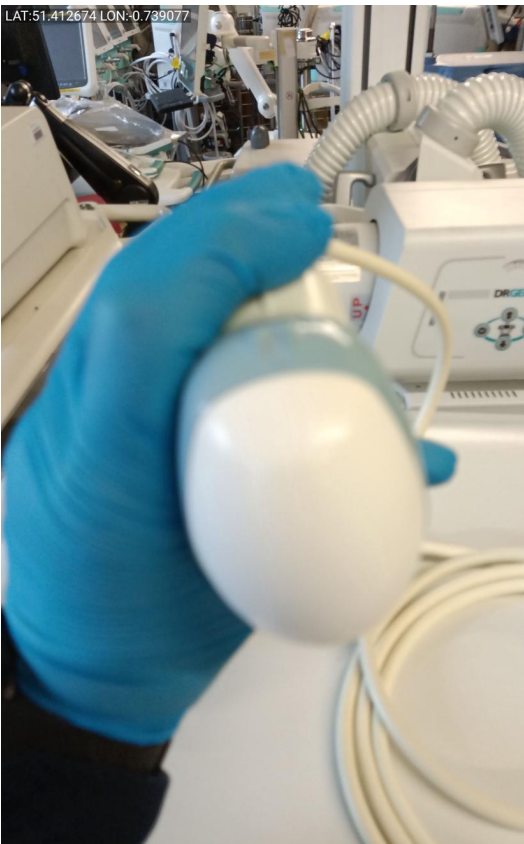


Photo 123



Photo 124



Photo 125



Photo 126



Photo 127



Photo 128



Photo 129



Photo 130



Photo 131



Photo 132



Photo 133

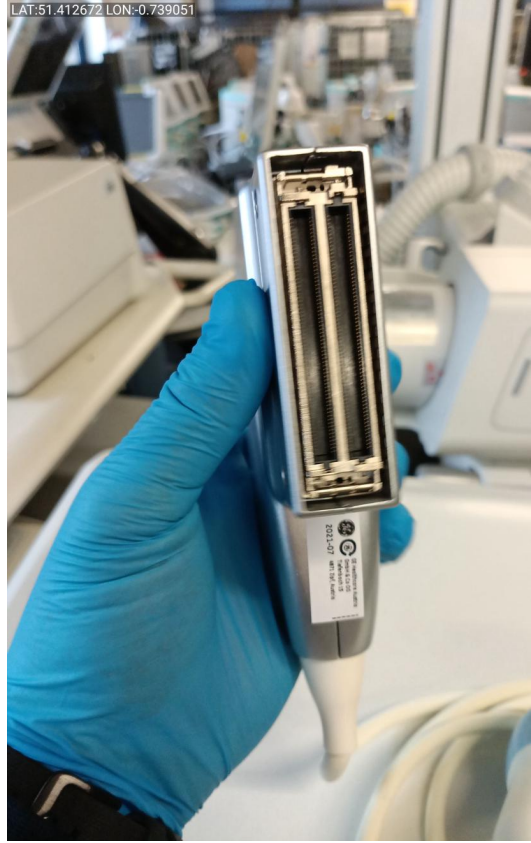


Photo 134



Photo 135



Photo 136



Photo 137

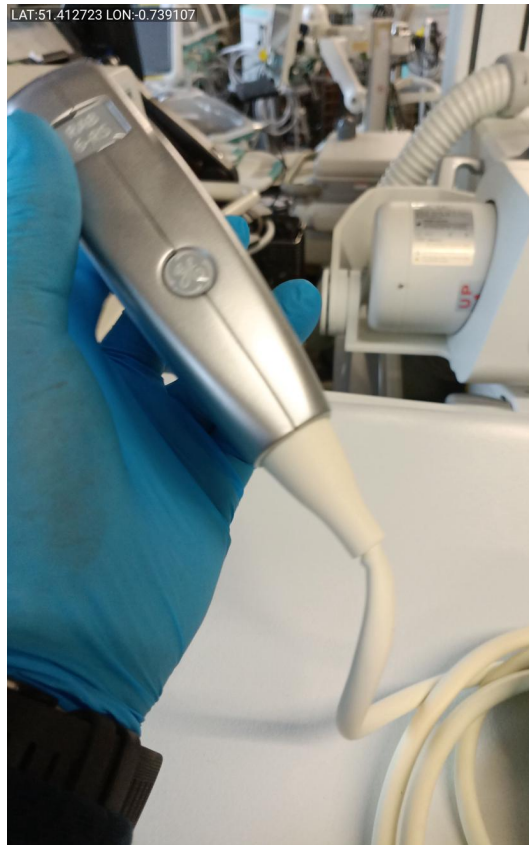


Photo 138



Photo 139



Photo 140



Photo 141

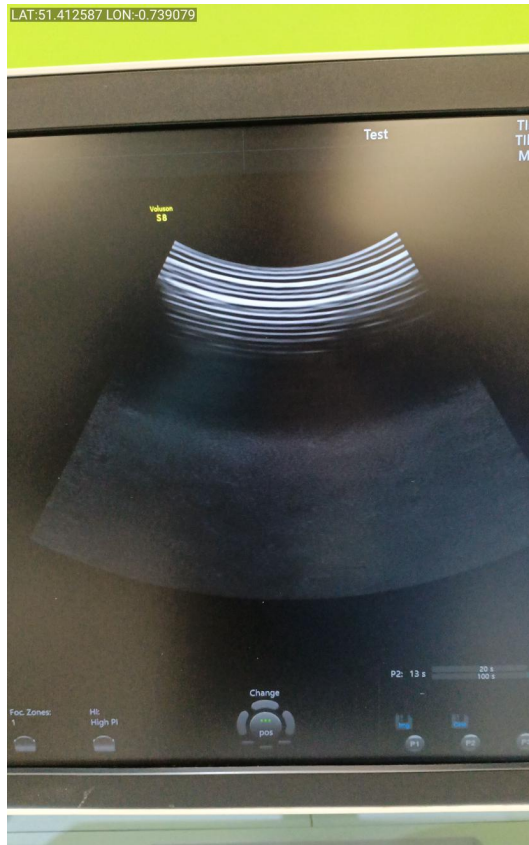


Photo 142



Photo 143



Photo 144

LAT:51.412572 LON:-0.739070



Photo 145

LAT:51.412572 LON:-0.739070



Photo 146

LAT:51.412572 LON:-0.739070



Photo 147