



British Medical Auctions Ultrasound Equipment Quality Check (EQC) Preventive Maintenance Service Report (PM Service)

GE / Voluson E8 / E45782

Complete

British Medical Auctions

Address: Unit 1, Sterling Centre, Eastern Road, Bracknell, RG12 2PW
Tel: +44 1252 299024
Email: enquiries@britishmedicauctions.co.uk
Web: www.britishmedicauctions.co.uk

Department: Service

System has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Safe

System Type:

Ultrasound System



Photo 1

System Manufacture:

GE

Model:

Voluson E8

Serial No.:

E45782



Photo 2

Year of Manufacture:

06.2020

Software Level:

BT20

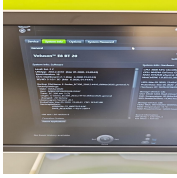


Photo 3

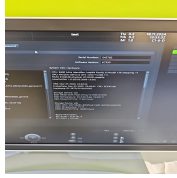


Photo 4

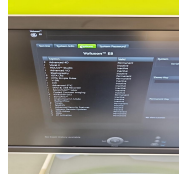


Photo 5

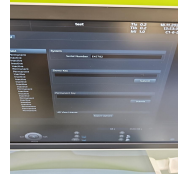


Photo 6

Service Code: 721

Electrical Safety Test Asset ID: 743

Test Equipment No. (Electrical Safety Analyzer):
Rigel 288+
SN.:10M-0263

Calibration Due Date:
05.2025

Test Equipment No. (Ultrasound Phantom):
ATS Laboratories MODEL: 539
Multipurpose Ultrasound Phantom
SN.: 200493148

Date Performed 18.11.2024 13:20 CET

Performed By MS

1. PRELIMINARY CHECKS

Power Up

Yes



Photo 7

Database Cleared (Including all Patient Data and Previous Owner Details)

Yes



Photo 8

Password Protection

No

Time and Date Check

Yes



Photo 9

2. PHYSICAL INSPECTION

Input Power

Pass



Photo 10

External I/O

Pass

Wheels and Brakes

Pass



Photo 11



Photo 12



Photo 13



Photo 14

Control Panel Movement

Pass



Photo 15



Photo 16



Photo 17



Photo 18

Cables and Connectors

Pass

Shielding and Covers

Pass



Photo 19



Photo 20



Photo 21

Console

Pass



Photo 22



Photo 23

Control Panel

Pass



Photo 24

Monitor Arm

Pass



Photo 25

3. CLEANING

Monitor (CRT/LCD)

Yes



Photo 26

Touch screen

Yes

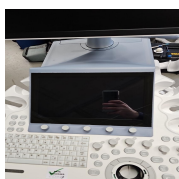


Photo 27

Console

Yes



Photo 28



Photo 29



Photo 30

Keyboard

Yes



Photo 31



Photo 32



Photo 33

Trackball / Optical mouse

Yes

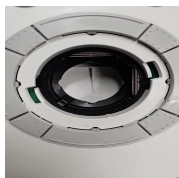


Photo 34

Probe Holder

Yes



Photo 35



Photo 36

Internal Cleaning

Yes



Photo 37

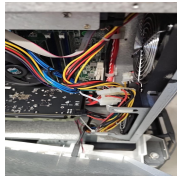


Photo 38



Photo 39



Photo 40

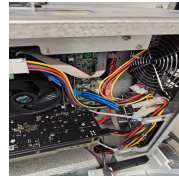


Photo 41



Photo 42



Photo 43



Photo 44

Filter

Yes



Photo 45

Shielding and Covers

Yes



Photo 46



Photo 47



Photo 48

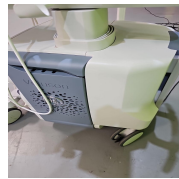


Photo 49



Photo 50



Photo 51



Photo 52



Photo 53

Thermal Printer Head

Yes

4. FUNCTIONAL CHECKS

Monitor

Pass

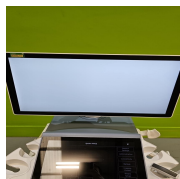


Photo 54



Photo 55

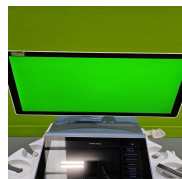


Photo 56

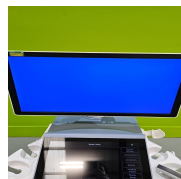


Photo 57

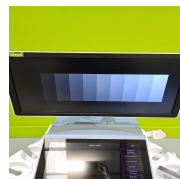


Photo 58

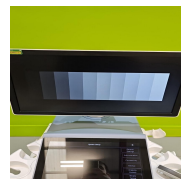


Photo 59

Touch Panel

Pass

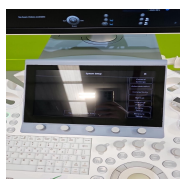


Photo 60



Photo 61

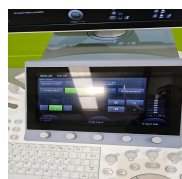


Photo 62

Console

Pass



Photo 63



Photo 64

Keyboard

Pass

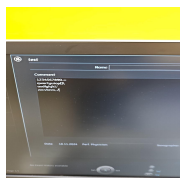


Photo 65

B-Mode

Pass



Photo 66

M-Mode

Pass



Photo 67



Photo 68

C-Mode

Pass



Photo 69

PD-Mode

Pass



Photo 70

PW-Mode

Pass



Photo 71



Photo 72

3D Mode

Pass

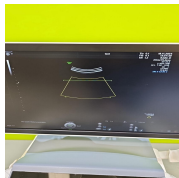


Photo 73

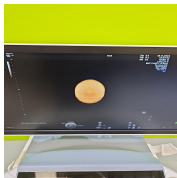


Photo 74

4D Mode

Pass



Photo 75



Photo 76

5. PROBE NO.: 1

Probe Make

GE

Probe Model

IC5-9-D

Probe Serial Number

855461WX0



Photo 77

Visual inspection

Pass

Cable

Pass



Photo 78

Lens

Pass

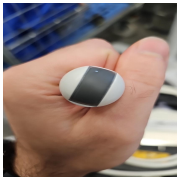


Photo 79

Probe Casing

Pass



Photo 80



Photo 81



Photo 82



Photo 83

Probe Casing Strain Relief

Pass



Photo 84

Connector

Pass



Photo 85



Photo 86

Connector Strain Relief

Pass



Photo 87

Lateral resolution

Pass



Photo 88

Axial resolution

Pass



Photo 89

Image benchmark

Pass



Photo 90



Photo 91

Probe passes inspection & test

Yes

Probe has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Yes

PROBE NO.: 2

Probe Make

GE

Probe Model

C2-9-D

Probe Serial Number

359840YP6



Photo 92

Visual inspection

Pass

Cable

Pass

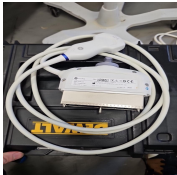


Photo 93

Lens

Pass



Photo 94

Probe Casing

Pass



Photo 95



Photo 96



Photo 97



Photo 98

Probe Casing Strain Relief

Pass



Photo 99

Connector

Pass



Photo 100



Photo 101

Connector Strain Relief

Pass



Photo 102

Lateral resolution

Pass

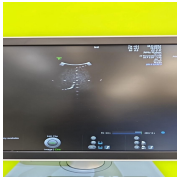


Photo 103

Axial resolution

Pass

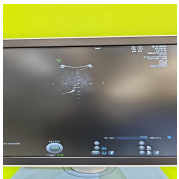


Photo 104

Image benchmark

Pass



Photo 105



Photo 106

Probe passes inspection & test

Yes

Probe has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Yes

PROBE NO.: 3

Probe Make

GE

Probe Model

RAB6-D

Probe Serial Number

862282WX1



Photo 107

Visual inspection

Pass

Cable

Pass



Photo 108

Lens

Pass



Photo 109

Probe Casing

Pass



Photo 110



Photo 111



Photo 112



Photo 113

Probe Casing Strain Relief

Pass



Photo 114

Connector

Pass



Photo 115



Photo 116

Connector Strain Relief

Pass



Photo 117

Lateral resolution

Pass

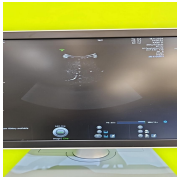


Photo 118

Axial resolution

Pass



Photo 119

Image benchmark

Pass



Photo 120



Photo 121

Probe passes inspection & test

Yes

Probe has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Yes

PROBE NO.: 4

Probe Make

GE

Probe Model

C1-6-D

Probe Serial Number

425141YP9



Photo 122

Visual inspection

Pass

Cable

Pass



Photo 123

Lens

Pass

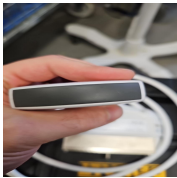


Photo 124

Probe Casing

Pass



Photo 125



Photo 126



Photo 127



Photo 128

Probe Casing Strain Relief

Pass



Photo 129

Connector

Pass



Photo 130



Photo 131

Connector Strain Relief

Pass



Photo 132

Lateral resolution

Pass

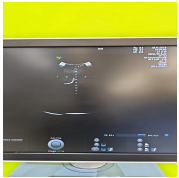


Photo 133

Axial resolution

Pass

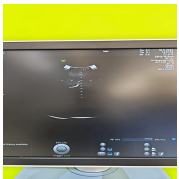


Photo 134

Image benchmark

Pass



Photo 135



Photo 136

Probe passes inspection & test

Yes

Probe has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Yes

6. SUMMARY

This service has been completed in accordance with our quality procedure.
If you have any questions or concerns, please contact us on
+44 1252 299024
or email: enquiries@britishmedicalauctions.co.uk to discuss further.

System has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Safe

Date Performed:

18.11.2024 14:42 CET

Service Engineer Signature

MS
18.11.2024 14:42 CET

British Medical Auctions
Unit 1, Sterling Centre, Eastern Road, Bracknell, RG12 2PW
Tel: +44 1252 299024
Email: enquiries@britishmedicalauctions.co.uk
Web: www.britishmedicalauctions.co.uk

British Medical Auctions
Ultrasound Equipment Quality Check (EQC)
Preventive Maintenance Service Report (PM Service)

V1.0 05.2022

7. PHOTOGRAPHIC DOCUMENTATION



Photo 137



Photo 138

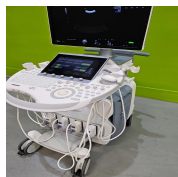


Photo 139



Photo 140

Media summary



Photo 1



Photo 2

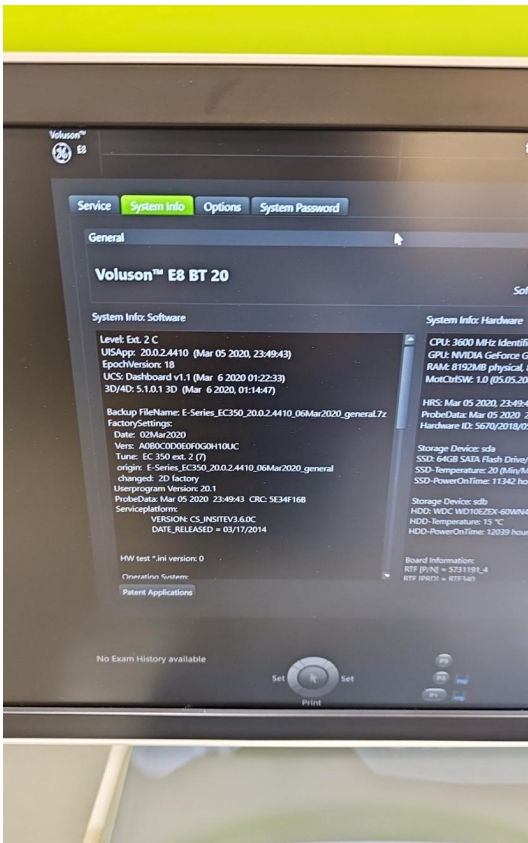


Photo 3

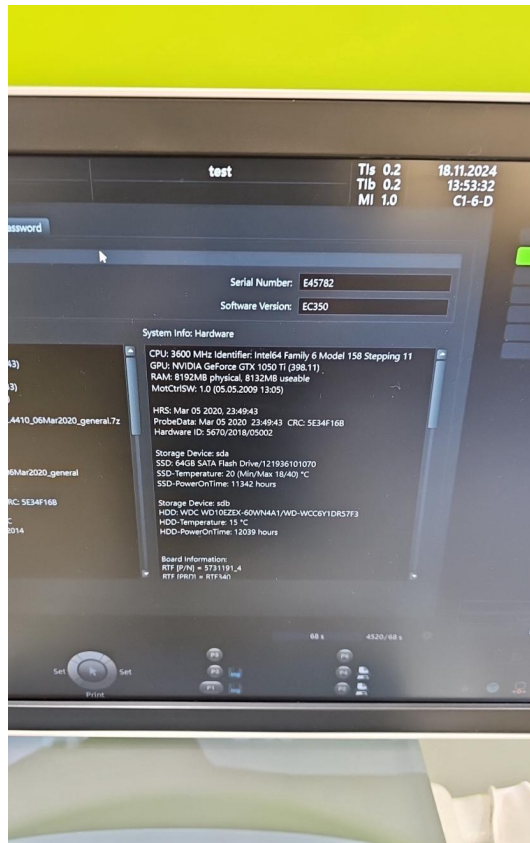


Photo 4

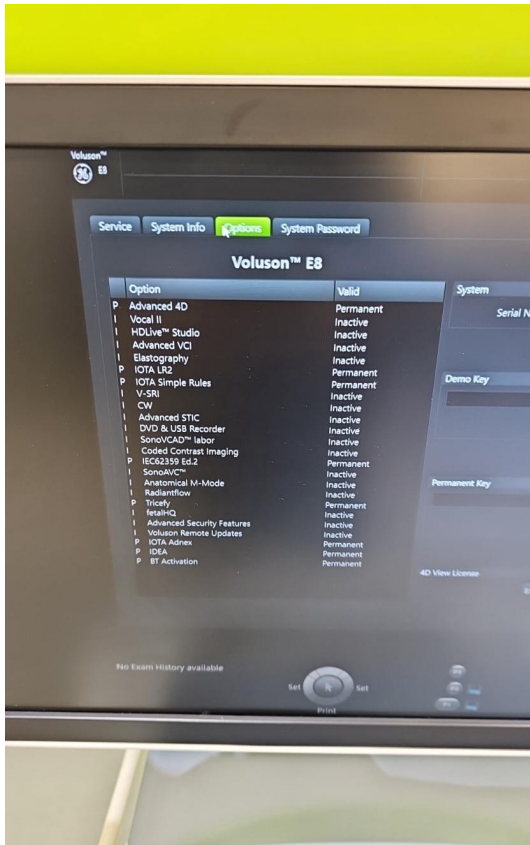


Photo 5

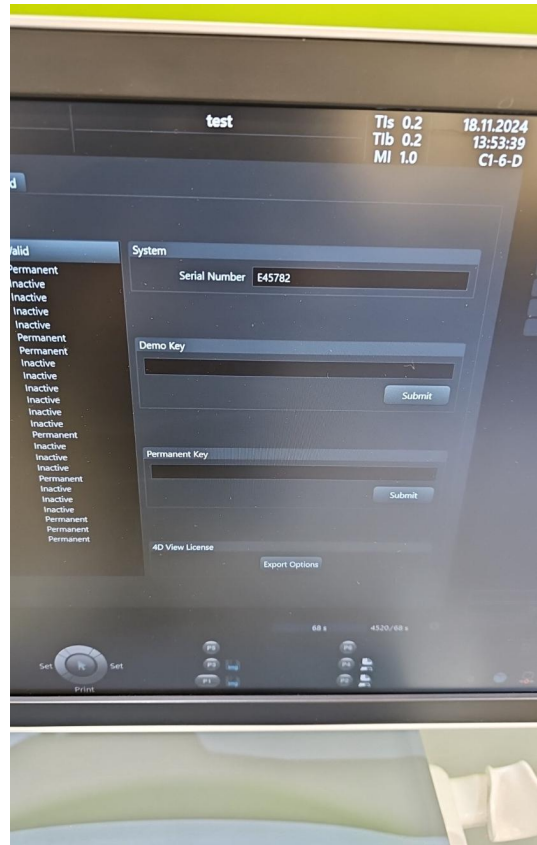


Photo 6



Photo 7

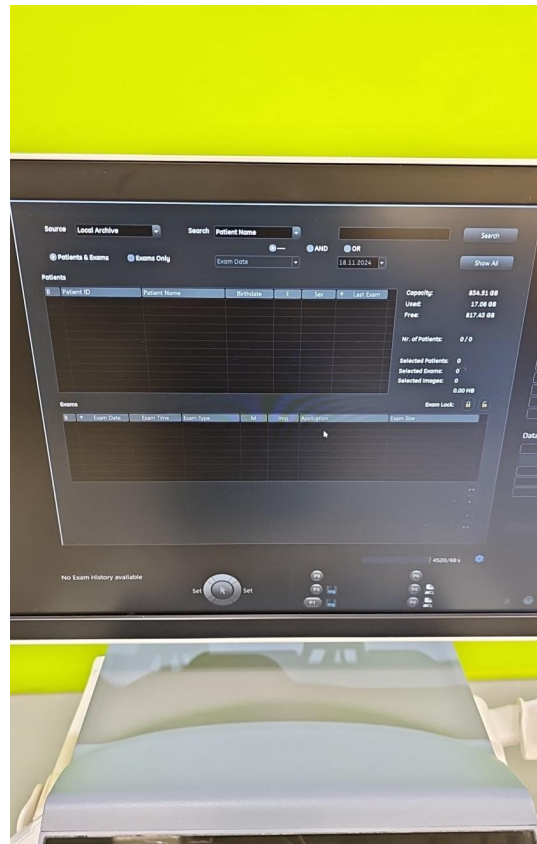


Photo 8

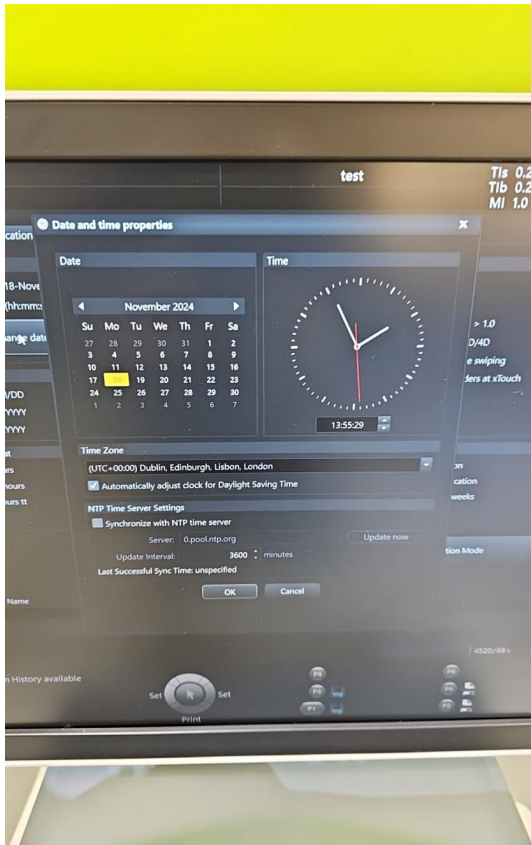


Photo 9



Photo 10



Photo 11



Photo 12



Photo 13



Photo 14



Photo 15



Photo 16



Photo 17



Photo 18



Photo 19



Photo 20



Photo 21



Photo 22



Photo 23



Photo 24



Photo 25



Photo 26

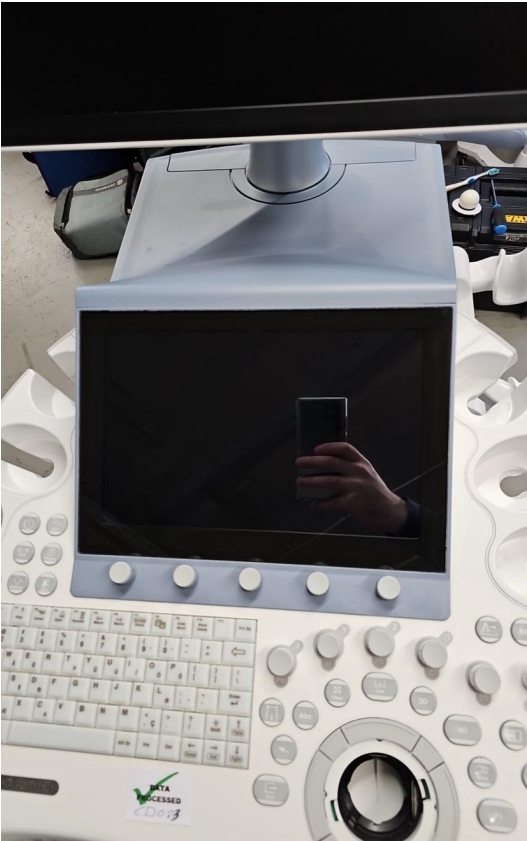


Photo 27



Photo 28



Photo 29

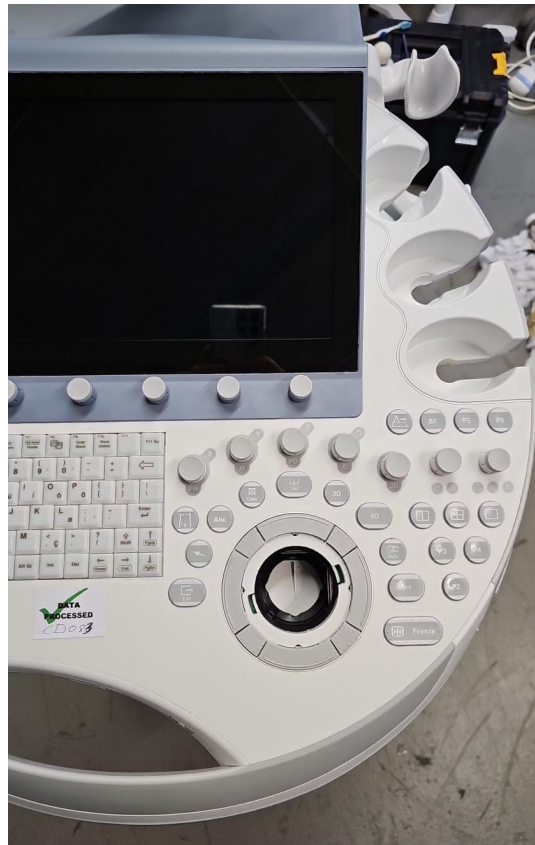


Photo 30

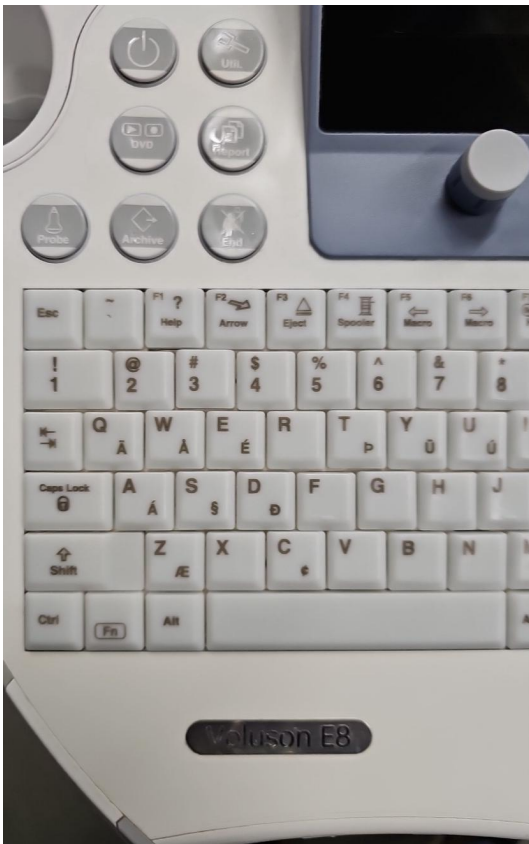


Photo 31

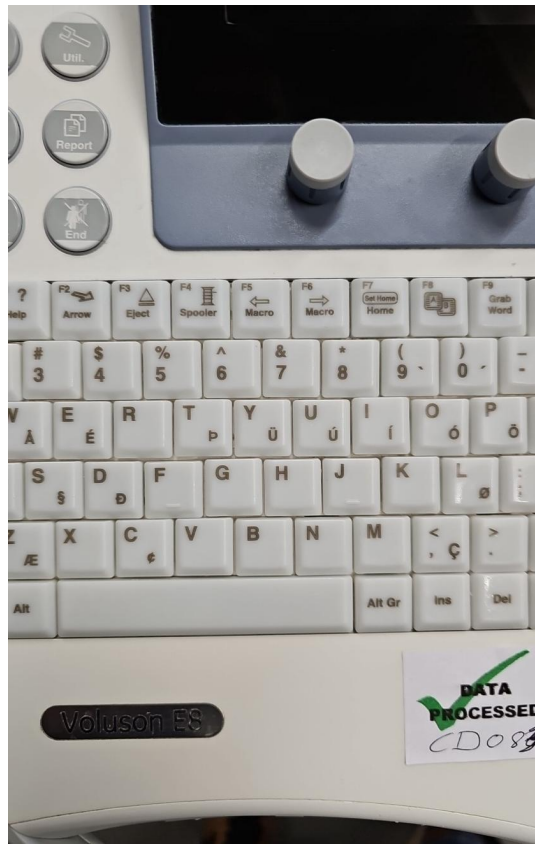


Photo 32

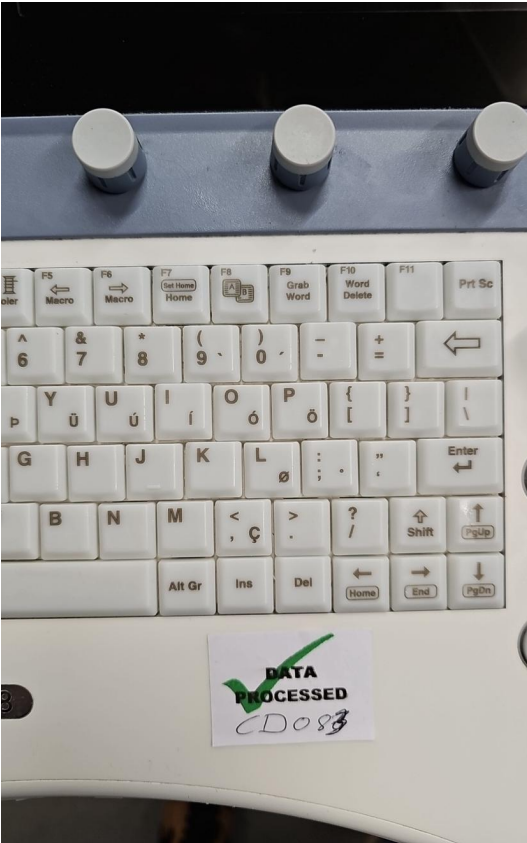


Photo 33

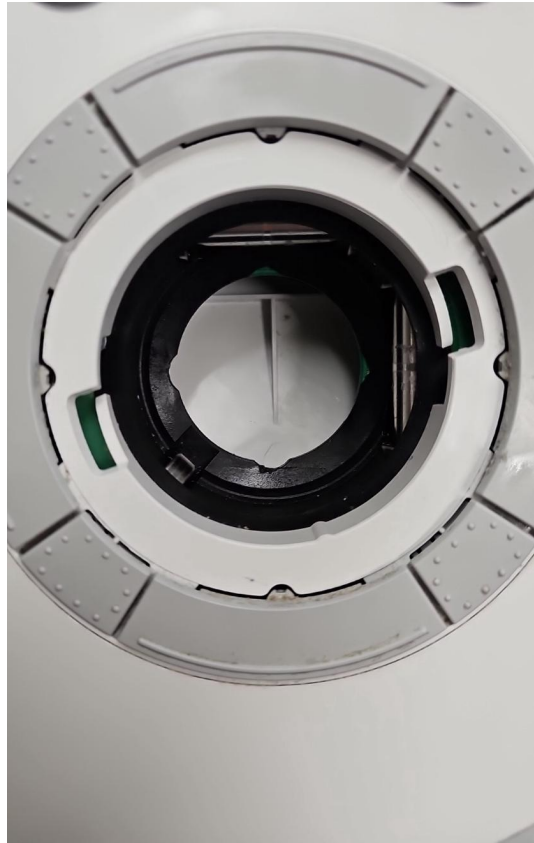


Photo 34



Photo 35



Photo 36

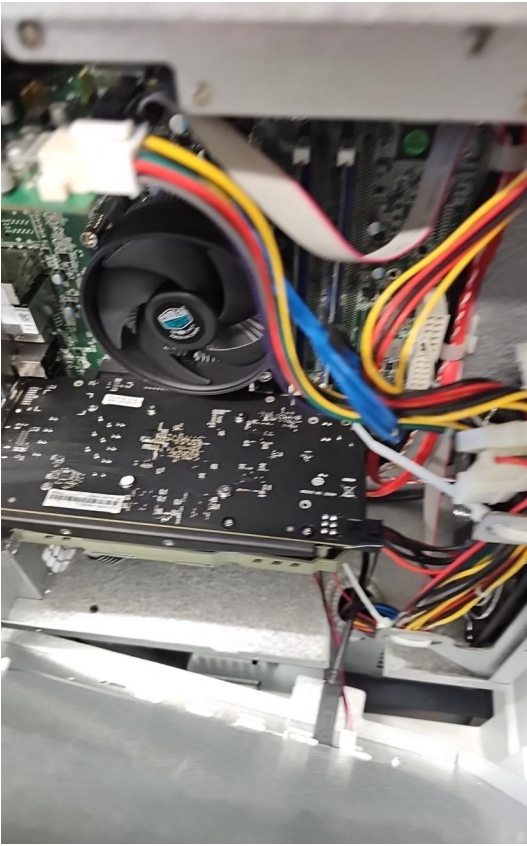


Photo 37

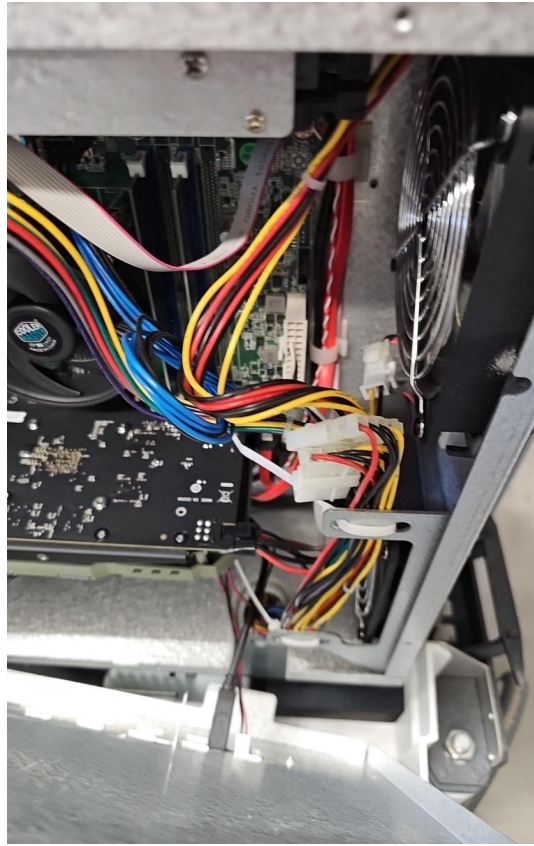


Photo 38

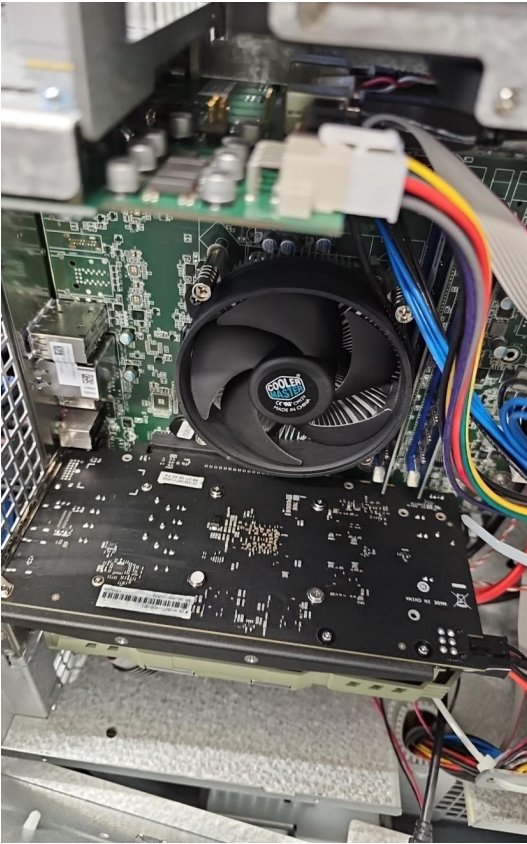


Photo 39



Photo 40

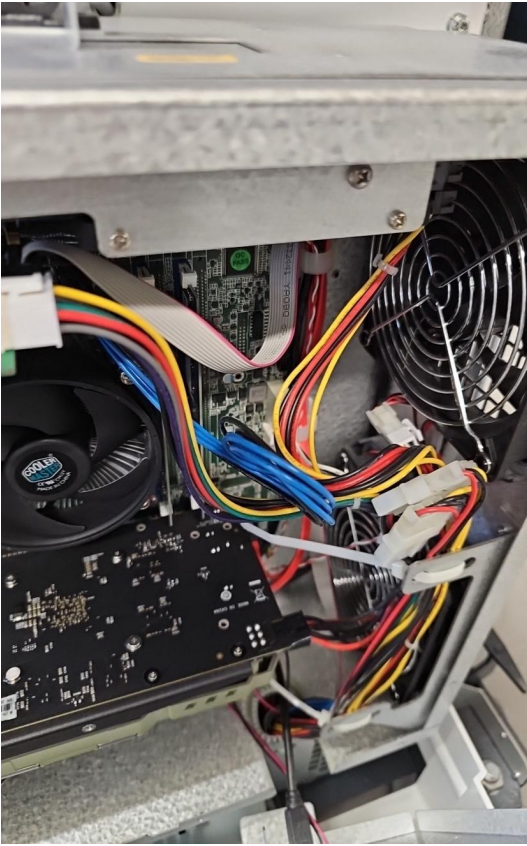


Photo 41



Photo 42



Photo 43



Photo 44



Photo 45



Photo 46



Photo 47



Photo 48



Photo 49



Photo 50



Photo 51

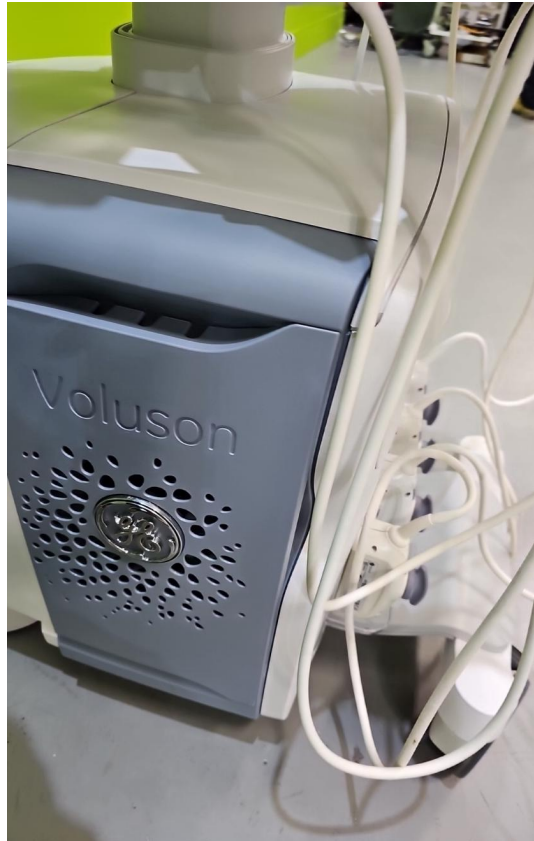


Photo 52



Photo 53

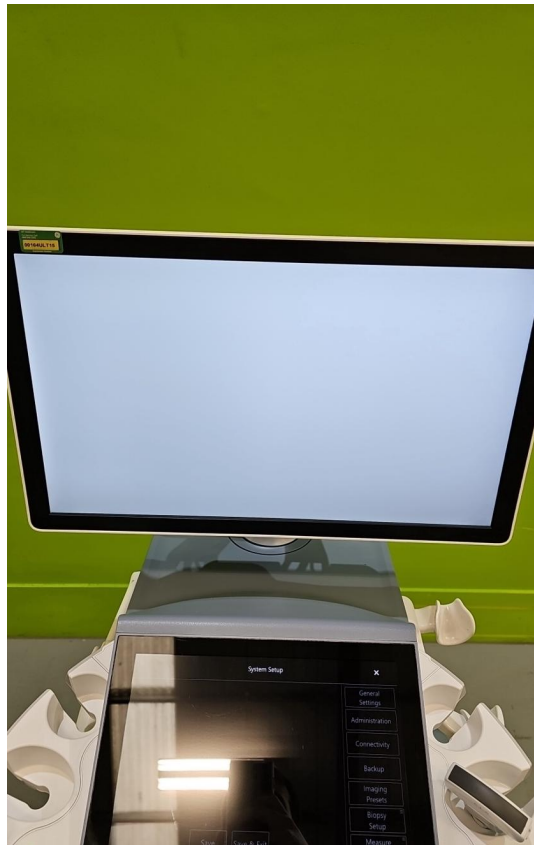


Photo 54



Photo 55

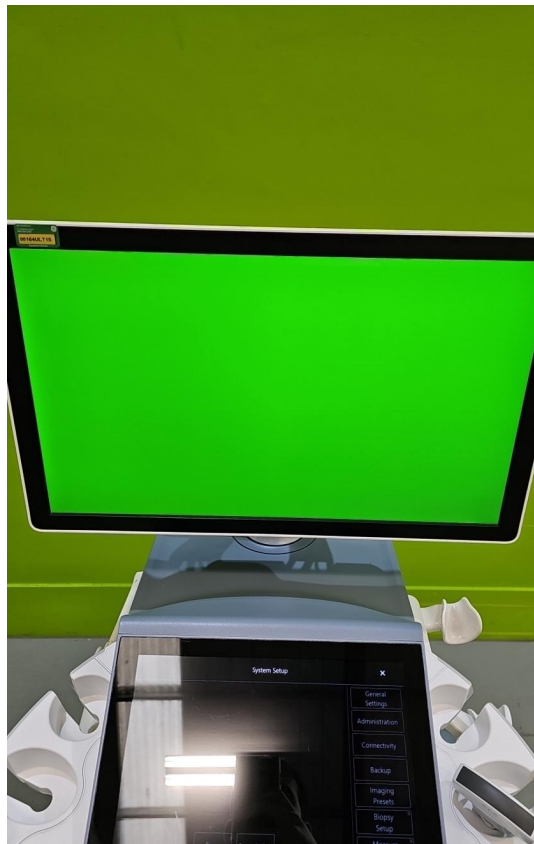


Photo 56



Photo 57

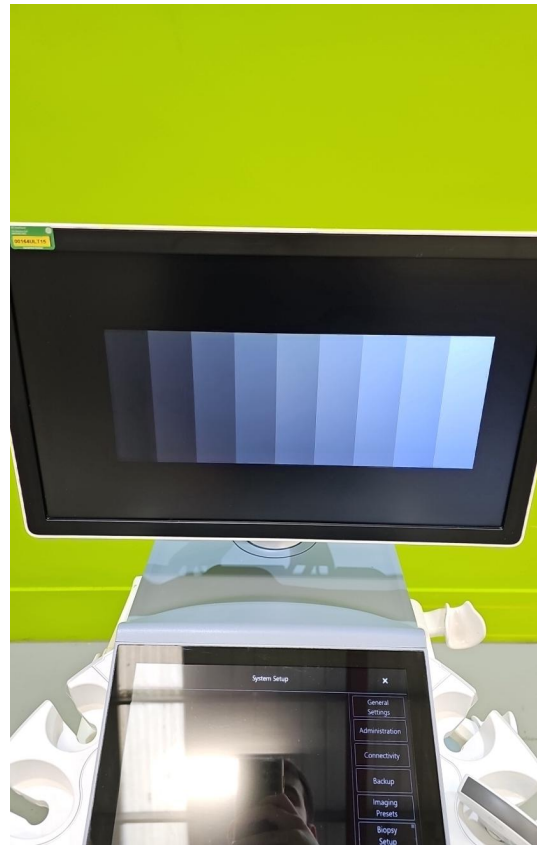


Photo 58

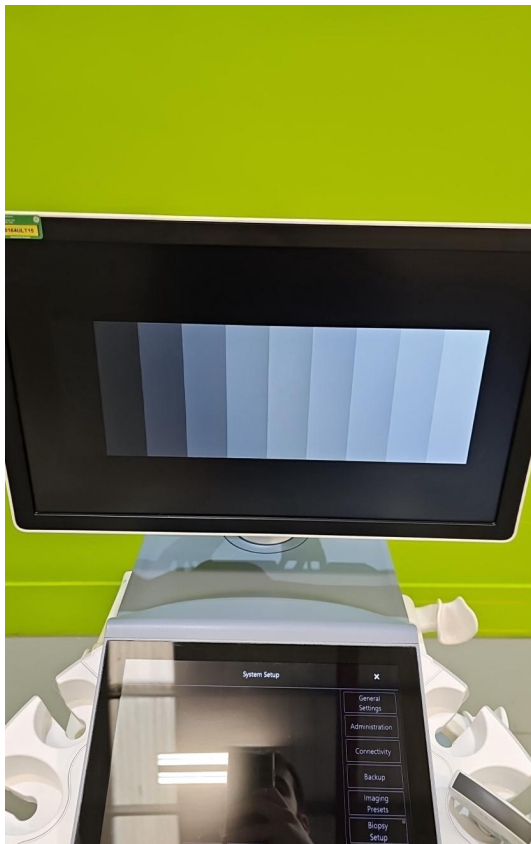


Photo 59

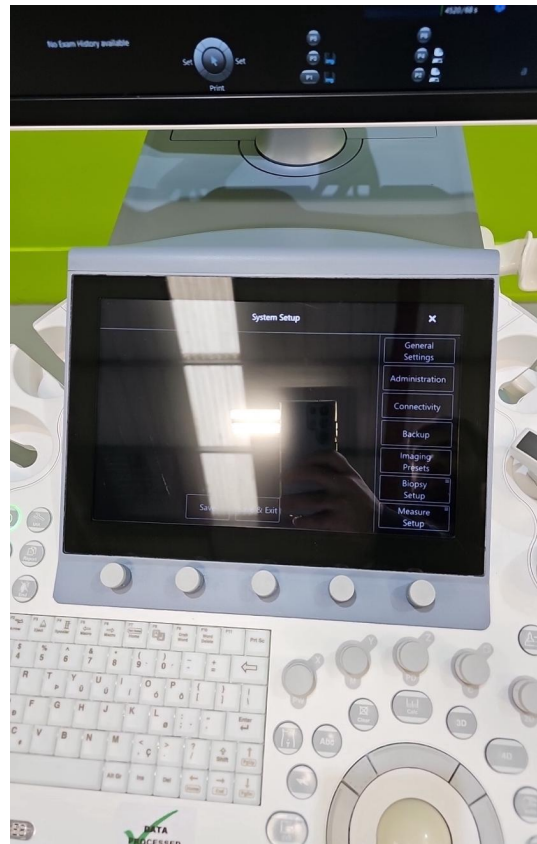


Photo 60



Photo 61



Photo 62



Photo 63



Photo 64

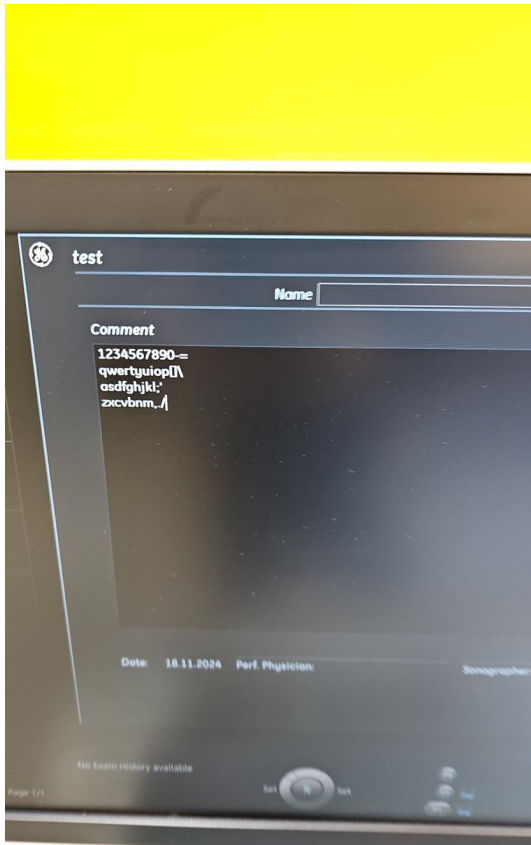


Photo 65

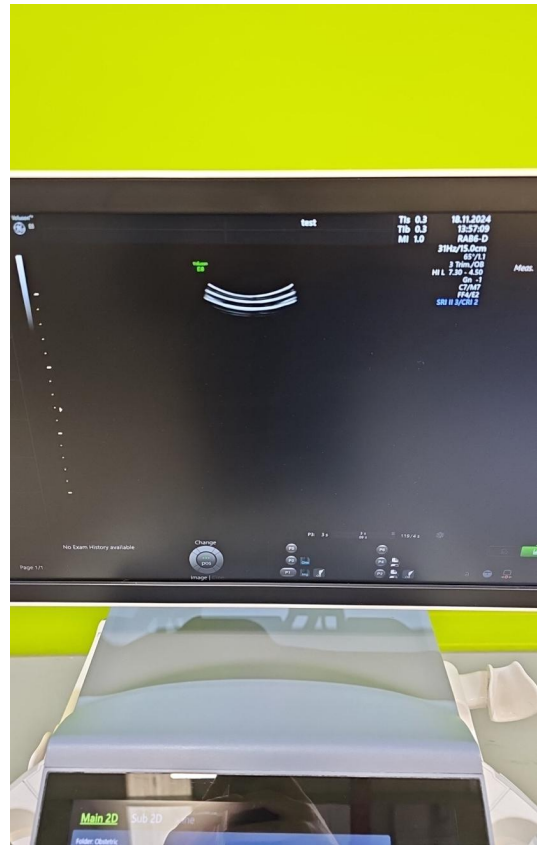


Photo 66

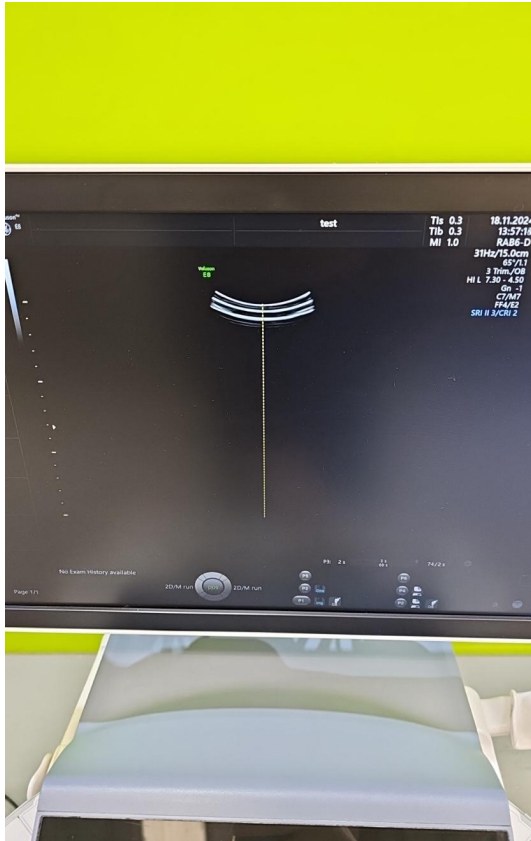


Photo 67

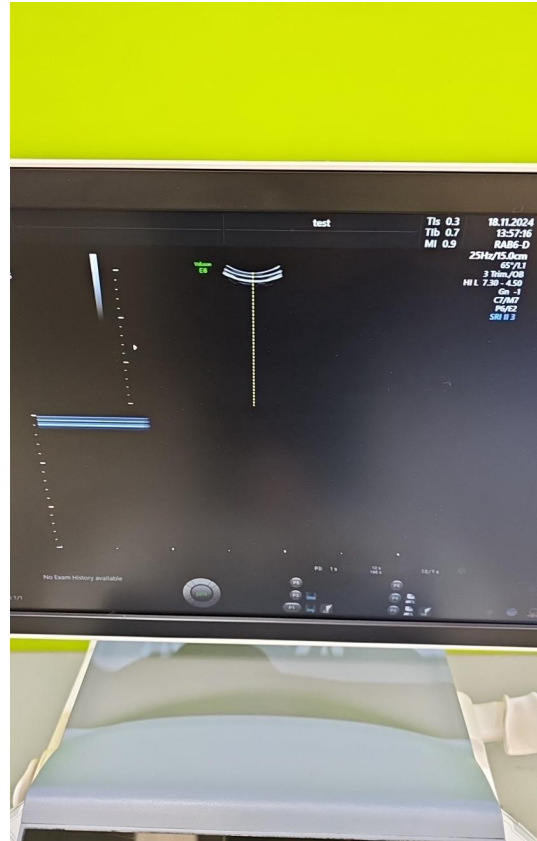


Photo 68

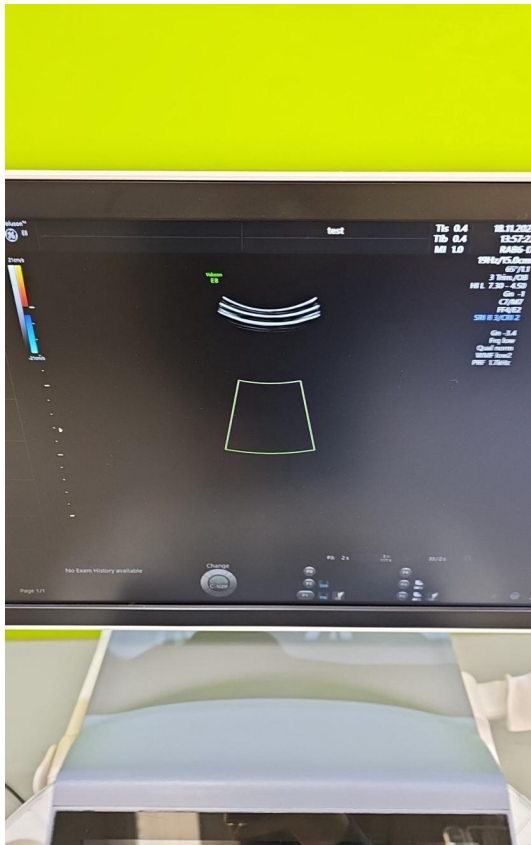


Photo 69



Photo 70

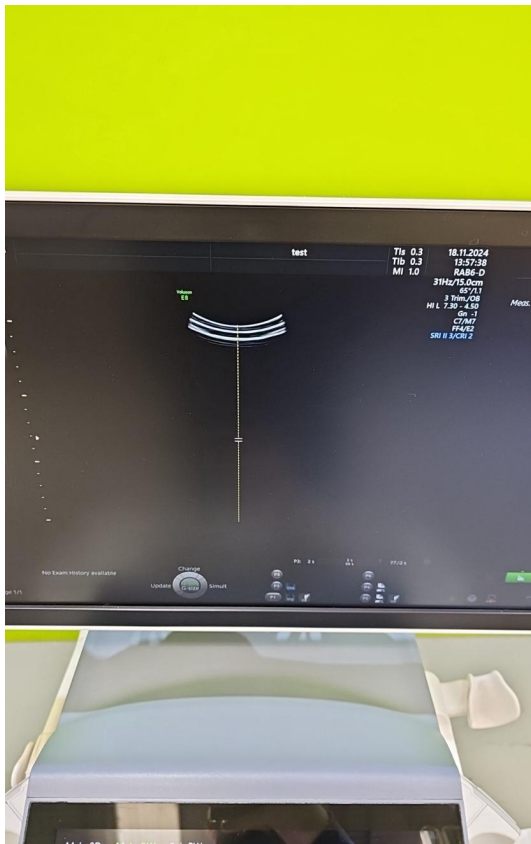


Photo 71

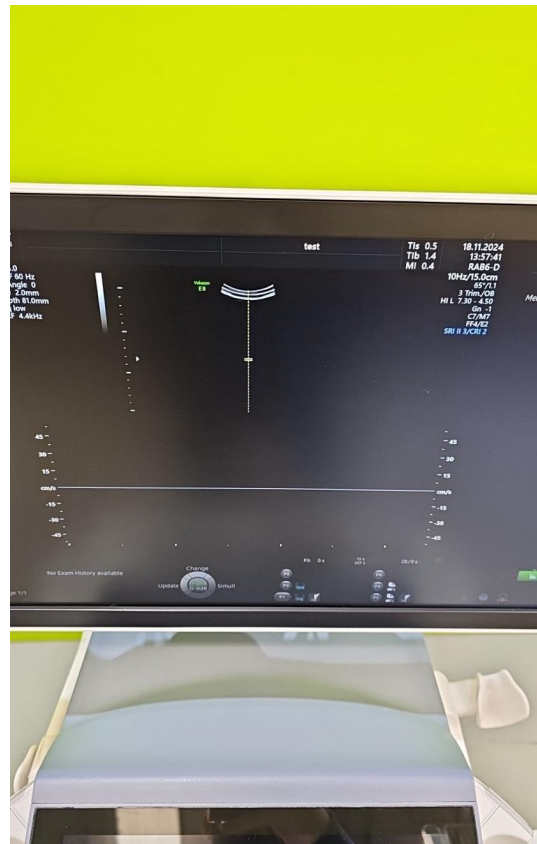


Photo 72



Photo 73

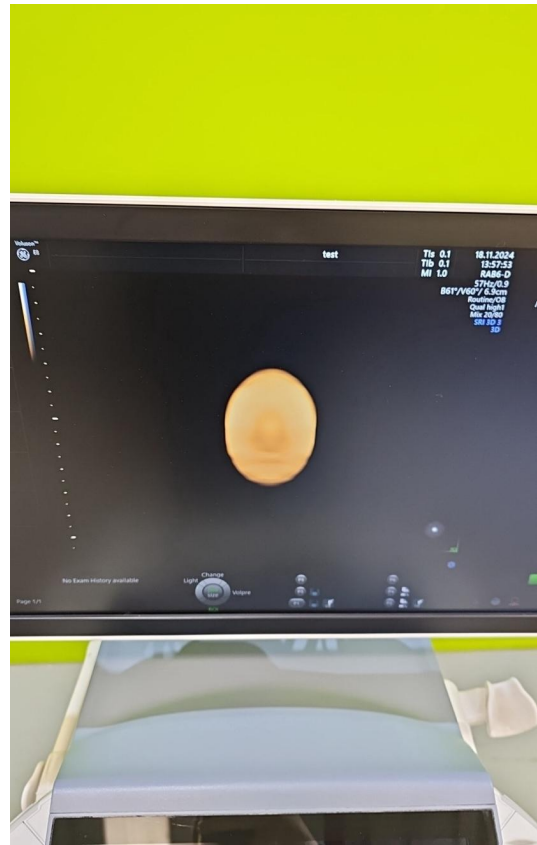


Photo 74



Photo 75



Photo 76



Photo 77



Photo 78



Photo 79



Photo 80



Photo 81



Photo 82



Photo 83



Photo 84



Photo 85



Photo 86



Photo 87

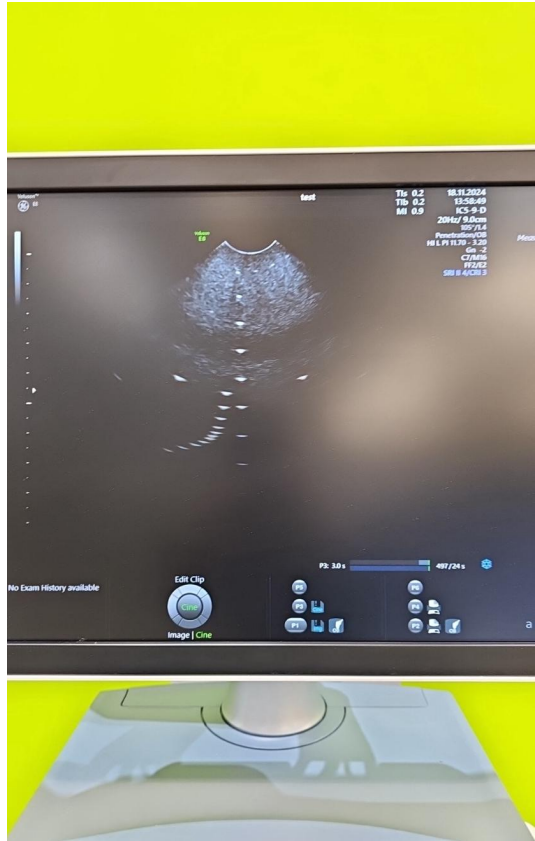


Photo 88



Photo 89

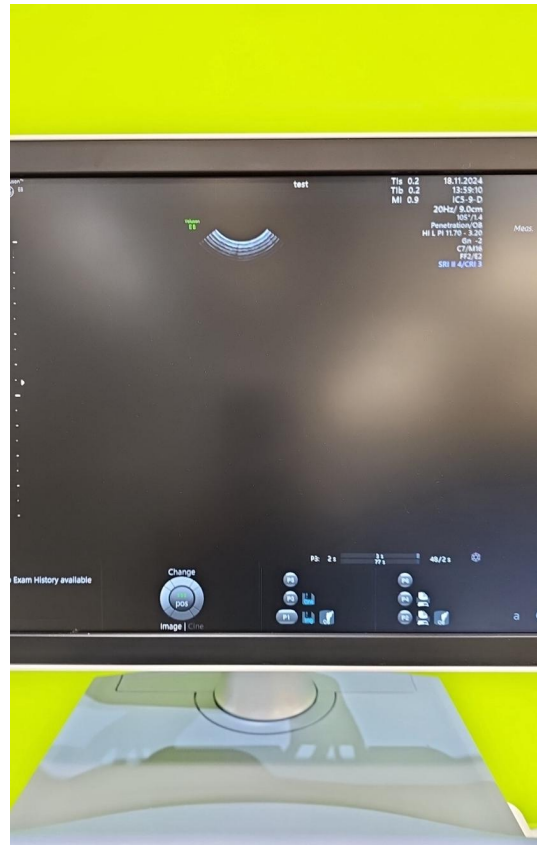


Photo 90



Photo 91



Photo 92



Photo 93



Photo 94

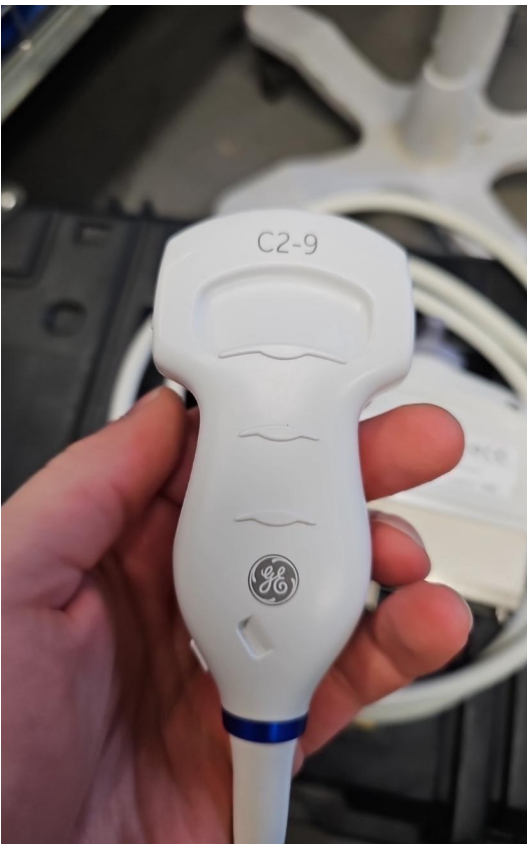


Photo 95



Photo 96



Photo 97



Photo 98



Photo 99



Photo 100

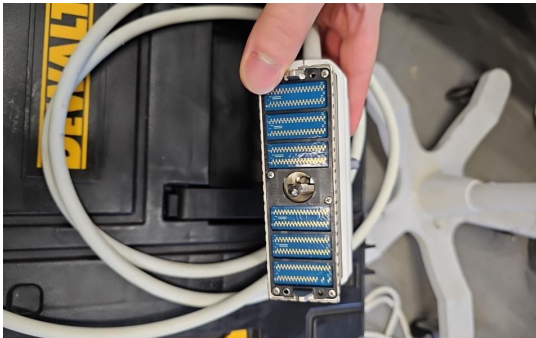


Photo 101



Photo 102

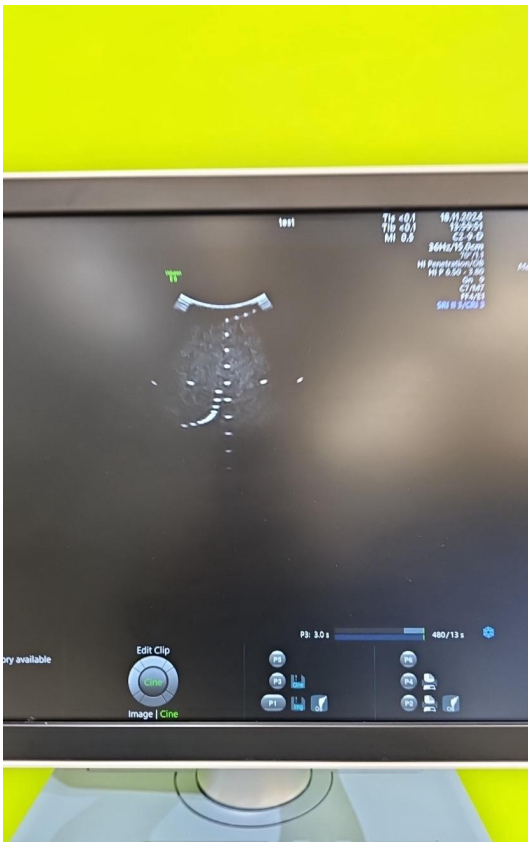


Photo 103

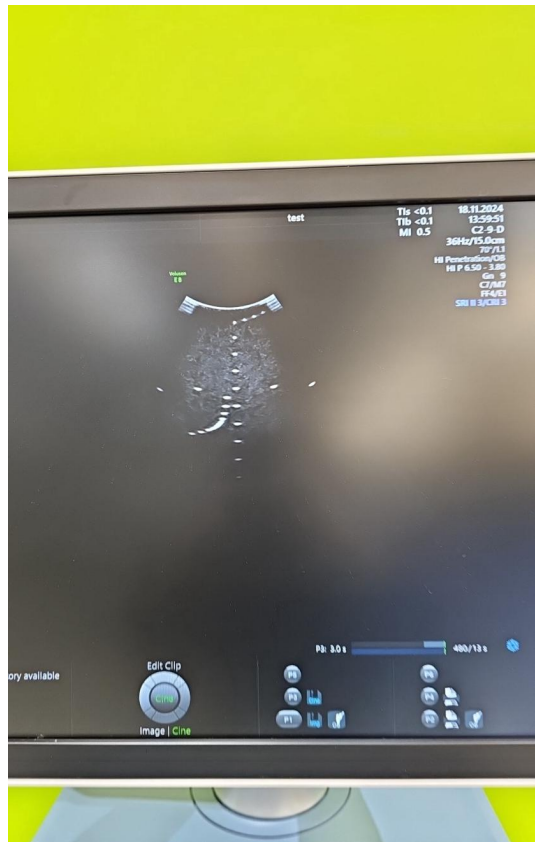


Photo 104

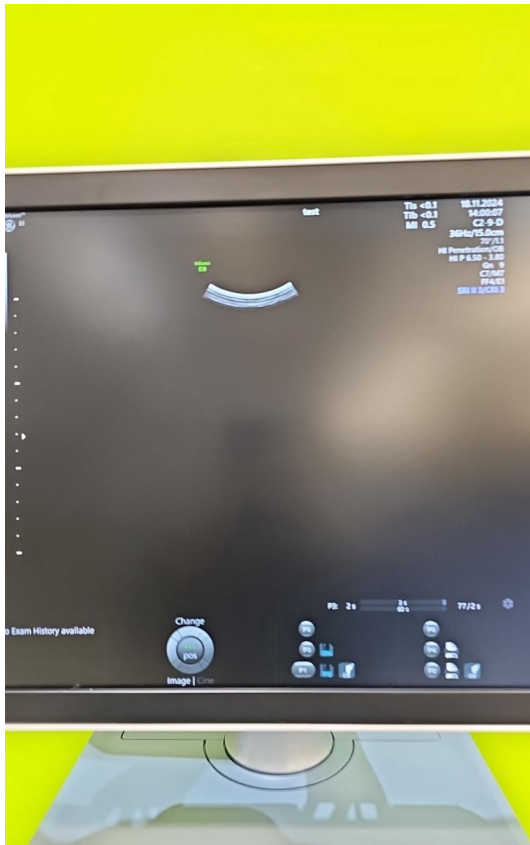


Photo 105

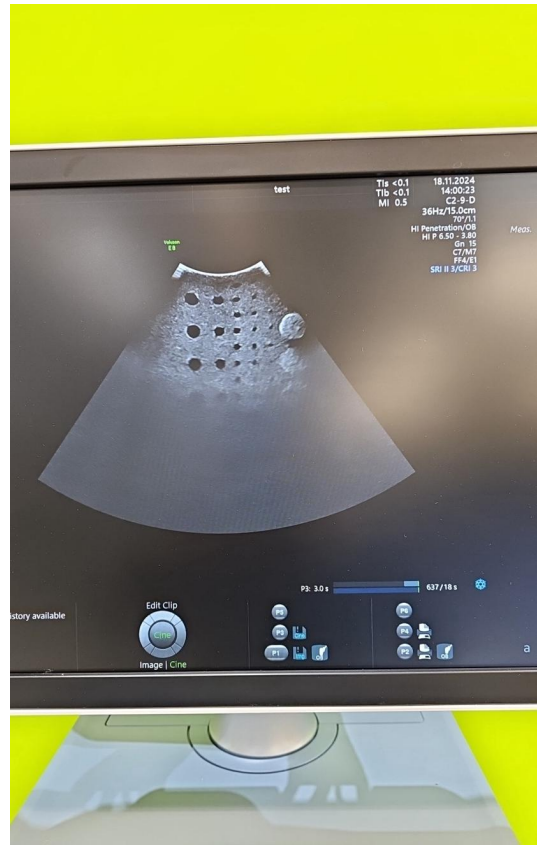


Photo 106

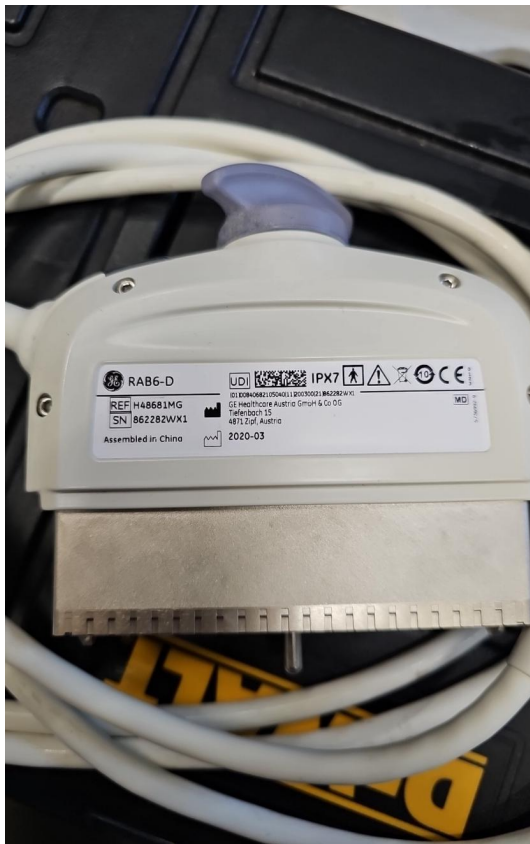


Photo 107



Photo 108



Photo 109



Photo 110



Photo 111



Photo 112



Photo 113



Photo 114



Photo 115



Photo 116



Photo 117

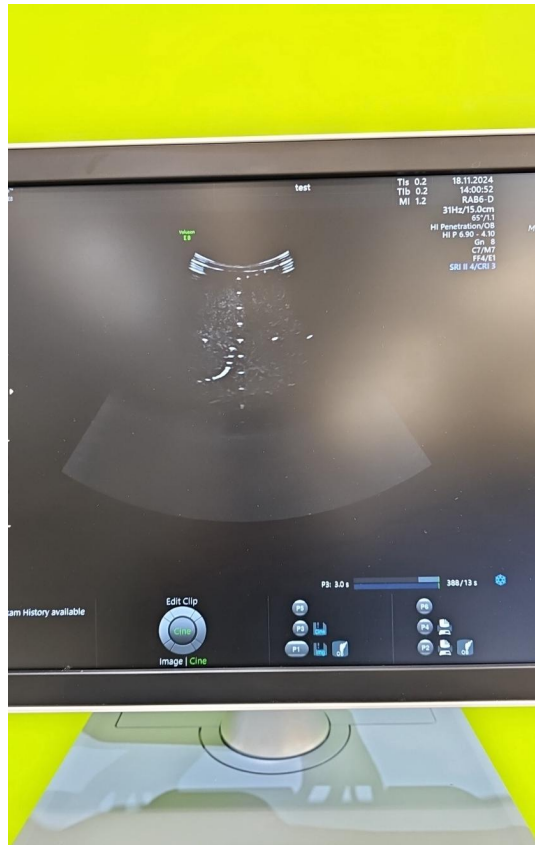


Photo 118



Photo 119

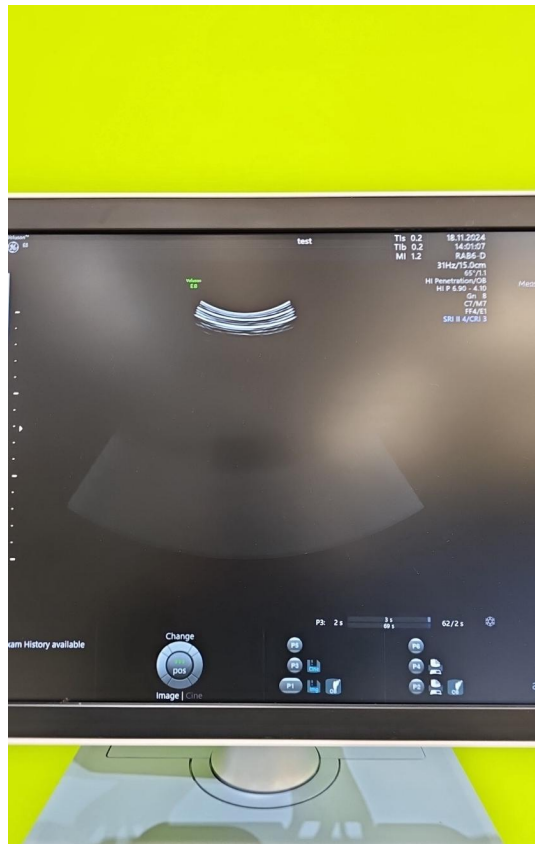


Photo 120

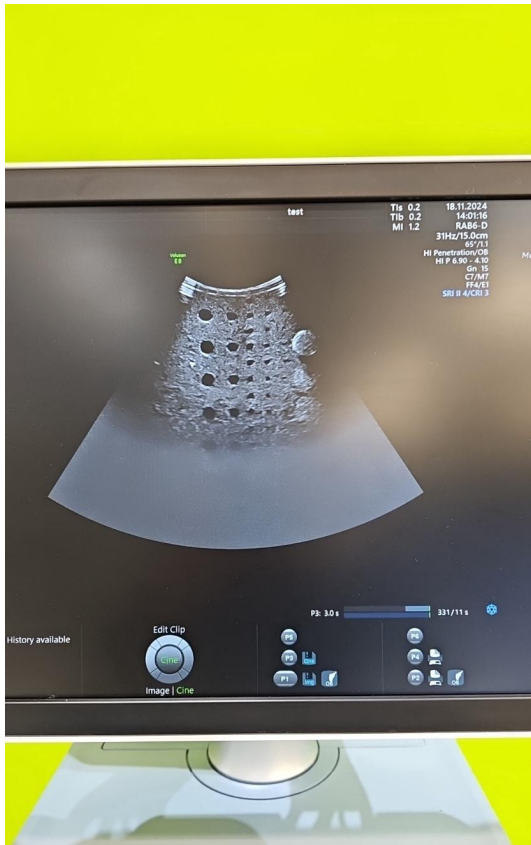


Photo 121



Photo 122



Photo 123



Photo 124



Photo 125



Photo 126

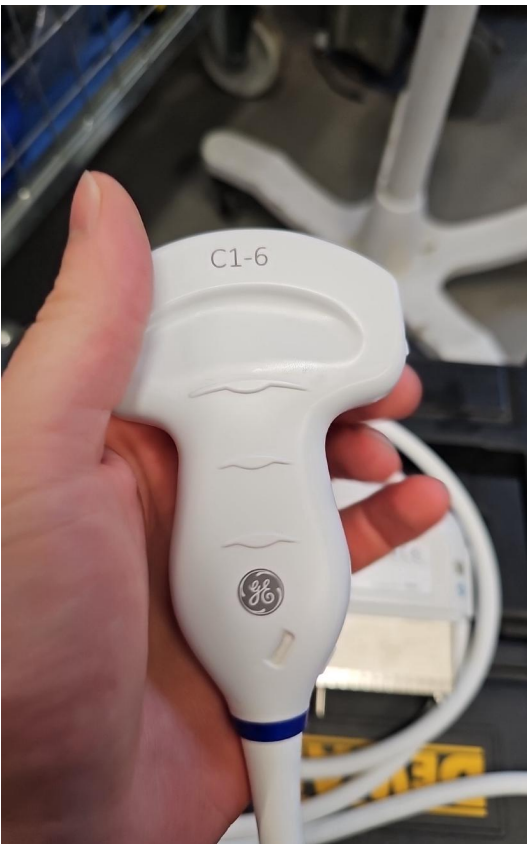


Photo 127



Photo 128



Photo 129



Photo 130



Photo 131



Photo 132

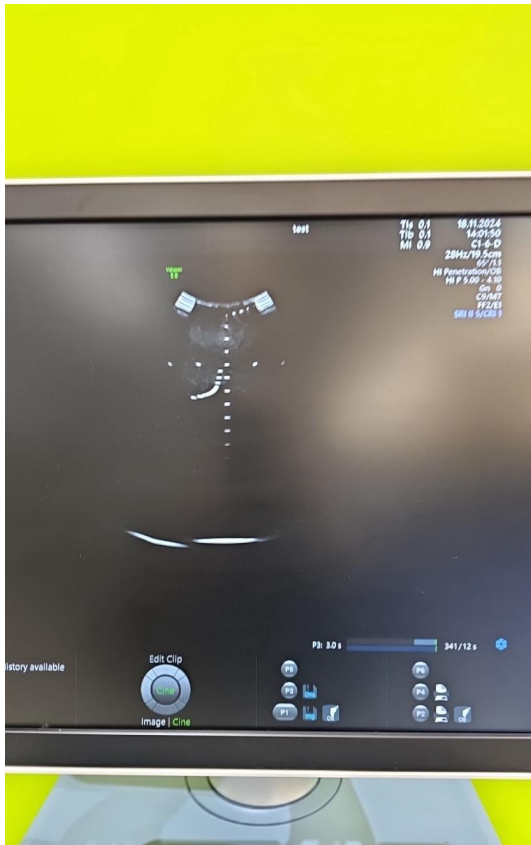


Photo 133

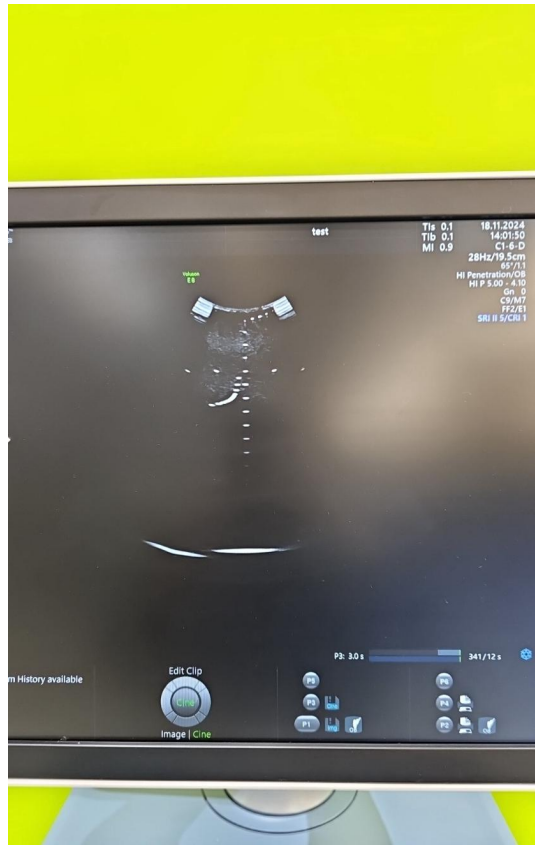


Photo 134

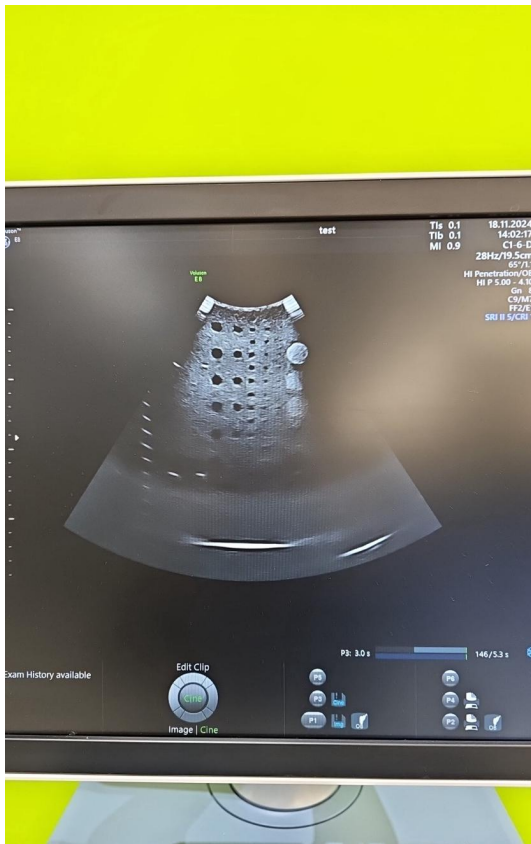


Photo 135

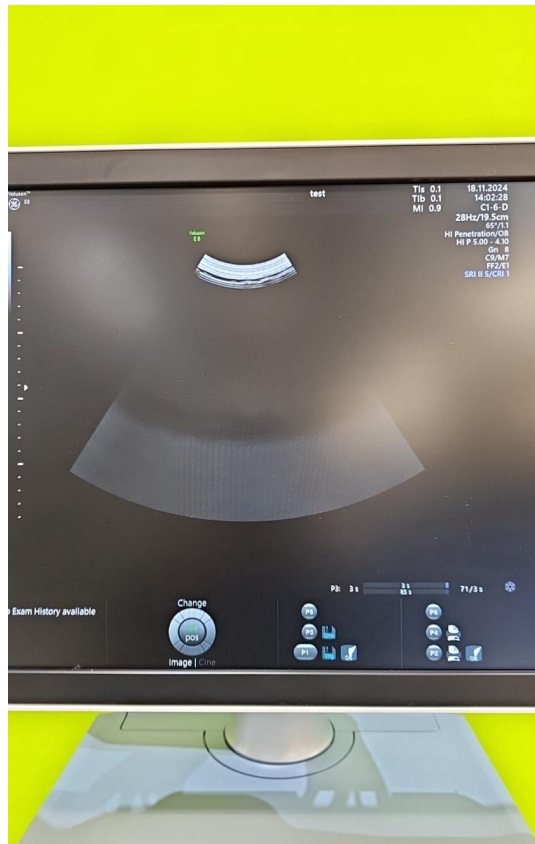


Photo 136



Photo 137



Photo 138



Photo 139

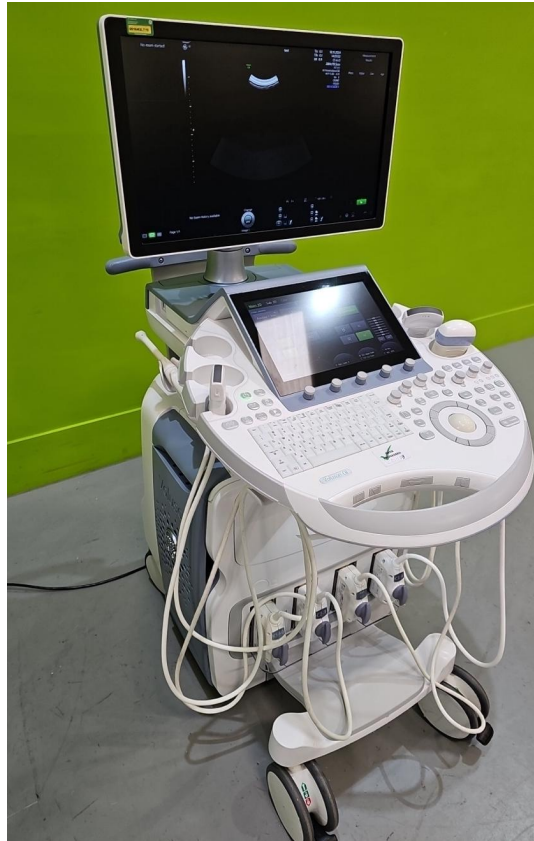


Photo 140