



British Medical Auctions Ultrasound Equipment Quality Check (EQC) Preventive Maintenance Service Report (PM Service)

GE / Voluson E8 / E43194

Complete

British Medical Auctions

Address: Unit 1, Sterling Centre, Eastern Road, Bracknell, RG12 2PW
Tel: +44 1252 299024
Email: enquiries@britishmedicalauctions.co.uk
Web: www.britishmedicalauctions.co.uk

Department: Service

System has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Safe

System Type:

Ultrasound System



Photo 1

System Manufacture:

GE

Model:

Voluson E8

Serial No.:

E43194



Photo 2

Year of Manufacture:

09.2019

Software Level:

BT19 EC340

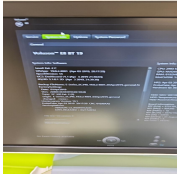


Photo 3

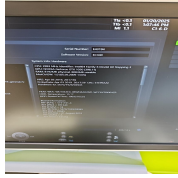


Photo 4

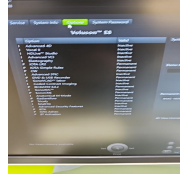


Photo 5

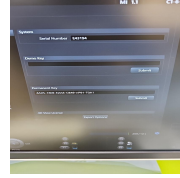


Photo 6

Service Code:

UK739

Electrical Safety Test Asset ID:

UK762

Test Equipment No. (Electrical Safety Analyzer):
Rigel 288+
SN.:10M-0263

Calibration Due Date:
05.2025

Test Equipment No. (Ultrasound Phantom):
ATS Laboratories MODEL: 539
Multipurpose Ultrasound Phantom
SN.: 200493148

Date Performed

20.01.2025 13:42 CET

Performed By

MS

1. PRELIMINARY CHECKS

Power Up

Yes



Photo 7

Database Cleared (Including all Patient Data and Previous Owner Details)

Yes

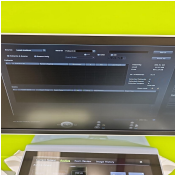


Photo 8

Password Protection

No

Time and Date Check

Yes



Photo 9

2. PHYSICAL INSPECTION

Input Power

Pass



Photo 10

External I/O

Pass

Wheels and Brakes

Pass



Photo 11



Photo 12



Photo 13



Photo 14

Control Panel Movement

Pass



Photo 15



Photo 16



Photo 17



Photo 18

Cables and Connectors

Pass

Shielding and Covers

Pass



Photo 19



Photo 20



Photo 21



Photo 22



Photo 23

Console

Pass



Photo 24



Photo 25

Control Panel

Pass



Photo 26

Monitor Arm

Pass



Photo 27

3. CLEANING

Monitor (CRT/LCD)

Yes



Photo 28

Touch screen

Yes



Photo 29

Console

Yes



Photo 30



Photo 31



Photo 32

Keyboard

Yes

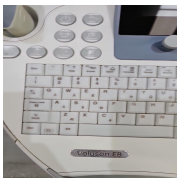


Photo 33

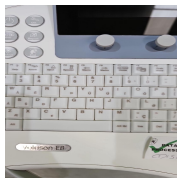


Photo 34

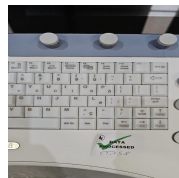


Photo 35

Trackball / Optical mouse

Yes



Photo 36

Probe Holder

Yes

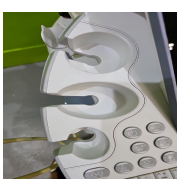


Photo 37



Photo 38

Internal Cleaning

Yes



Photo 39

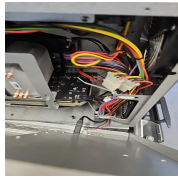


Photo 40

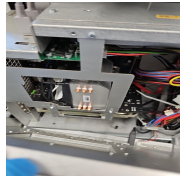


Photo 41

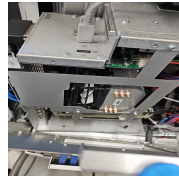


Photo 42



Photo 43



Photo 44

Filter

Yes



Photo 45

Shielding and Covers

Yes



Photo 46



Photo 47



Photo 48

Thermal Printer Head

Yes

4. FUNCTIONAL CHECKS

Monitor

Pass



Photo 49

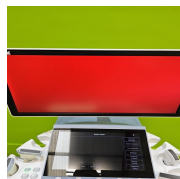


Photo 50

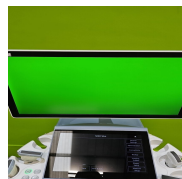


Photo 51

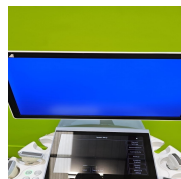


Photo 52



Photo 53

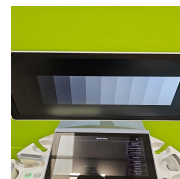


Photo 54

Touch Panel

Pass

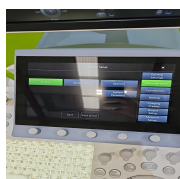


Photo 55



Photo 56

Console

Pass



Photo 57



Photo 58

Keyboard

Pass

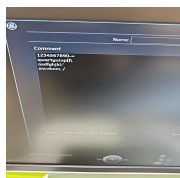


Photo 59

B-Mode

Pass

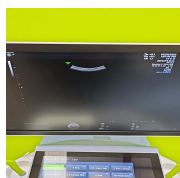


Photo 60

M-Mode

Pass



Photo 61



Photo 62

C-Mode

Pass



Photo 63

PD-Mode

Pass



Photo 64

PW-Mode

Pass



Photo 65

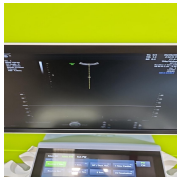


Photo 66

5. PROBE NO.: 1

Probe Make

GE

Probe Model

C1-6-D

Probe Serial Number

456436YP5



Photo 67

Visual inspection

Fail

Cable

Pass



Photo 68

Lens

Pass

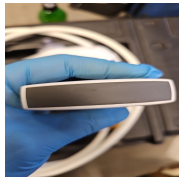


Photo 69

Probe Casing

Pass



Photo 70



Photo 71



Photo 72



Photo 73

Probe Casing Strain Relief

Fail



Photo 74



Photo 75



Photo 76

Connector

Pass



Photo 77



Photo 78

Connector Strain Relief

Pass



Photo 79

Lateral resolution

Pass

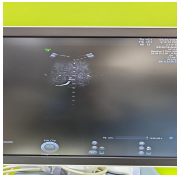


Photo 80

Axial resolution

Pass

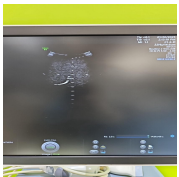


Photo 81

Image benchmark

Pass

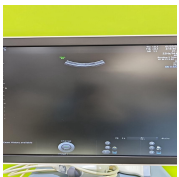


Photo 82



Photo 83

Probe passes inspection & test

No

Probe has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

No

PROBE NO.: 2

Probe Make

GE

Probe Model

C2-9-D

Probe Serial Number

331456YP4



Photo 84

Visual inspection

Pass

Cable

Pass



Photo 85

Lens

Pass



Photo 86

Probe Casing

Pass



Photo 87



Photo 88



Photo 89



Photo 90

Probe Casing Strain Relief

Pass



Photo 91

Connector

Pass



Photo 92

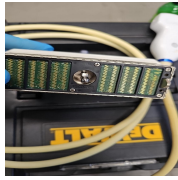


Photo 93

Connector Strain Relief

Pass



Photo 94

Lateral resolution

Pass



Photo 95

Axial resolution

Pass

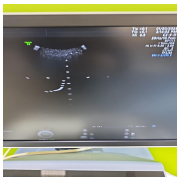


Photo 96

Image benchmark

Pass



Photo 97



Photo 98

Probe passes inspection & test

Yes

Probe has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Yes

PROBE NO.: 3

Probe Make

GE

Probe Model

ML6-15-D

Probe Serial Number

227521WP2



Photo 99

Visual inspection

Pass

Cable

Pass



Photo 100

Lens

Pass



Photo 101

Probe Casing

Pass



Photo 102



Photo 103



Photo 104



Photo 105

Probe Casing Strain Relief

Pass



Photo 106

Connector

Pass

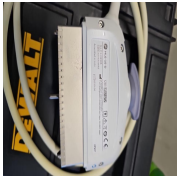


Photo 107

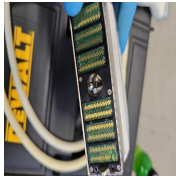


Photo 108

Connector Strain Relief

Pass



Photo 109

Lateral resolution

Pass

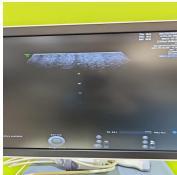


Photo 110

Axial resolution

Pass

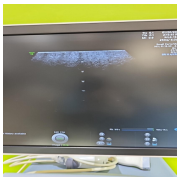


Photo 111

Image benchmark

Pass

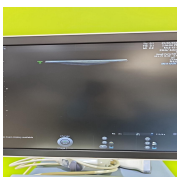


Photo 112



Photo 113

Probe passes inspection & test

Yes

Probe has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Yes

PROBE NO.: 4

Probe Make

GE

Probe Model

9L-D

Probe Serial Number

217130WP4 / 115830WP2



Photo 114

Visual inspection

Pass

Cable

Pass



Photo 115



Photo 116

Lens

Pass



Photo 117



Photo 118

Probe Casing

Pass



Photo 119



Photo 120



Photo 121



Photo 122



Photo 123



Photo 124



Photo 125



Photo 126

Probe Casing Strain Relief

Pass

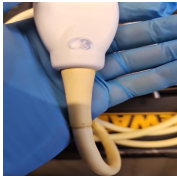


Photo 127

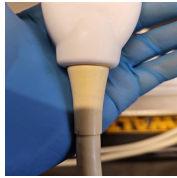


Photo 128

Connector

Pass



Photo 129

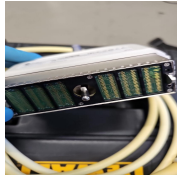


Photo 130



Photo 131

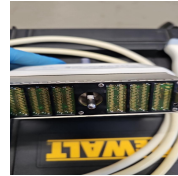


Photo 132

Connector Strain Relief

Pass



Photo 133



Photo 134

Lateral resolution

Pass

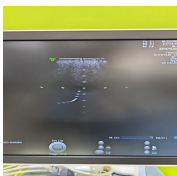


Photo 135



Photo 136

Axial resolution

Pass

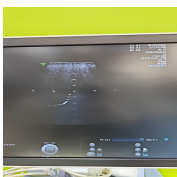


Photo 137



Photo 138

Image benchmark

Pass

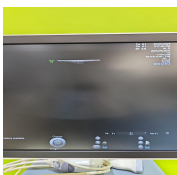


Photo 139



Photo 140



Photo 141



Photo 142

Probe passes inspection & test

Yes

Probe has passed performance and safety assessments and

Yes

is issued as fit for use, subject to final acceptance assessment upon fit.



6. SUMMARY

This service has been completed in accordance with our quality procedure.
If you have any questions or concerns, please contact us on
+44 1252 299024
or email: enquiries@britishmedicalauctions.co.uk to discuss further.

System has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Safe

Date Performed:

20.01.2025 15:15 CET

Service Engineer Signature

MS
20.01.2025 15:15 CET

British Medical Auctions
Unit 1, Sterling Centre, Eastern Road, Bracknell, RG12 2PW
Tel: +44 1252 299024
Email: enquiries@britishmedicalauctions.co.uk
Web: www.britishmedicalauctions.co.uk

British Medical Auctions
Ultrasound Equipment Quality Check (EQC)
Preventive Maintenance Service Report (PM Service)

V1.0 05.2022

7. PHOTOGRAPHIC DOCUMENTATION



Photo 143

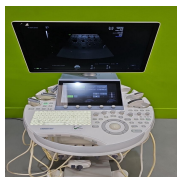


Photo 144



Photo 145



Photo 146

Media summary



Photo 1



Photo 2

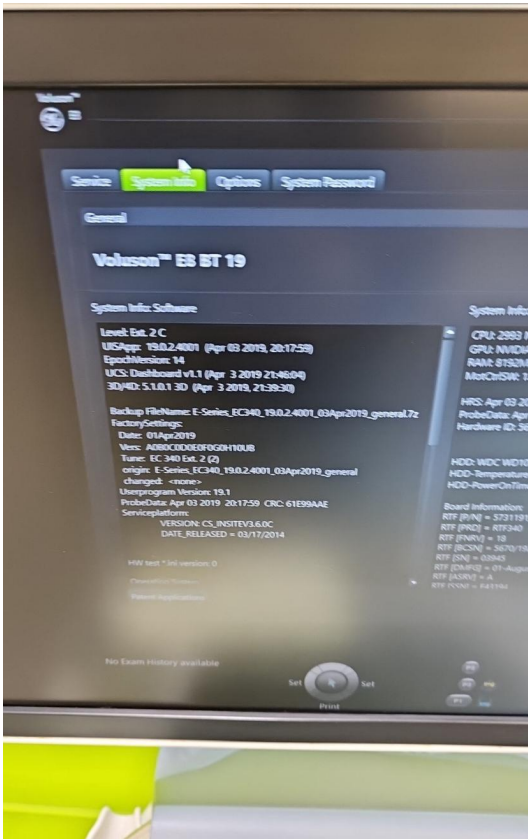


Photo 3

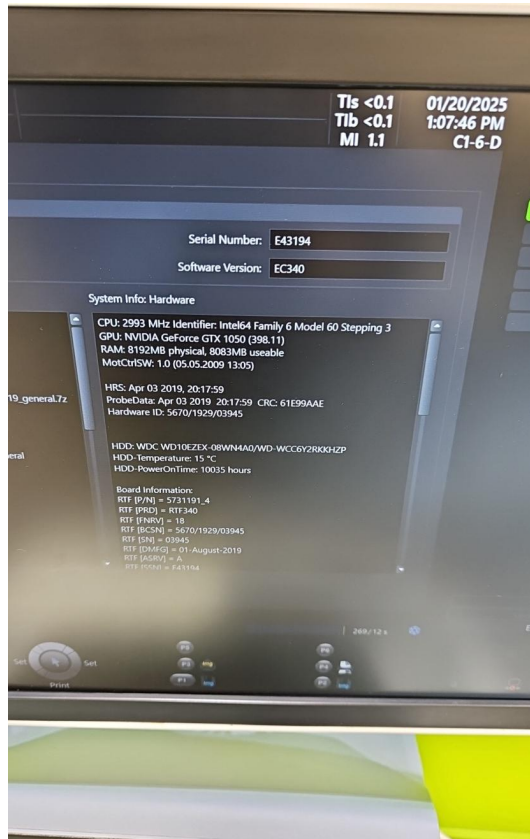


Photo 4

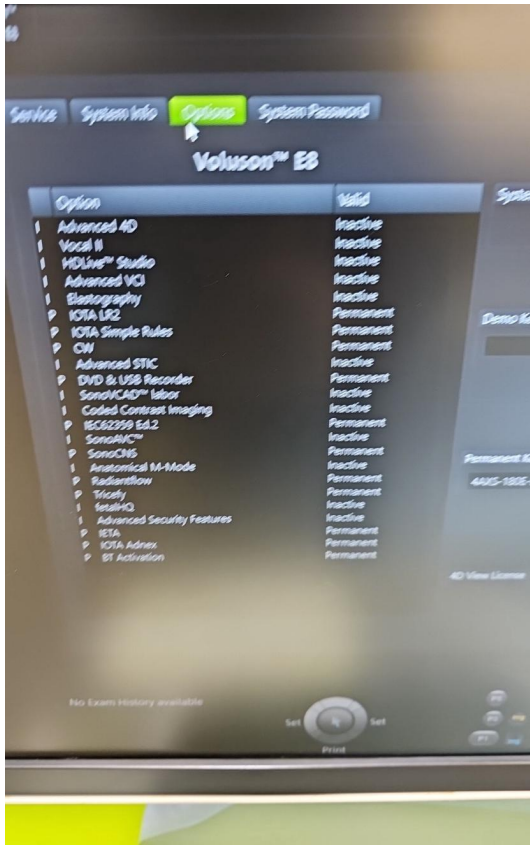


Photo 5

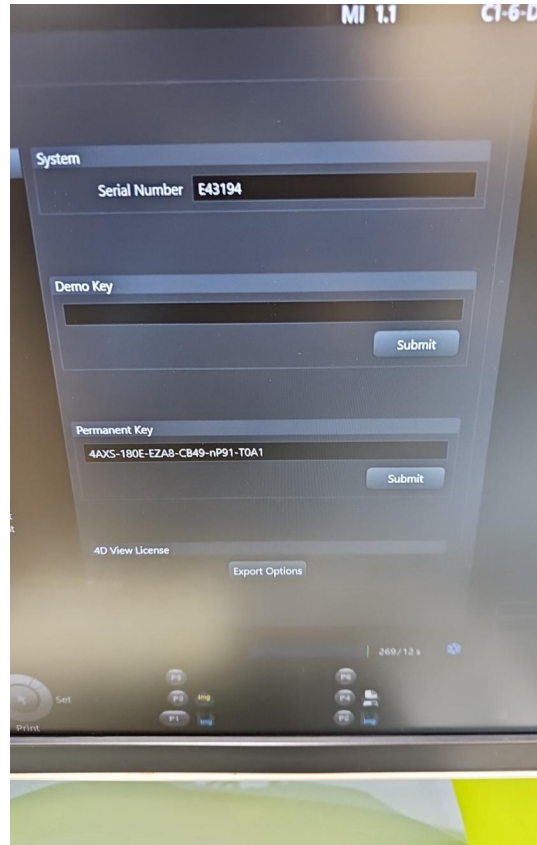


Photo 6



Photo 7

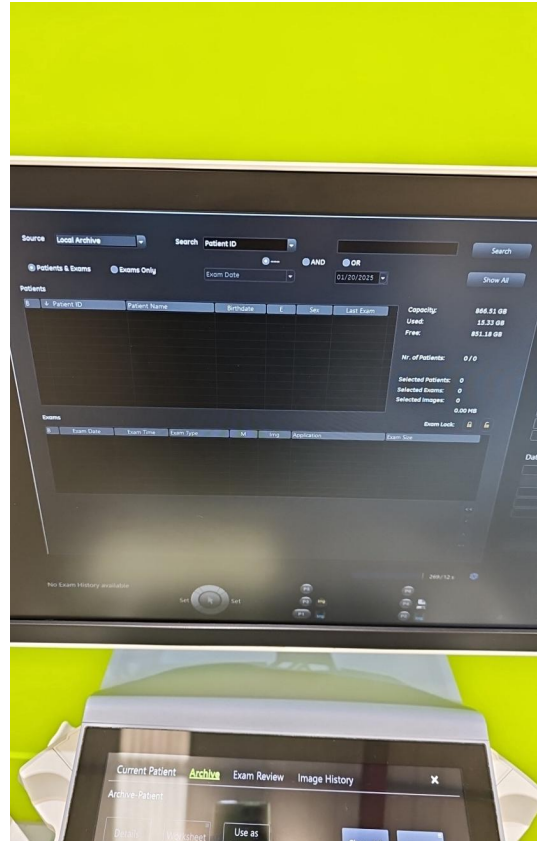


Photo 8

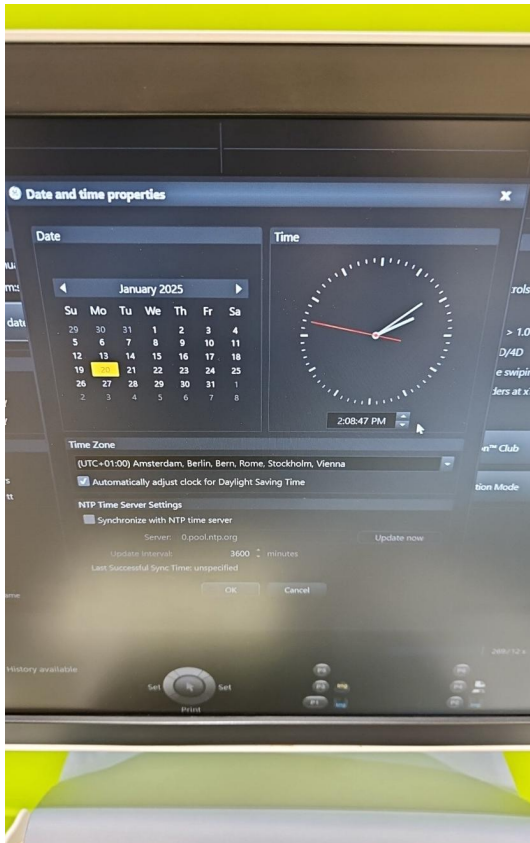


Photo 9



Photo 10



Photo 11



Photo 12



Photo 13



Photo 14



Photo 15



Photo 16



Photo 17



Photo 18



Photo 19



Photo 20

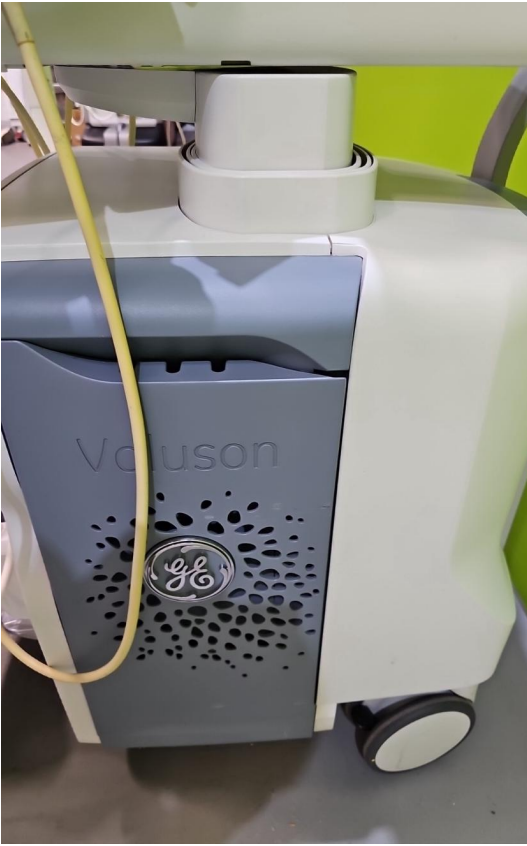


Photo 21

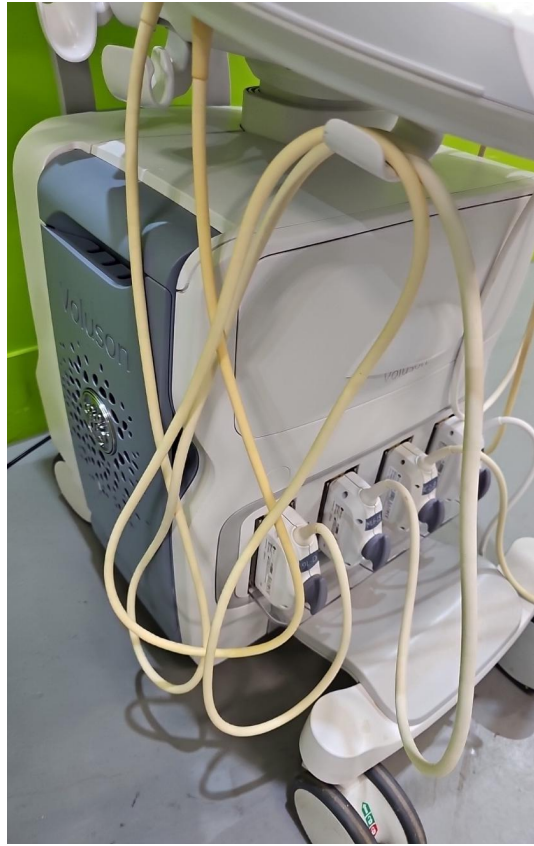


Photo 22



Photo 23



Photo 24



Photo 25



Photo 26



Photo 27



Photo 28



Photo 29

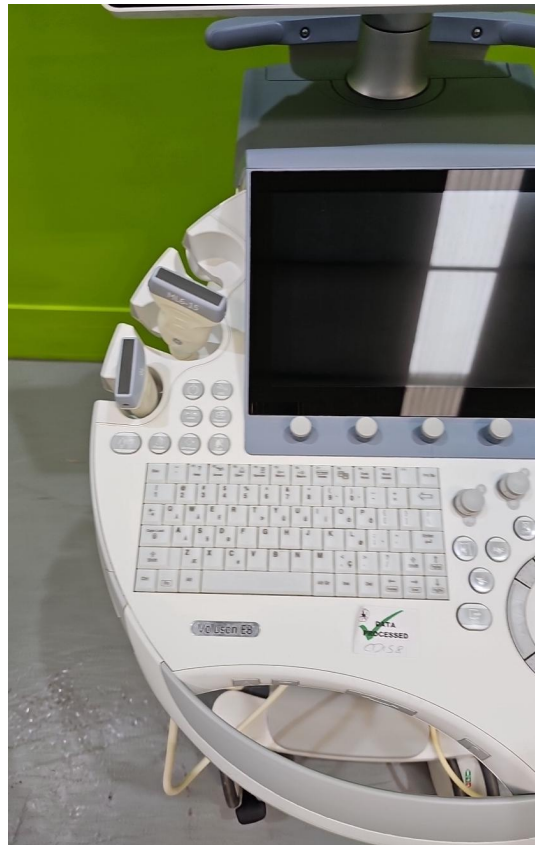


Photo 30



Photo 31

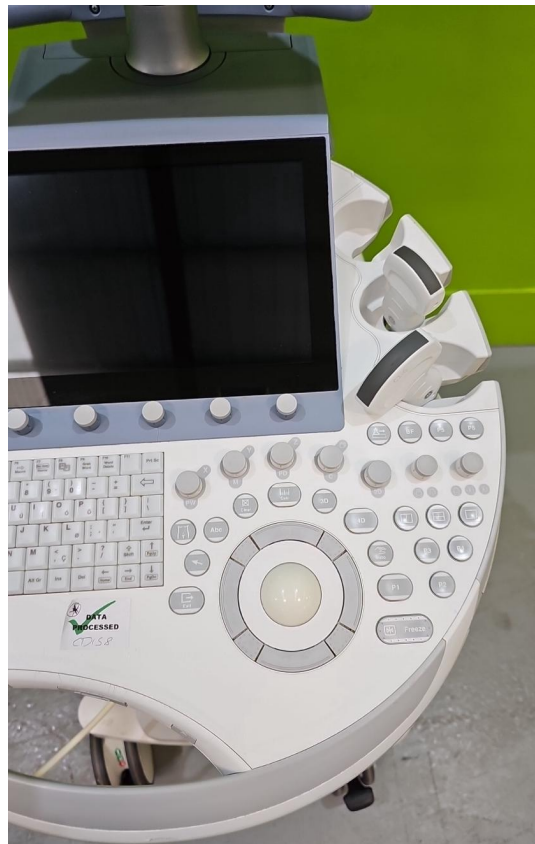


Photo 32

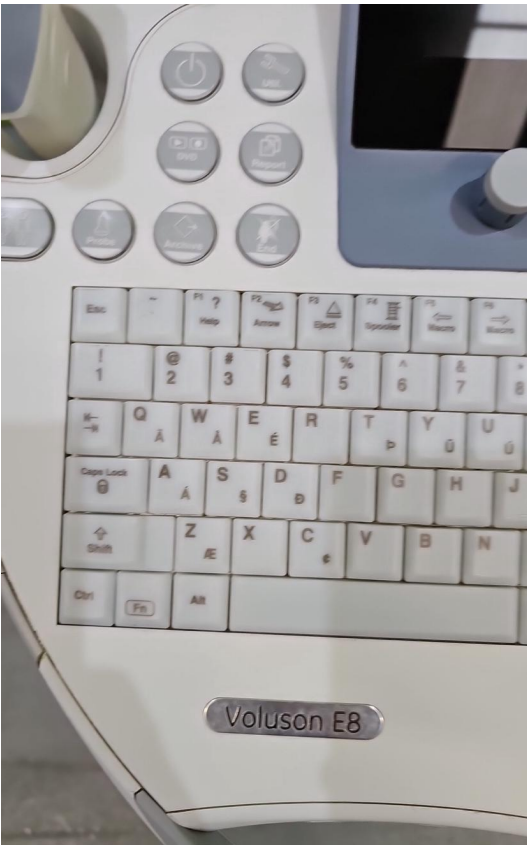


Photo 33

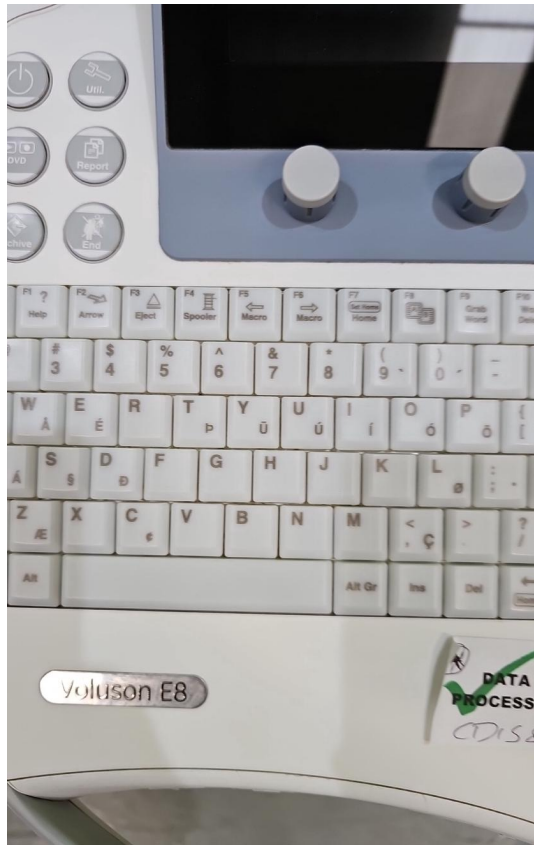


Photo 34

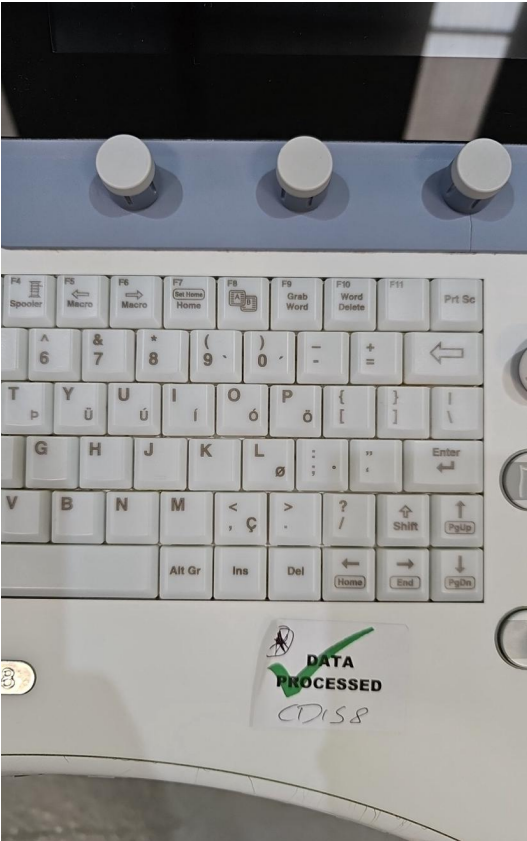


Photo 35



Photo 36

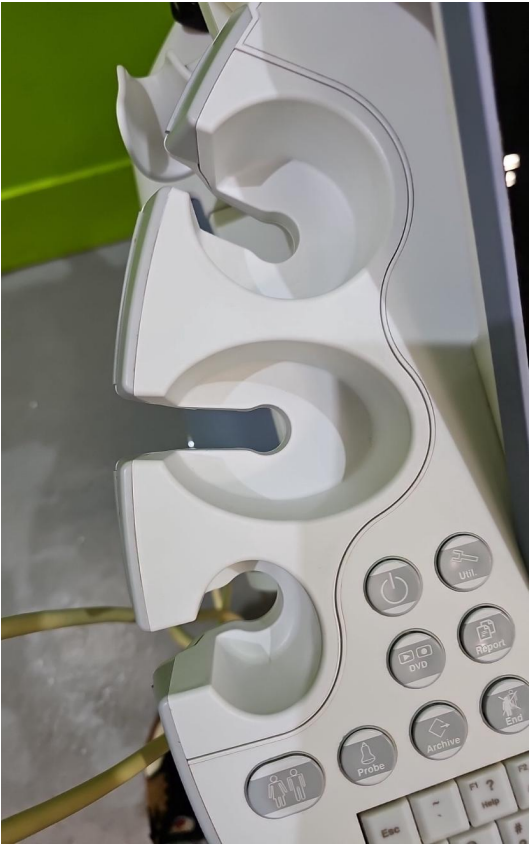


Photo 37



Photo 38

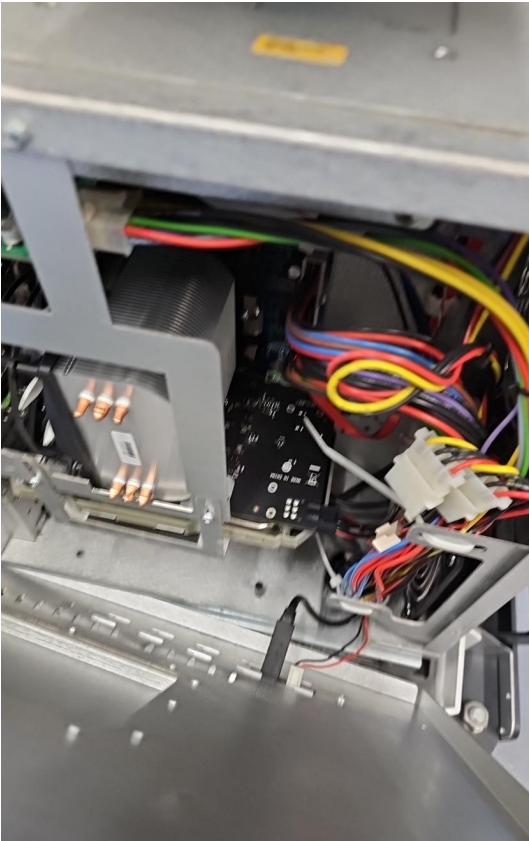


Photo 39

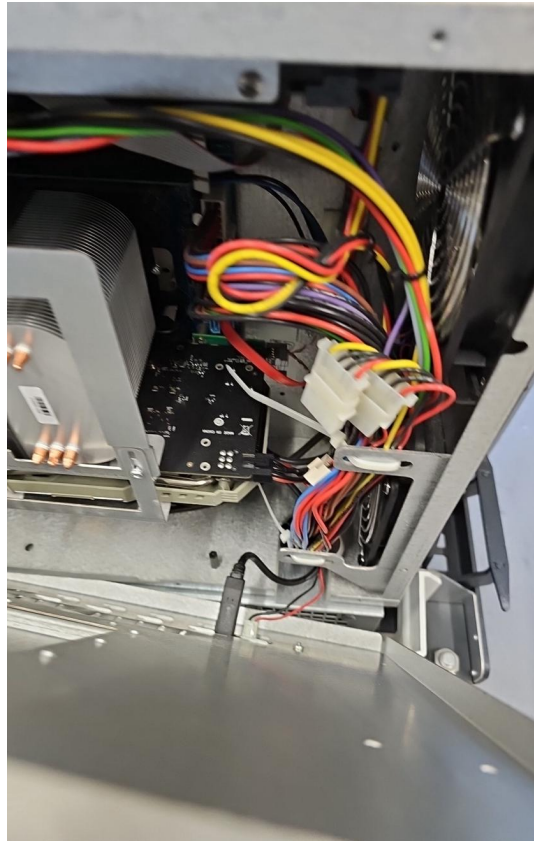


Photo 40

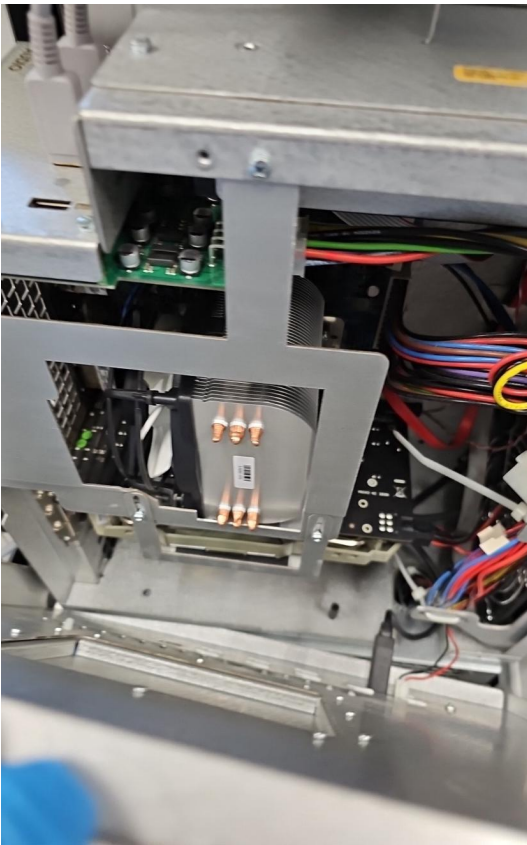


Photo 41

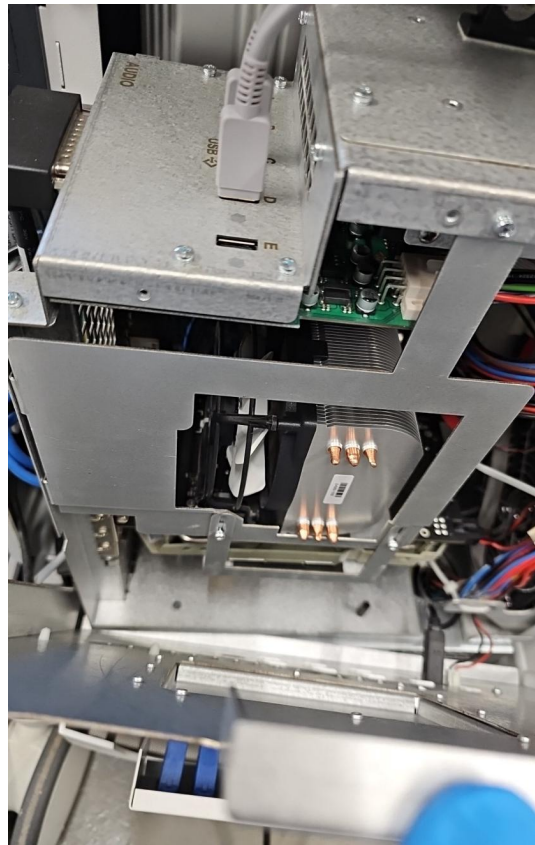


Photo 42

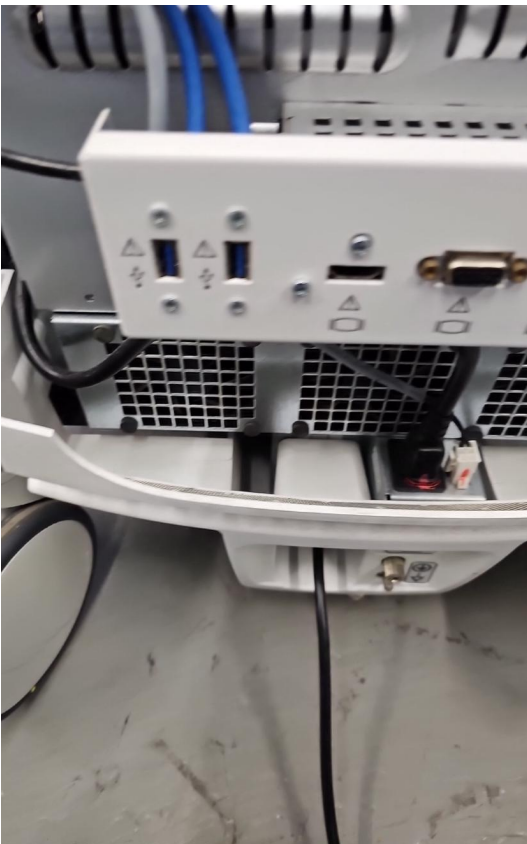


Photo 43



Photo 44



Photo 45



Photo 46

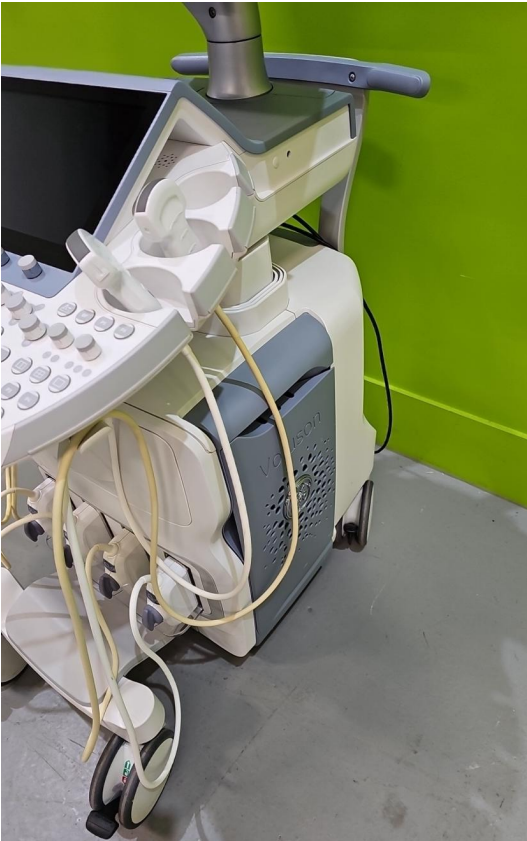


Photo 47



Photo 48

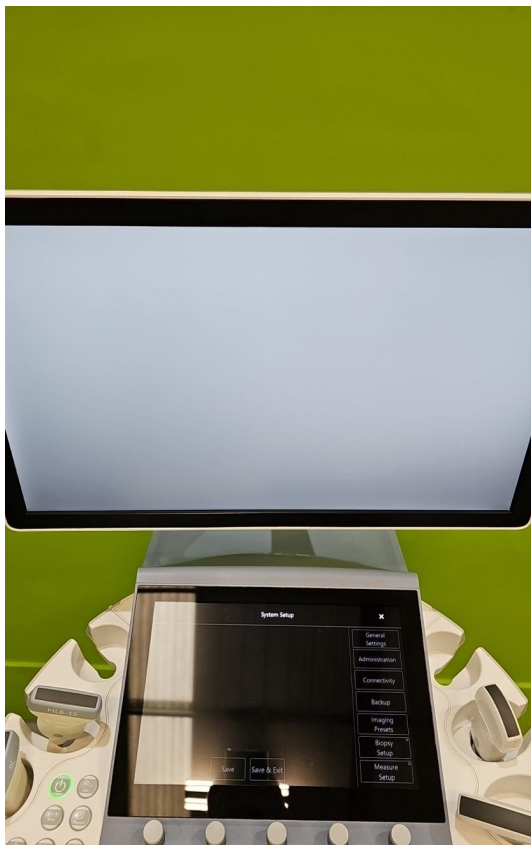


Photo 49



Photo 50



Photo 51



Photo 52



Photo 53

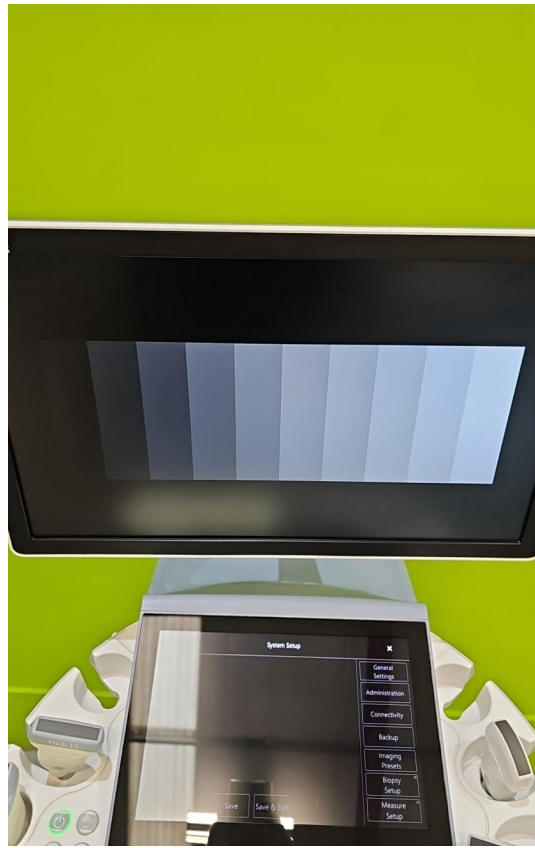


Photo 54



Photo 55



Photo 56



Photo 57



Photo 58

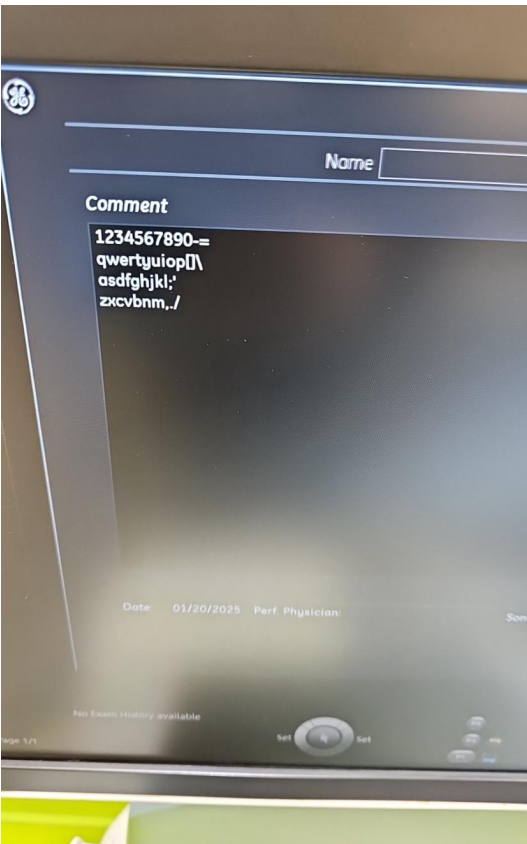


Photo 59

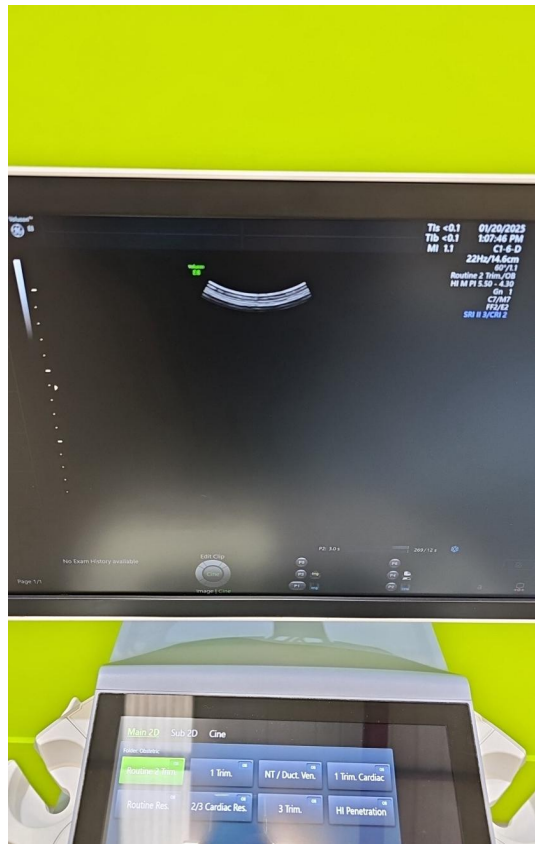


Photo 60

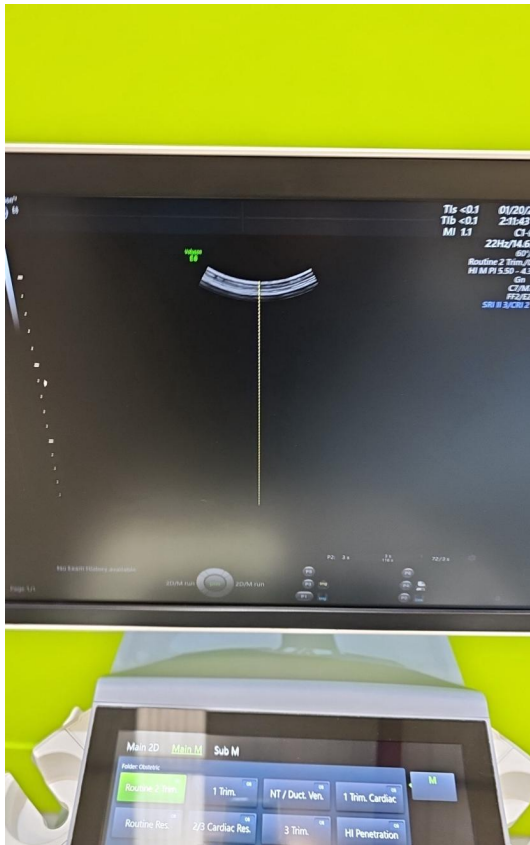


Photo 61

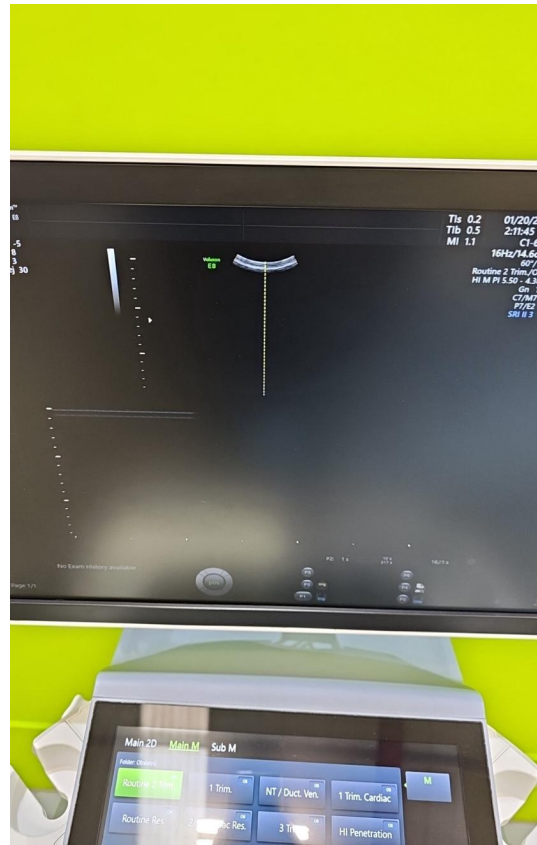


Photo 62

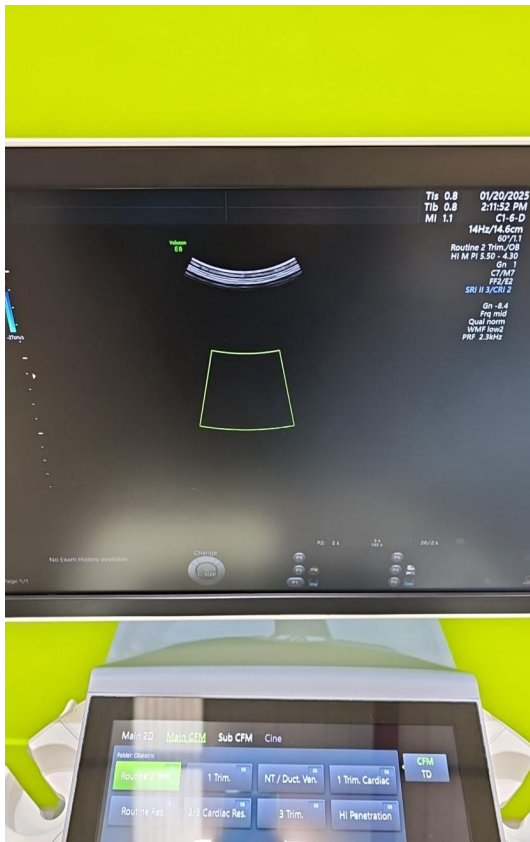


Photo 63



Photo 64

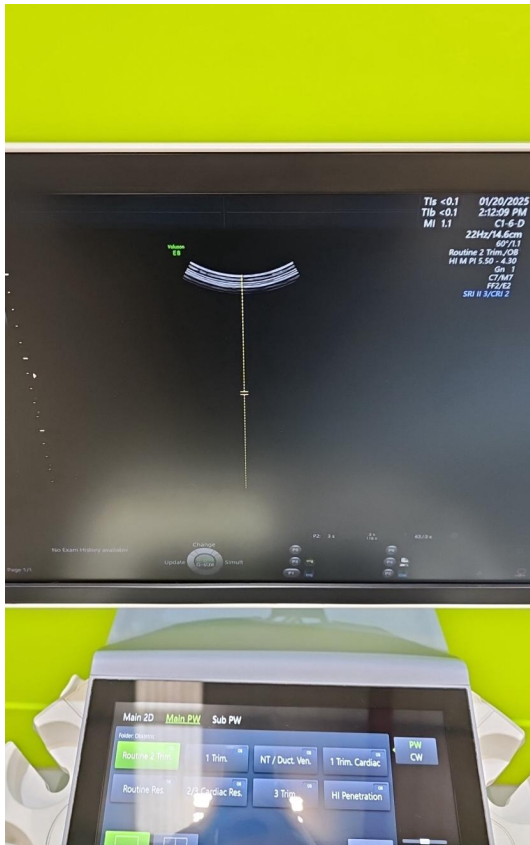


Photo 65

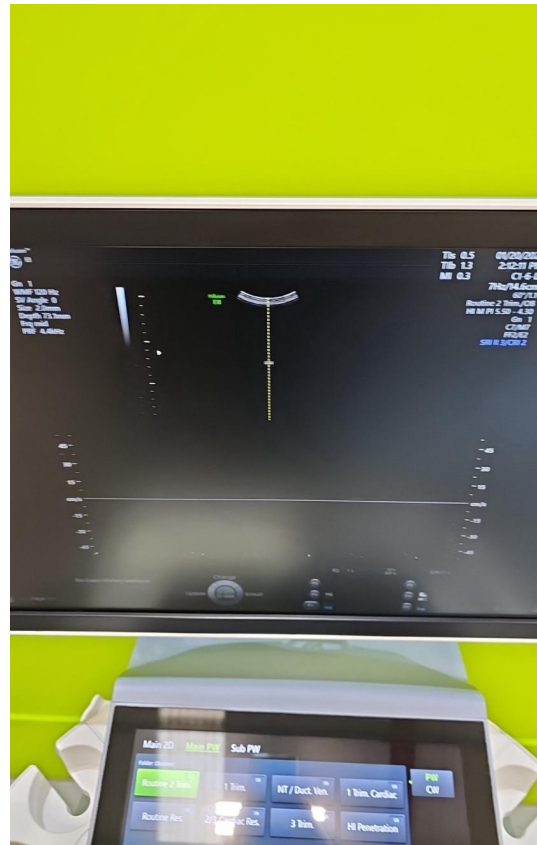


Photo 66

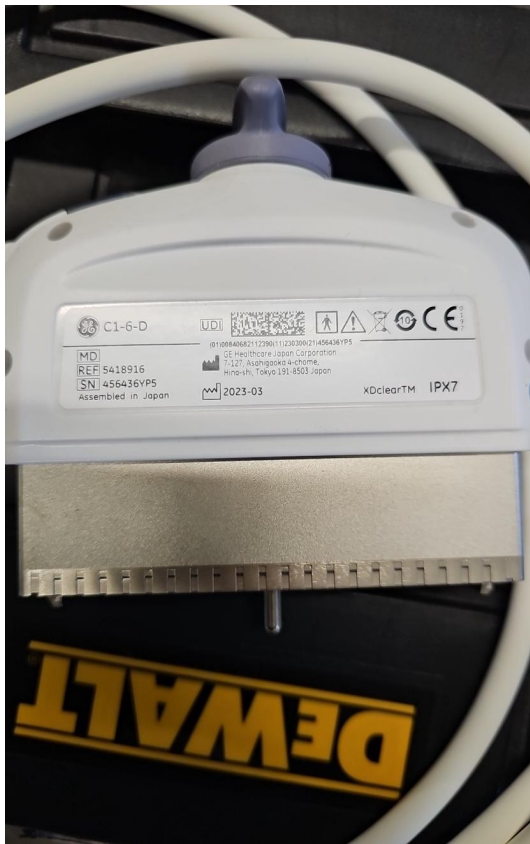


Photo 67



Photo 68



Photo 69

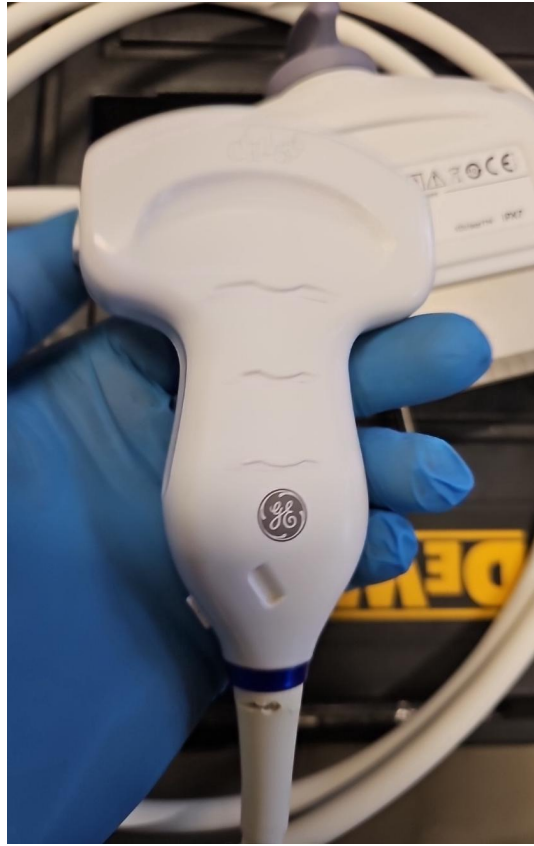


Photo 70



Photo 71

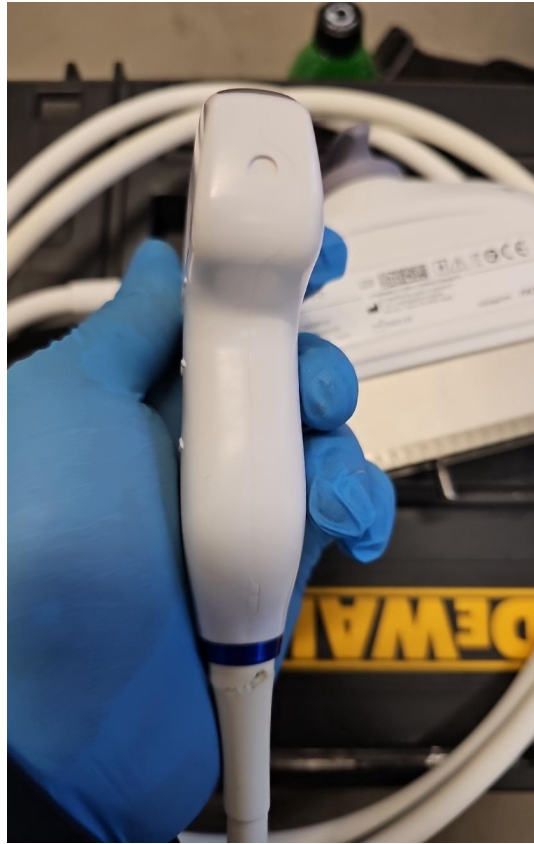


Photo 72

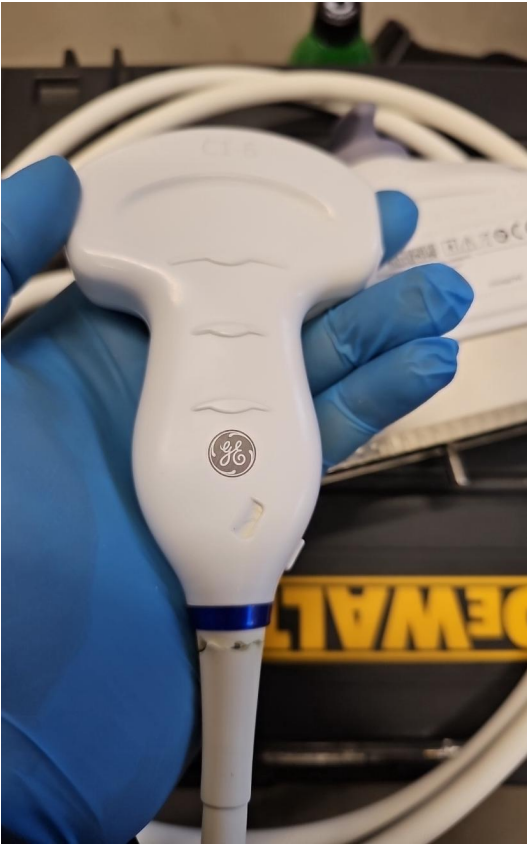


Photo 73



Photo 74



Photo 75



Photo 76



Photo 77

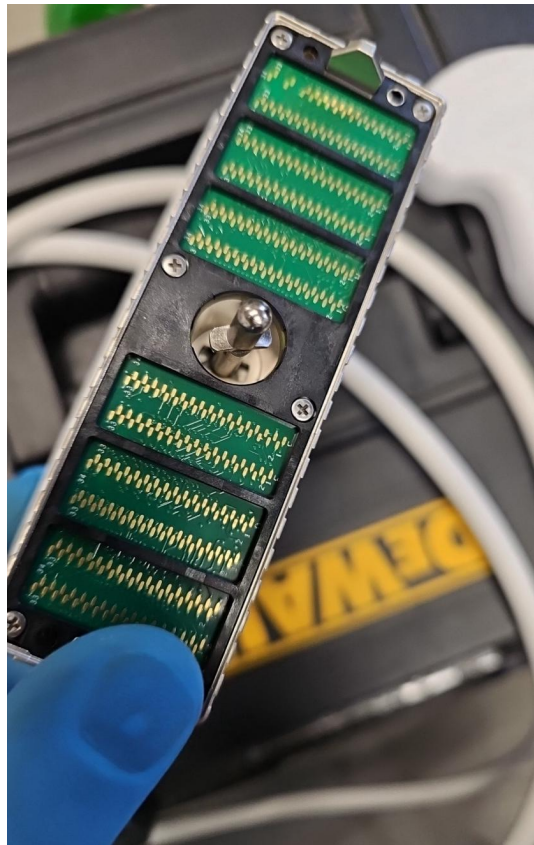


Photo 78

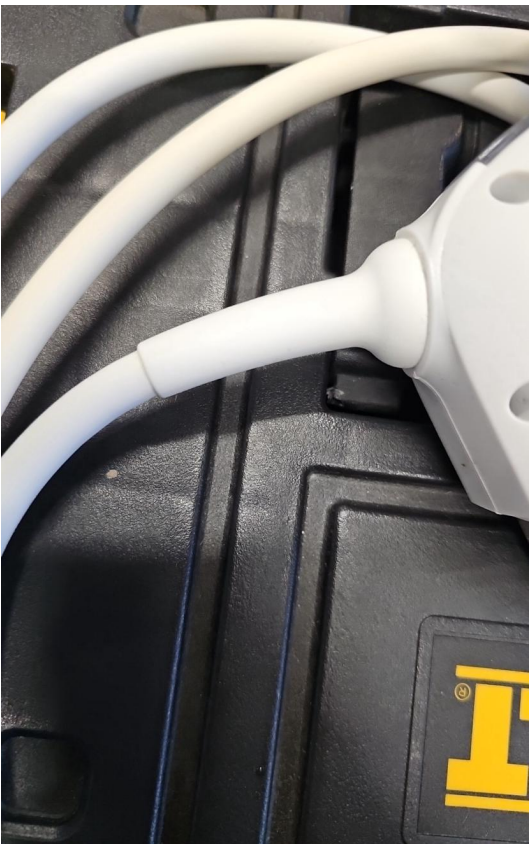


Photo 79

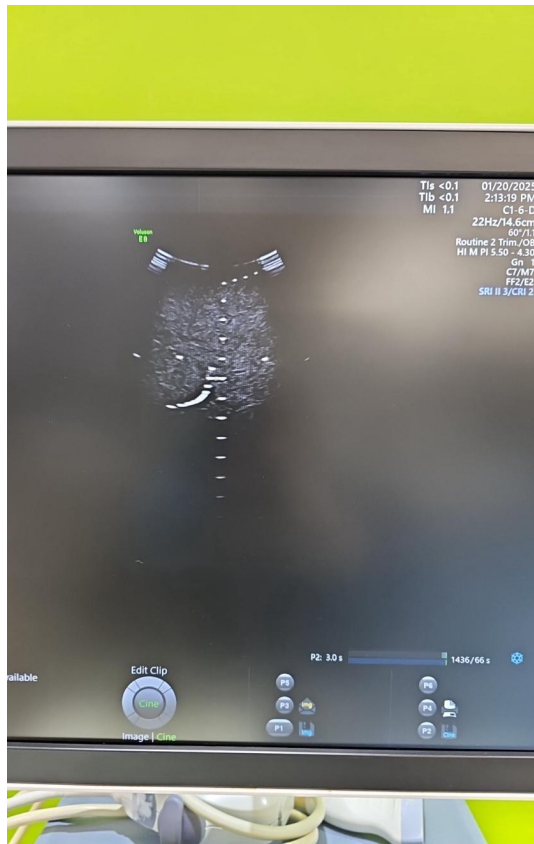


Photo 80



Photo 81

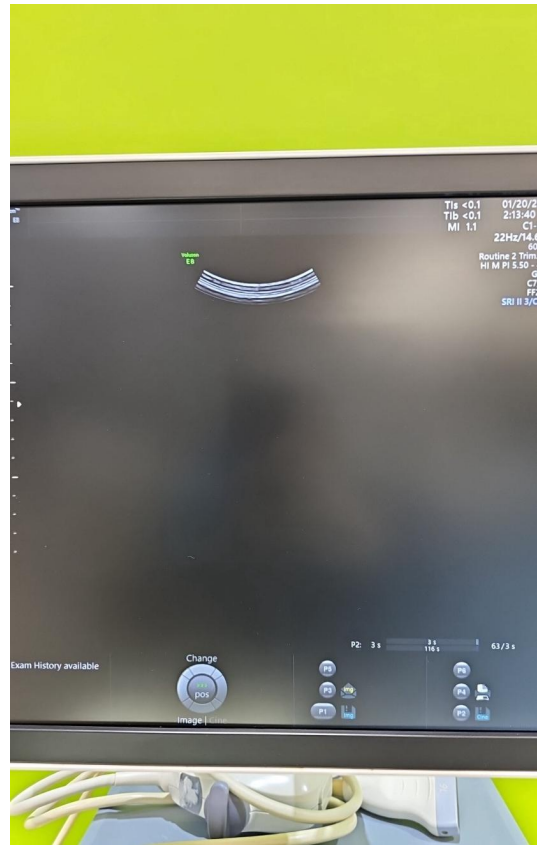


Photo 82

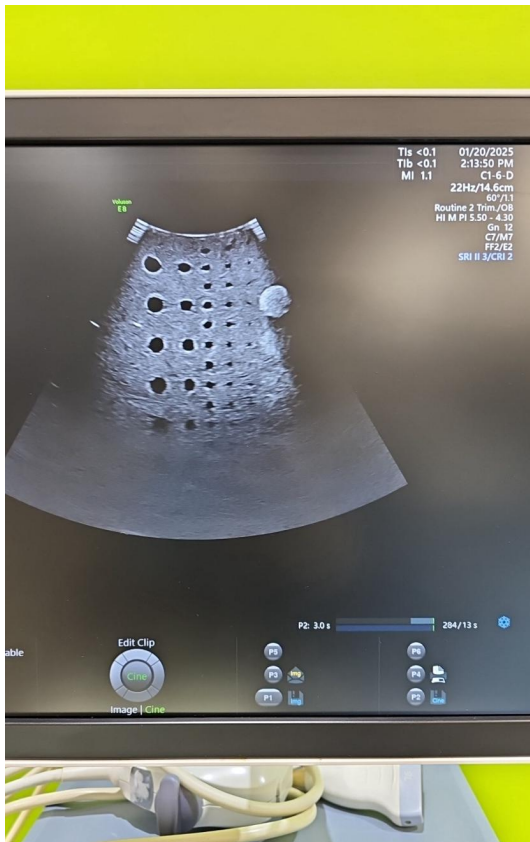


Photo 83



Photo 84



Photo 85



Photo 86

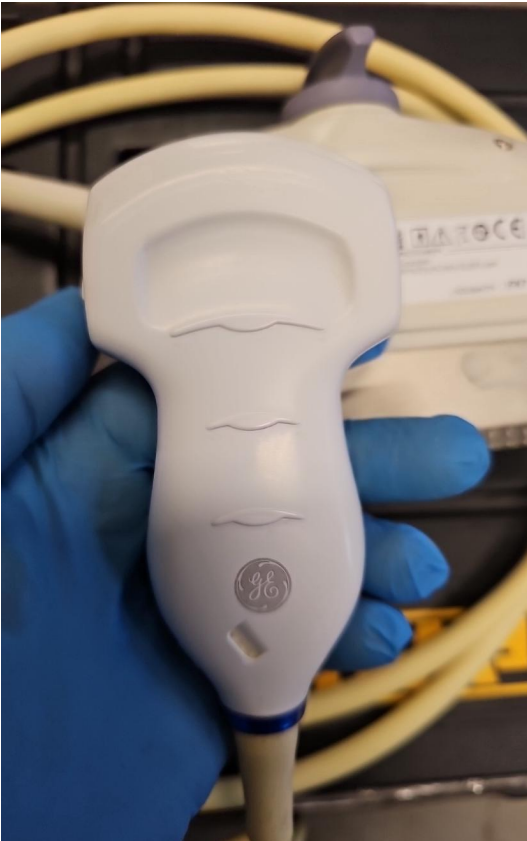


Photo 87



Photo 88



Photo 89

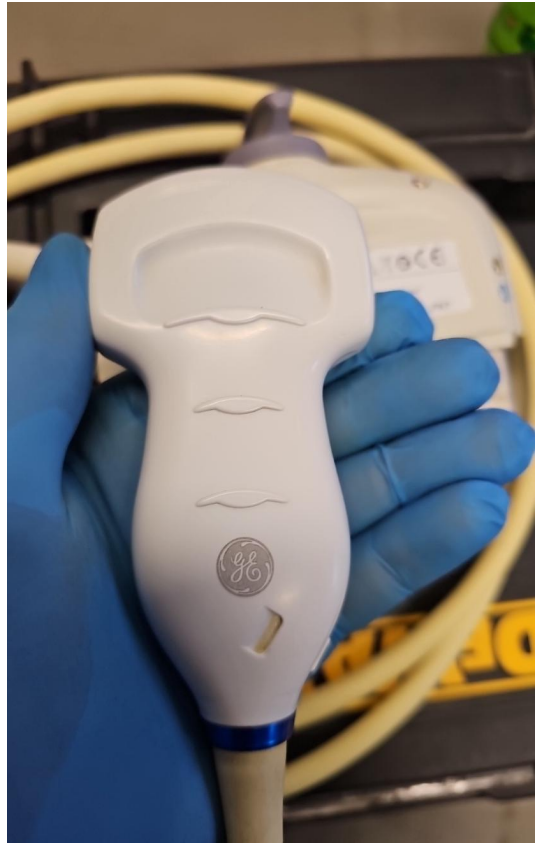


Photo 90



Photo 91



Photo 92

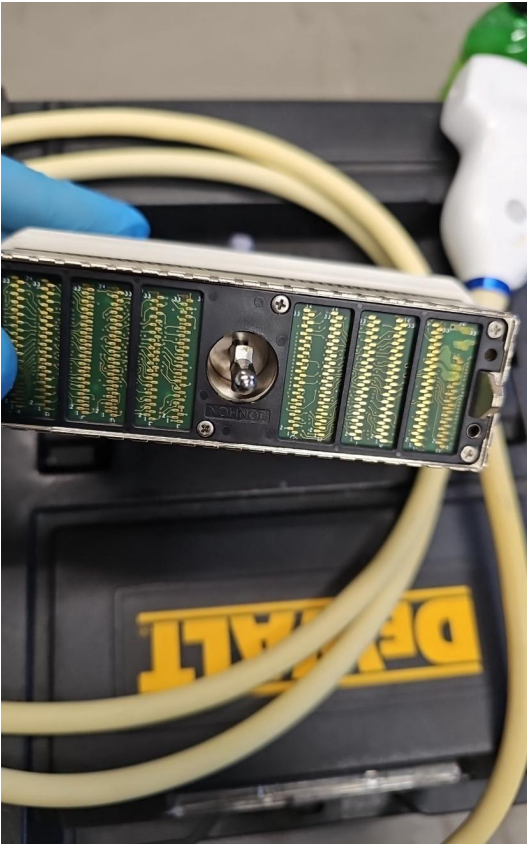


Photo 93



Photo 94

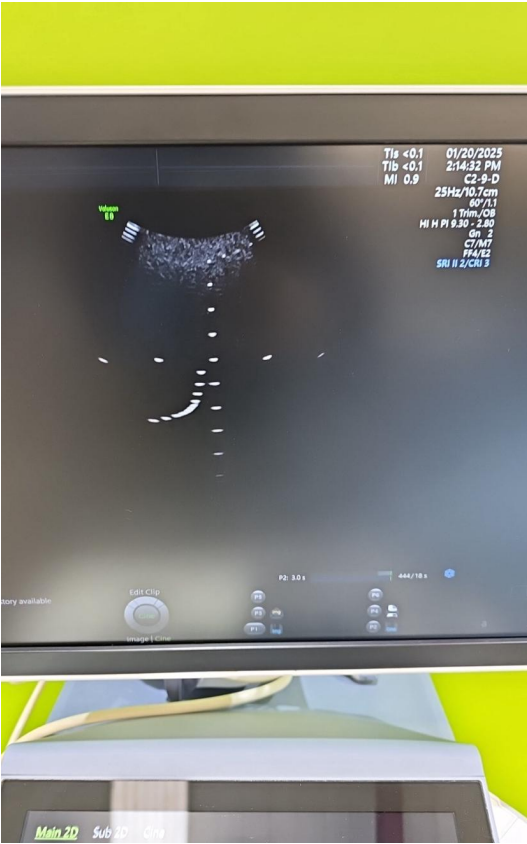


Photo 95



Photo 96

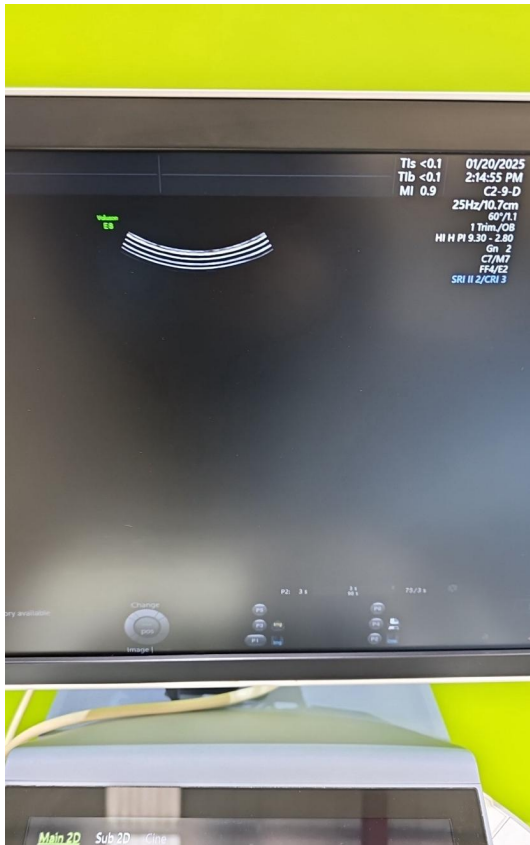


Photo 97

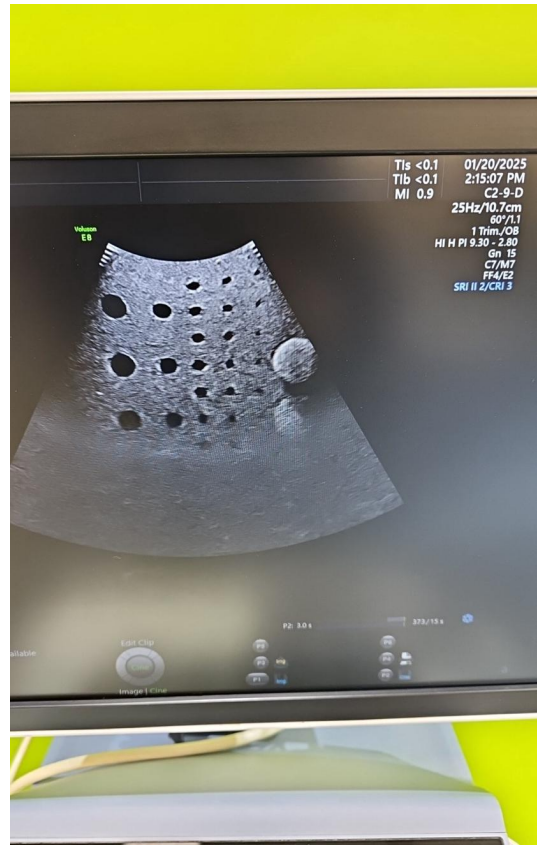


Photo 98



Photo 99

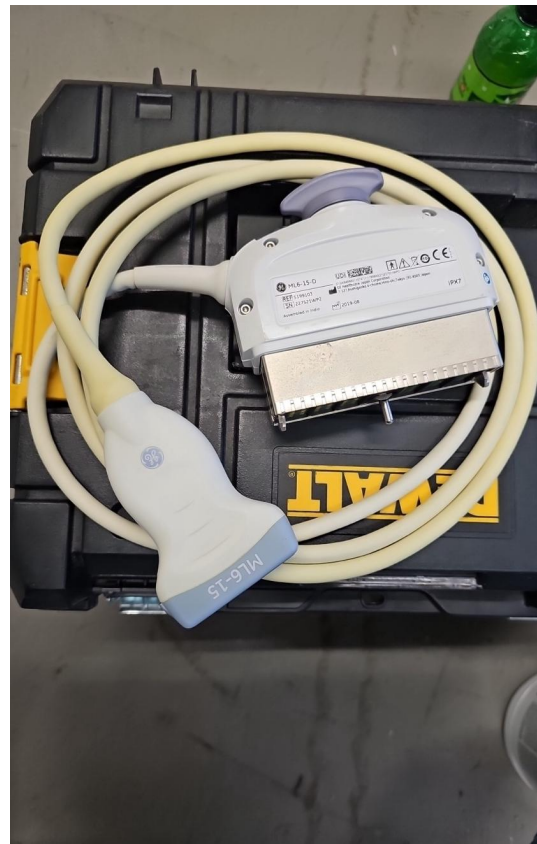


Photo 100



Photo 101



Photo 102

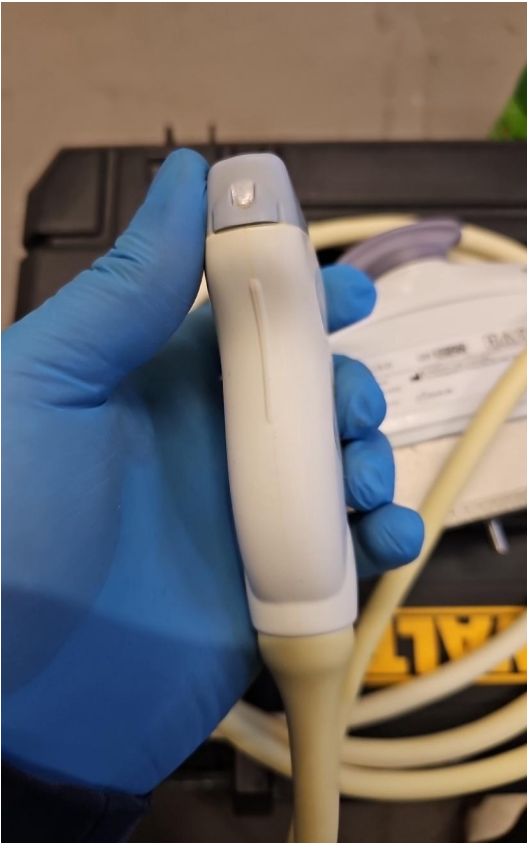


Photo 103

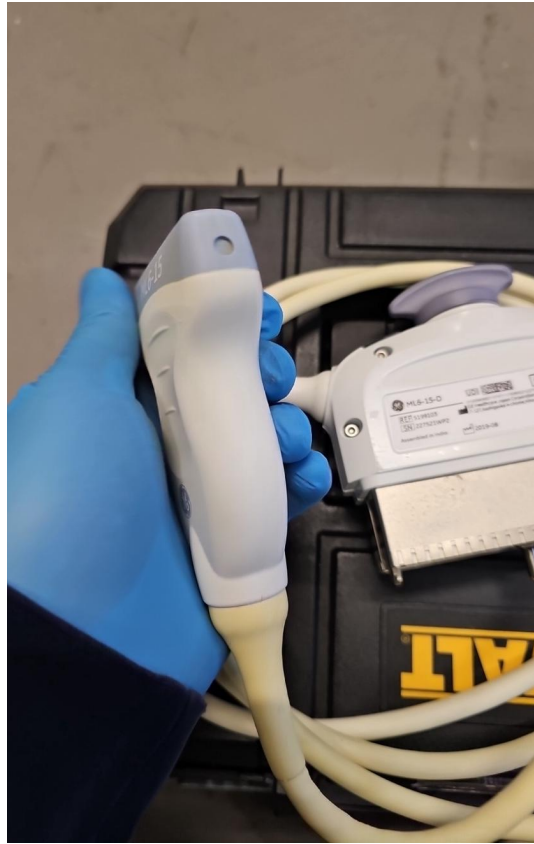


Photo 104



Photo 105

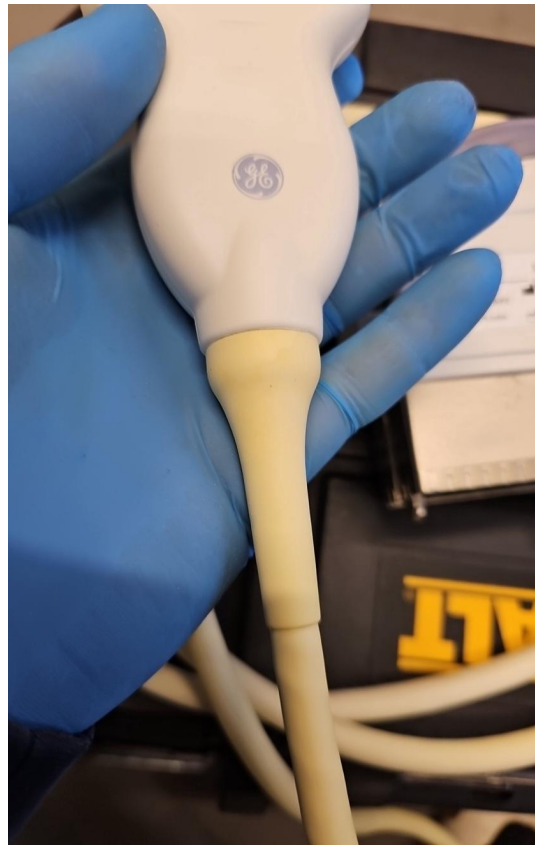


Photo 106



Photo 107

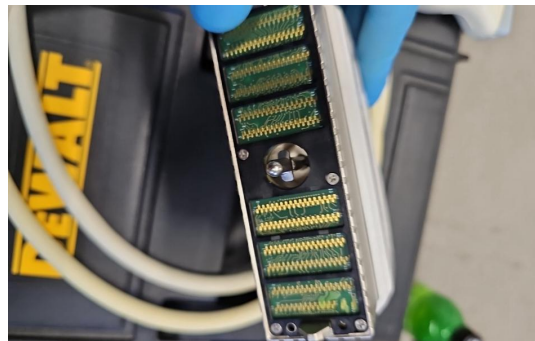


Photo 108



Photo 109

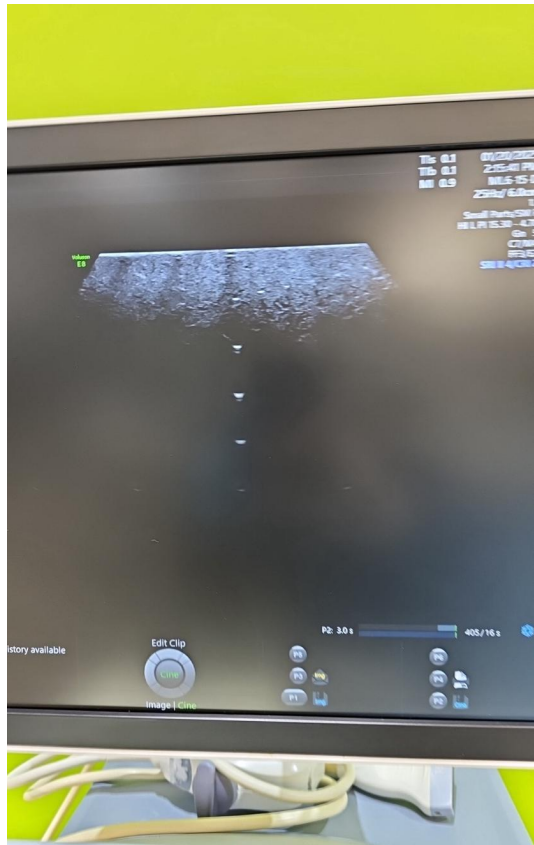


Photo 110

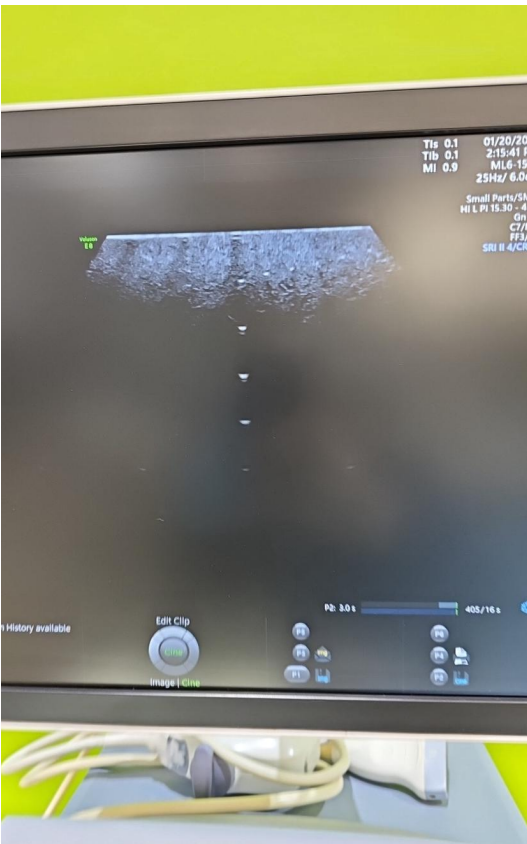


Photo 111

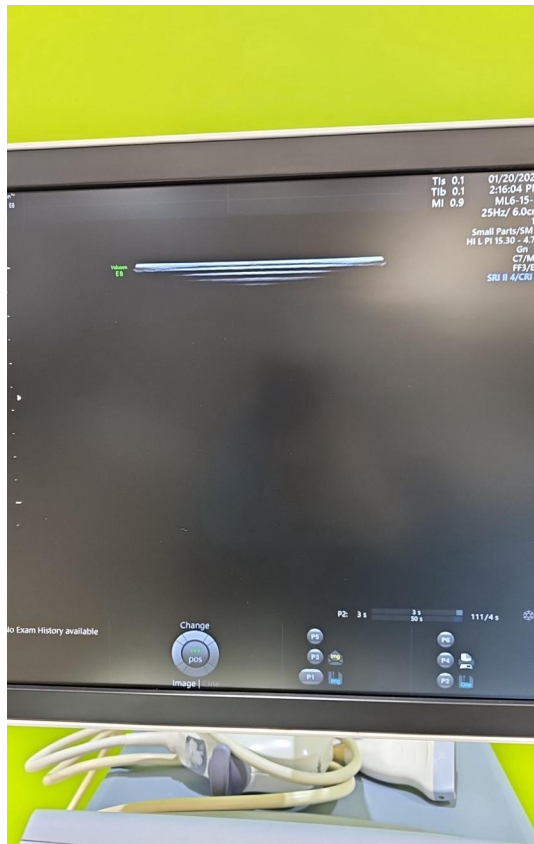


Photo 112

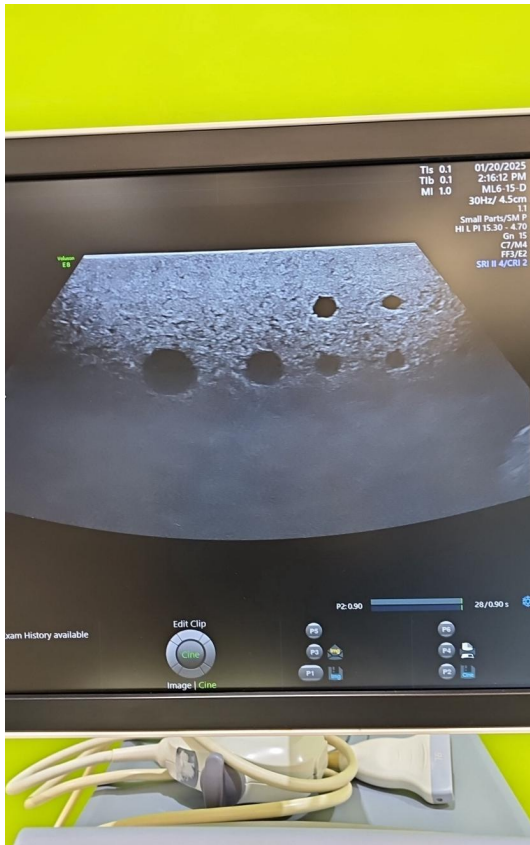


Photo 113



Photo 114



Photo 115



Photo 116



Photo 117



Photo 118



Photo 119



Photo 120

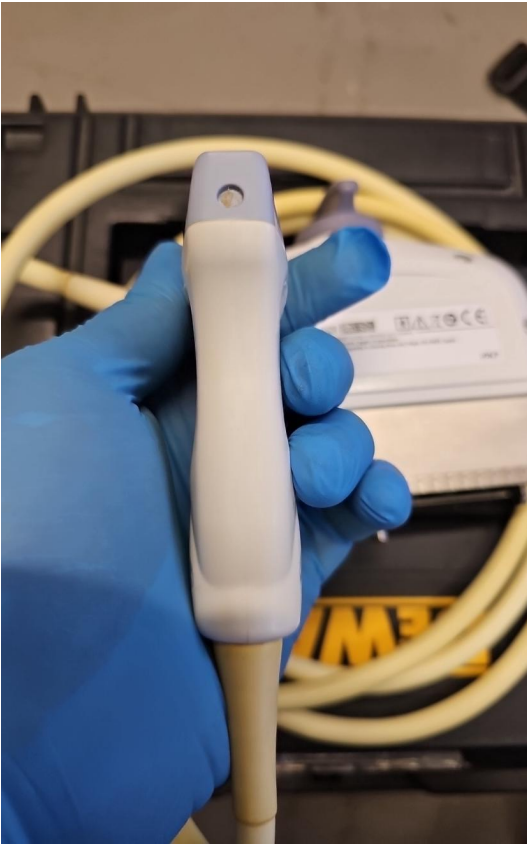


Photo 121

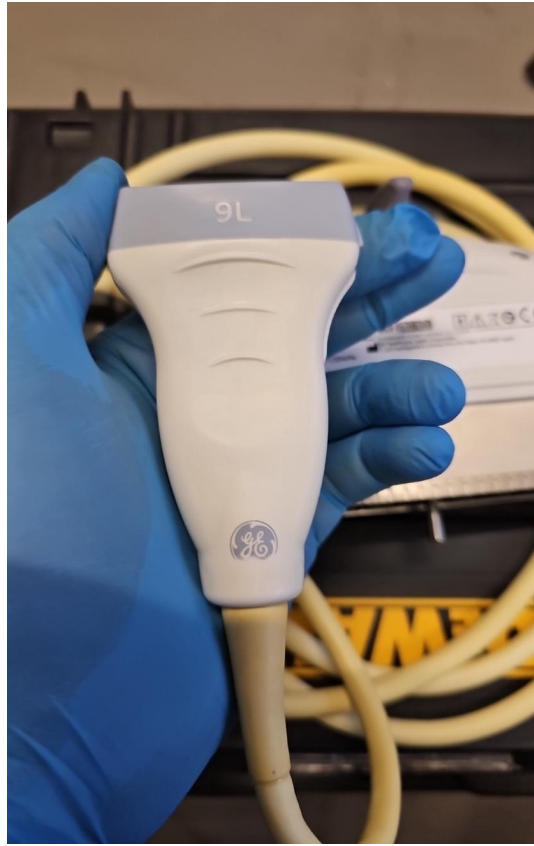


Photo 122



Photo 123



Photo 124



Photo 125



Photo 126



Photo 127



Photo 128



Photo 129

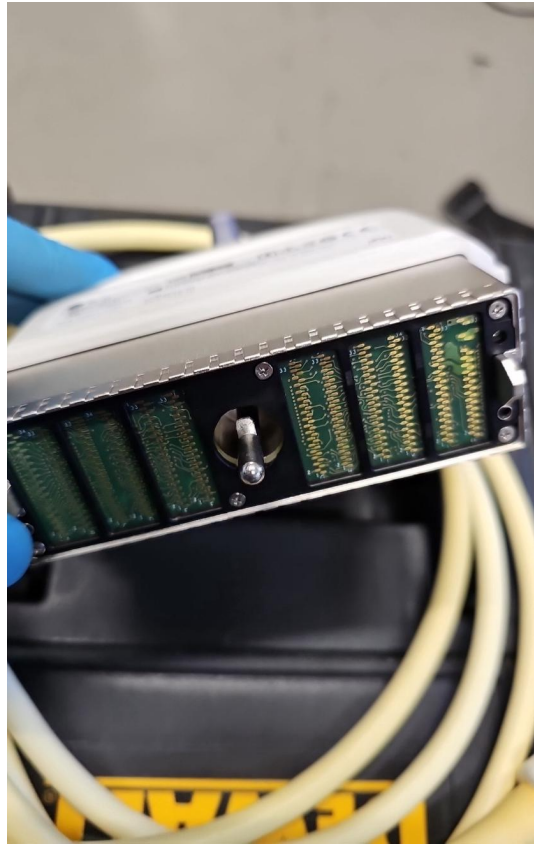


Photo 130



Photo 131



Photo 132



Photo 133



Photo 134



Photo 135

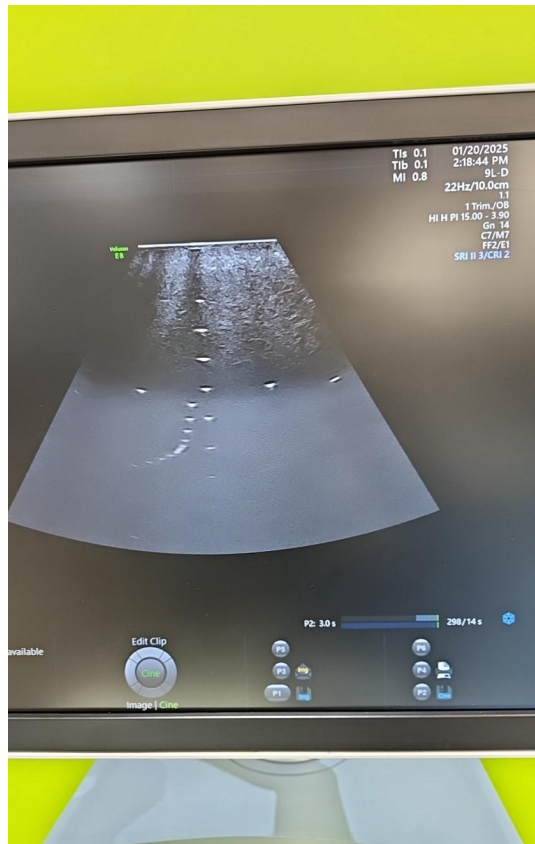


Photo 136

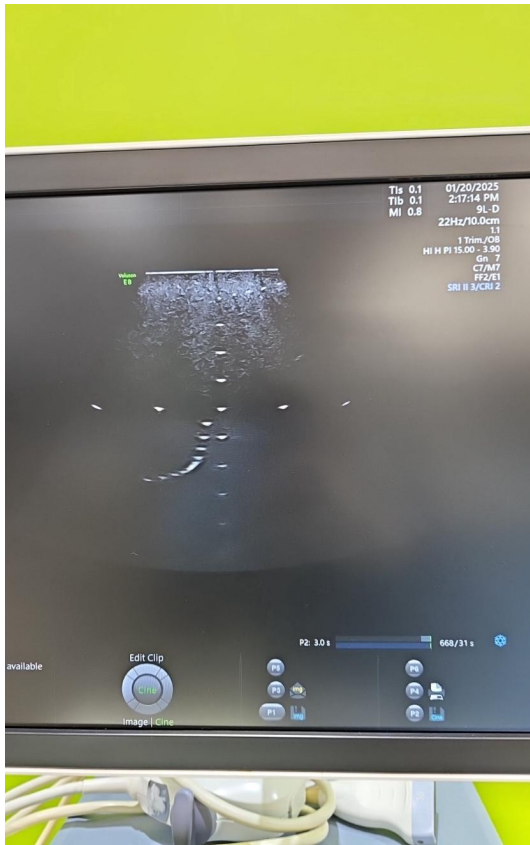


Photo 137



Photo 138

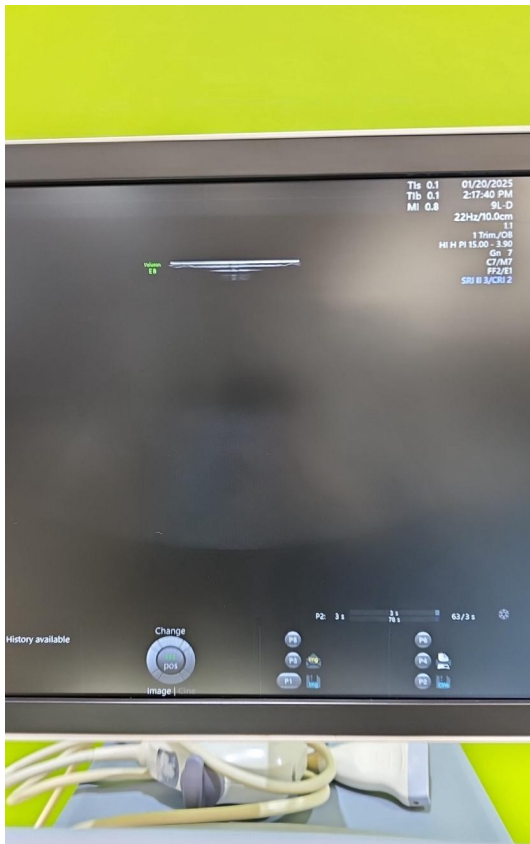


Photo 139

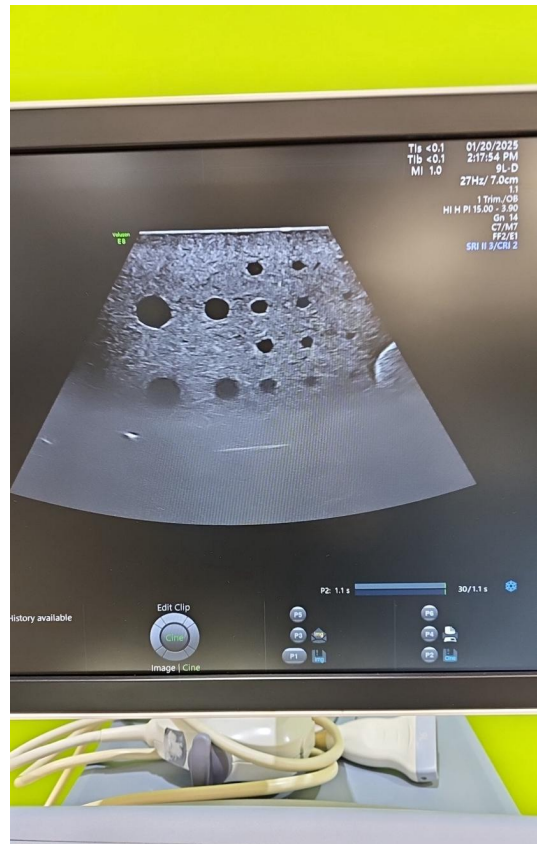


Photo 140

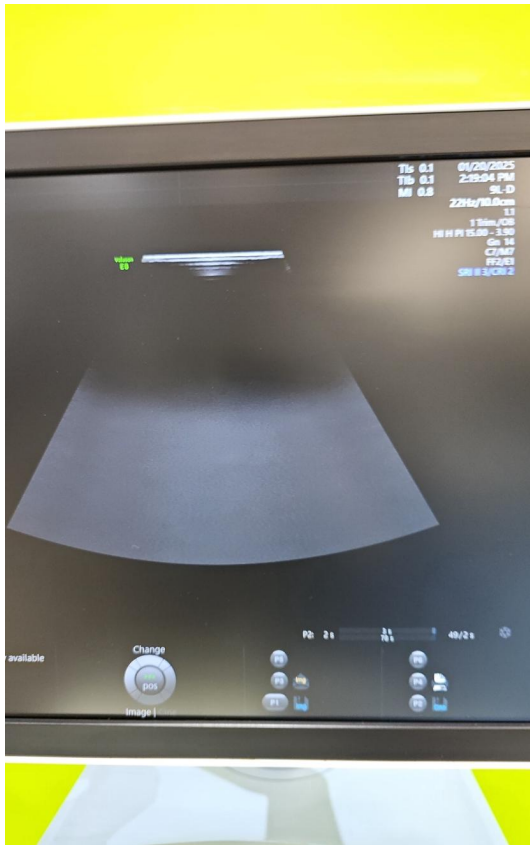


Photo 141



Photo 142



Photo 143

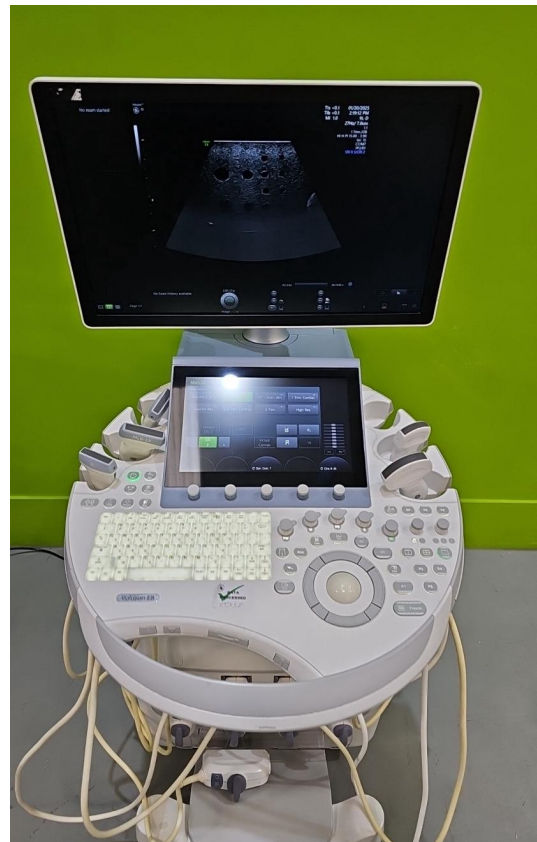


Photo 144



Photo 145



Photo 146