



British Medical Auctions Ultrasound Equipment Quality Check (EQC) Preventive Maintenance Service Report (PM Service)

GE / Voluson E8 / E39664

Complete

British Medical Auctions

Address: Unit 1, Sterling Centre, Eastern Road, Bracknell, RG12 2PW
Tel: +44 1252 299024
Email: enquiries@britishmedicalauctions.co.uk
Web: www.britishmedicalauctions.co.uk

Department: Service

System has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Safe

System Type:

Ultrasound System



Photo 1

System Manufacture:

GE

Model:

Voluson E8

Serial No.:

E39664



Photo 2

Year of Manufacture:

03.2018

Software Level:

BT18 EC330

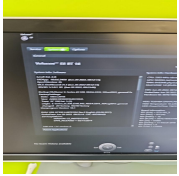


Photo 3

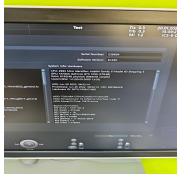


Photo 4

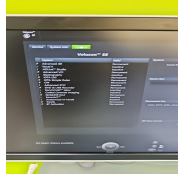


Photo 5

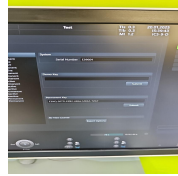


Photo 6

Service Code:

UK740

Electrical Safety Test Asset ID:

UK 763

Test Equipment No. (Electrical Safety Analyzer):
Rigel 288+
SN.:10M-0263

Calibration Due Date:
05.2025

Test Equipment No. (Ultrasound Phantom):
ATS Laboratories MODEL: 539
Multipurpose Ultrasound Phantom
SN.: 200493148

Date Performed

20.01.2025 15:46 CET

Performed By

MS

1. PRELIMINARY CHECKS

Power Up

Yes



Photo 7

Database Cleared (Including all Patient Data and Previous Owner Details)

Yes



Photo 8

Password Protection

No

Time and Date Check

Yes



Photo 9

2. PHYSICAL INSPECTION

Input Power

Pass



Photo 10

External I/O

Pass

Wheels and Brakes

Pass



Photo 11



Photo 12

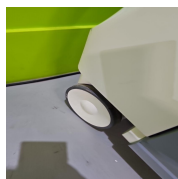


Photo 13



Photo 14

Control Panel Movement

Pass



Photo 15



Photo 16



Photo 17



Photo 18

Cables and Connectors

Pass

Shielding and Covers

Pass



Photo 19



Photo 20



Photo 21

Console

Pass

Missing trackball



Photo 22

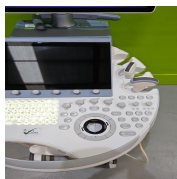


Photo 23



Photo 24

Control Panel

Pass



Photo 25

Monitor Arm

Pass



Photo 26

3. CLEANING

Monitor (CRT/LCD)

Yes



Photo 27

Touch screen

Yes



Photo 28

Console

Yes



Photo 29



Photo 30

Keyboard

Yes

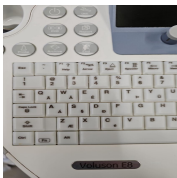


Photo 31



Photo 32



Photo 33

Trackball / Optical mouse

Yes



Photo 34

Probe Holder

Yes

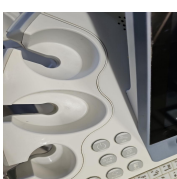


Photo 35



Photo 36

Internal Cleaning

Yes

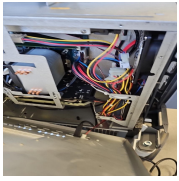


Photo 37

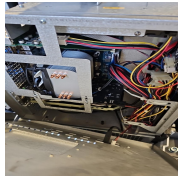


Photo 38

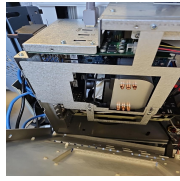


Photo 39

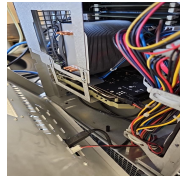


Photo 40



Photo 41



Photo 42



Photo 43



Photo 44

Filter

Yes



Photo 45

Shielding and Covers

Yes



Photo 46



Photo 47



Photo 48

Thermal Printer Head

Yes

4. FUNCTIONAL CHECKS

Monitor

Pass



Photo 49



Photo 50



Photo 51



Photo 52

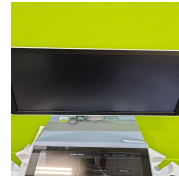


Photo 53



Photo 54



Photo 55

Touch Panel

Pass

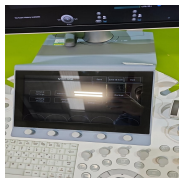


Photo 56

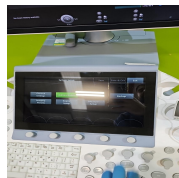


Photo 57



Photo 58

Console

Pass

Missing trackball



Photo 59



Photo 60

Keyboard

Pass

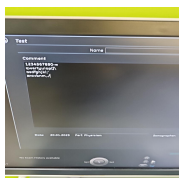


Photo 61

B-Mode

Pass

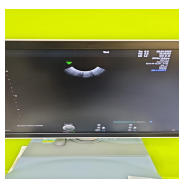


Photo 62

M-Mode

Pass



Photo 63



Photo 64

C-Mode

Pass



Photo 65

PD-Mode

Pass



Photo 66

PW-Mode

Pass

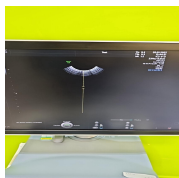


Photo 67



Photo 68

5. PROBE NO.: 1

Probe Make

GE

Probe Model

IC5-9-D

Probe Serial Number

739630WX2



Photo 69

Visual inspection

Fail

Cable

Fail



Photo 70



Photo 71

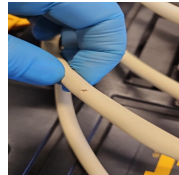


Photo 72



Photo 73

Lens

Pass



Photo 74

Probe Casing

Pass

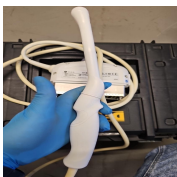


Photo 75



Photo 76



Photo 77



Photo 78

Probe Casing Strain Relief

Pass

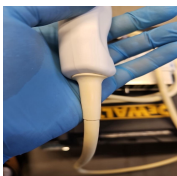


Photo 79

Connector

Pass



Photo 80



Photo 81

Connector Strain Relief

Pass



Photo 82

Lateral resolution

Pass

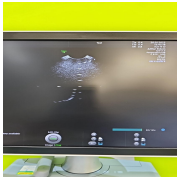


Photo 83

Axial resolution

Pass



Photo 84

Image benchmark

Pass



Photo 85



Photo 86

Probe passes inspection & test

No

Probe has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

No

PROBE NO.: 2

Probe Make

GE

Probe Model

C2-9-D

Probe Serial Number

275736YP7



Photo 87

Visual inspection

Pass

Cable

Pass



Photo 88

Lens

Pass

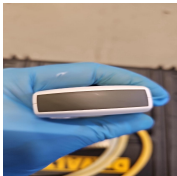


Photo 89

Probe Casing

Pass



Photo 90



Photo 91



Photo 92



Photo 93

Probe Casing Strain Relief

Pass



Photo 94

Connector

Pass



Photo 95



Photo 96

Connector Strain Relief

Pass



Photo 97

Lateral resolution

Pass



Photo 98

Axial resolution

Pass

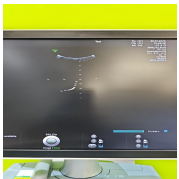


Photo 99

Image benchmark

Pass



Photo 100

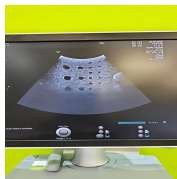


Photo 101

Probe passes inspection & test

Yes

Probe has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Yes

PROBE NO.: 3

Probe Make

GE

Probe Model

C1-5-D

Probe Serial Number

275451YP3



Photo 102

Visual inspection

Pass

Cable

Pass



Photo 103

Lens

Fail

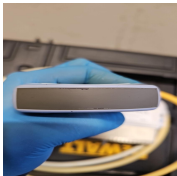


Photo 104

Probe Casing

Pass



Photo 105



Photo 106



Photo 107



Photo 108

Probe Casing Strain Relief

Pass

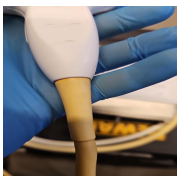


Photo 109

Connector

Pass



Photo 110



Photo 111

Connector Strain Relief

Pass



Photo 112

Lateral resolution

Pass

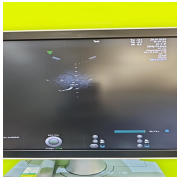


Photo 113

Axial resolution

Pass

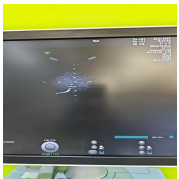


Photo 114

Image benchmark

Pass

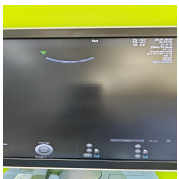


Photo 115

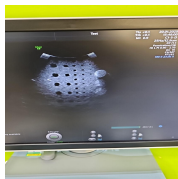


Photo 116

Probe passes inspection & test

Yes

Probe has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Yes

6. SUMMARY

This service has been completed in accordance with our quality procedure.
If you have any questions or concerns, please contact us on
+44 1252 299024
or email: enquiries@britishmedicalauctions.co.uk to discuss further.

System has passed performance and safety assessments and is issued as fit for use, subject to final acceptance assessment upon fit.

Safe

Date Performed:

20.01.2025 16:44 CET

Service Engineer Signature

MS
20.01.2025 16:44 CET

British Medical Auctions
Unit 1, Sterling Centre, Eastern Road, Bracknell, RG12 2PW
Tel: +44 1252 299024
Email: enquiries@britishmedicalauctions.co.uk
Web: www.britishmedicalauctions.co.uk

British Medical Auctions
Ultrasound Equipment Quality Check (EQC)
Preventive Maintenance Service Report (PM Service)

V1.0 05.2022

7. PHOTOGRAPHIC DOCUMENTATION



Photo 117



Photo 118

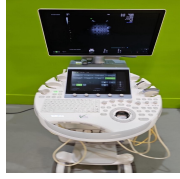


Photo 119

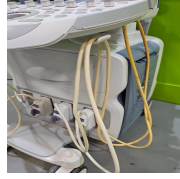


Photo 120



Photo 121

Media summary



Photo 1



Photo 2

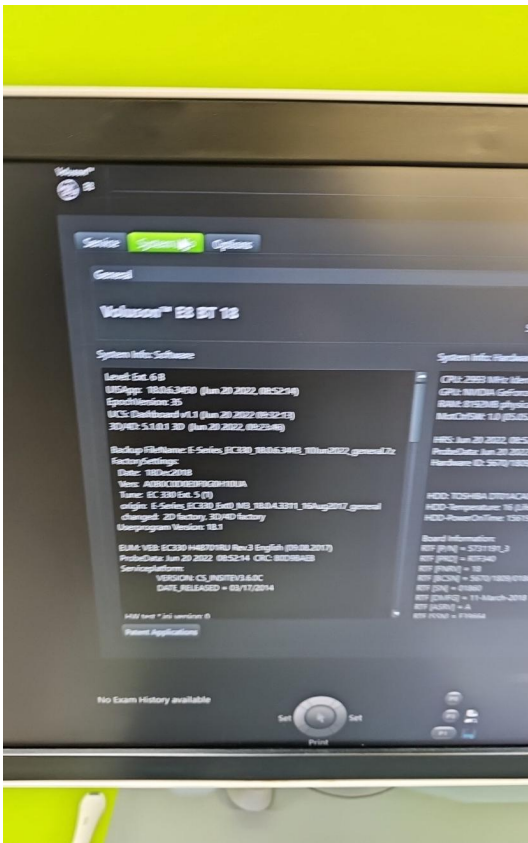


Photo 3

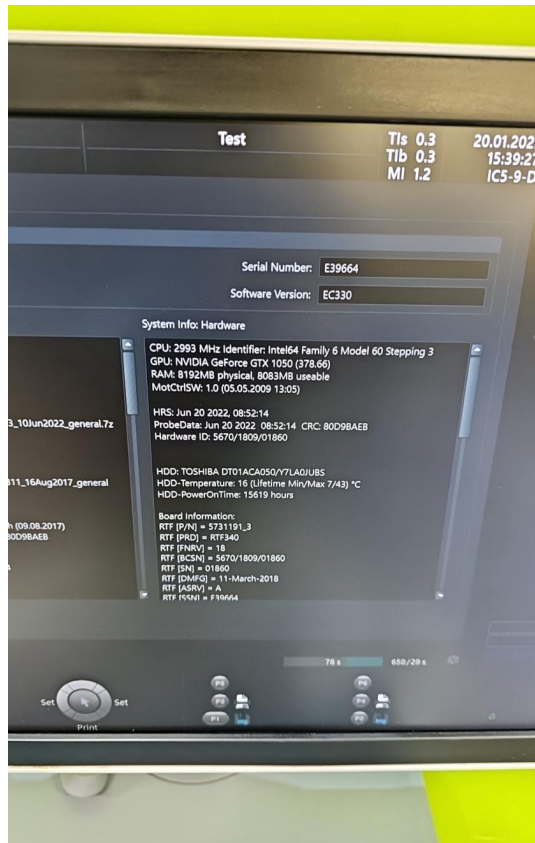


Photo 4

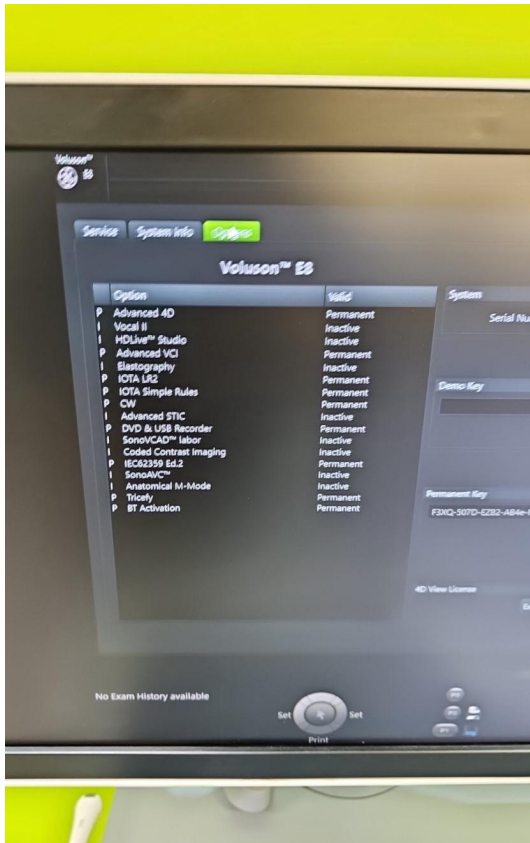


Photo 5

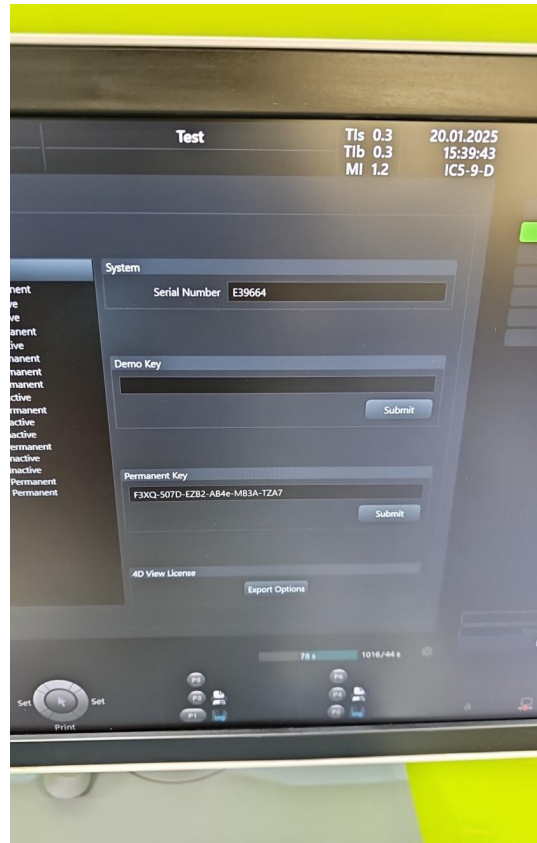


Photo 6



Photo 7



Photo 8

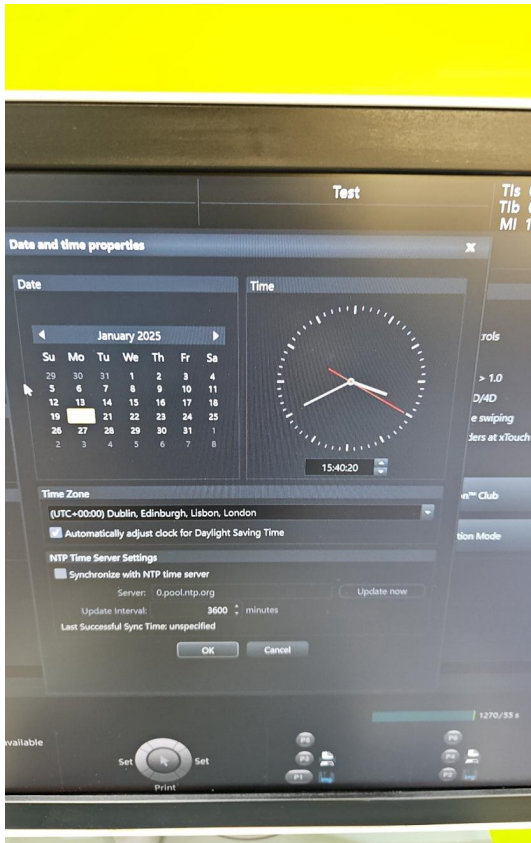


Photo 9



Photo 10



Photo 11



Photo 12



Photo 13



Photo 14



Photo 15



Photo 16



Photo 17

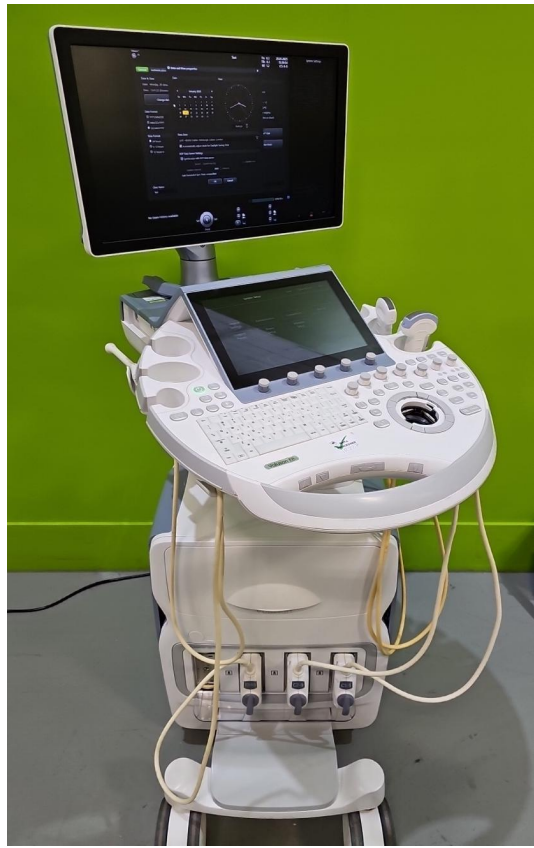


Photo 18



Photo 19



Photo 20



Photo 21



Photo 22



Photo 23



Photo 24



Photo 25



Photo 26



Photo 27

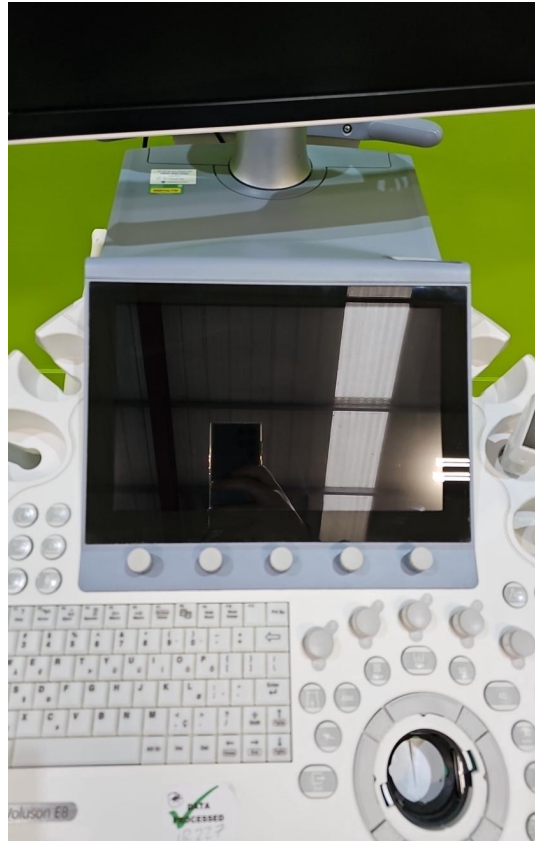


Photo 28



Photo 29



Photo 30

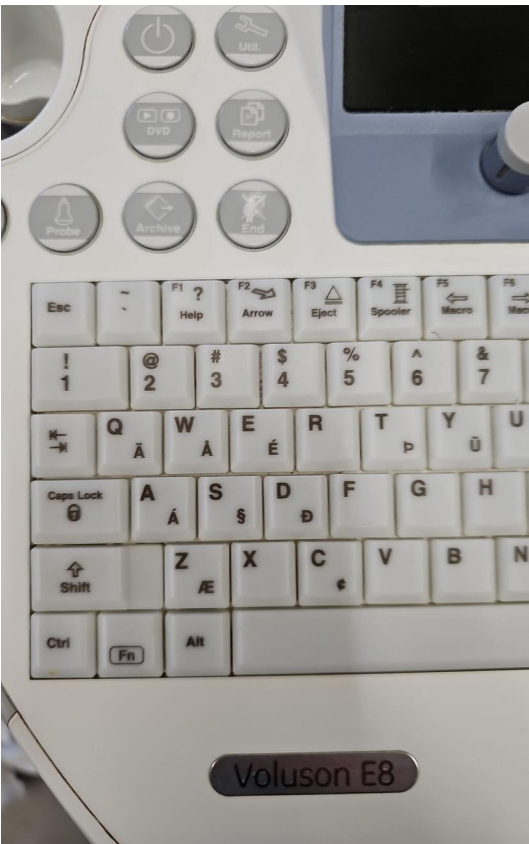


Photo 31

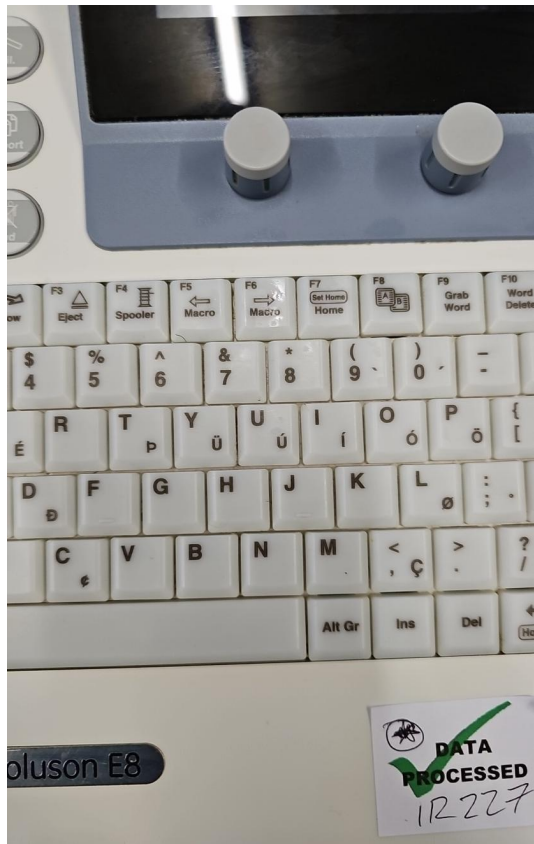


Photo 32

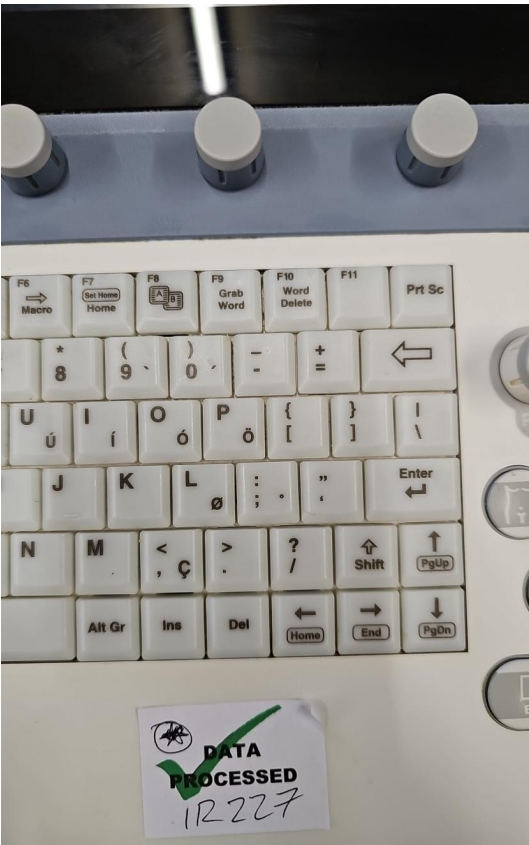


Photo 33

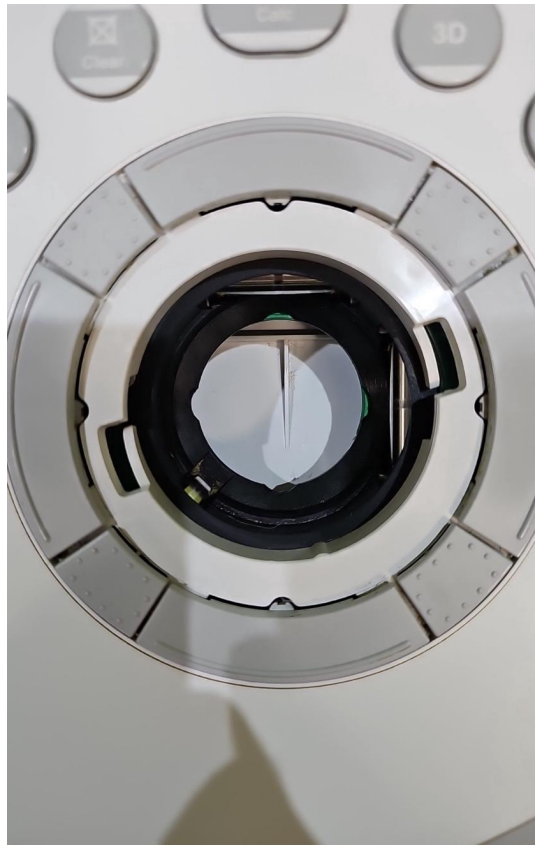


Photo 34



Photo 35



Photo 36

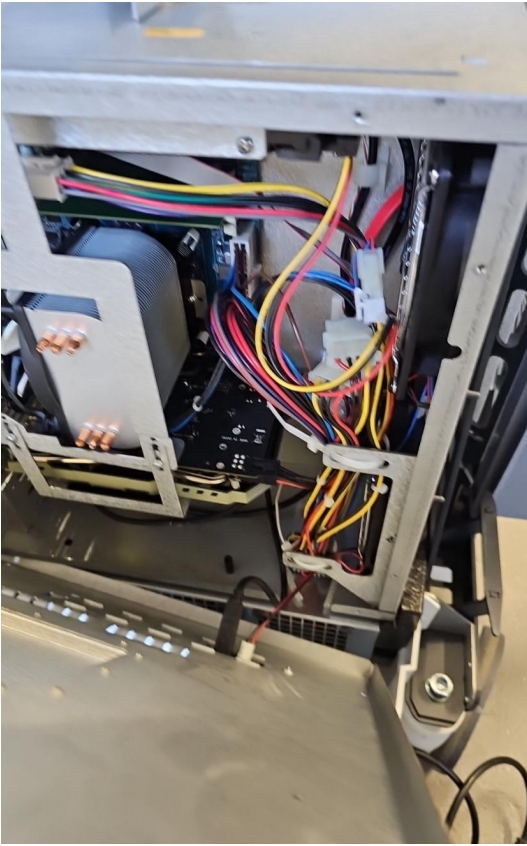


Photo 37

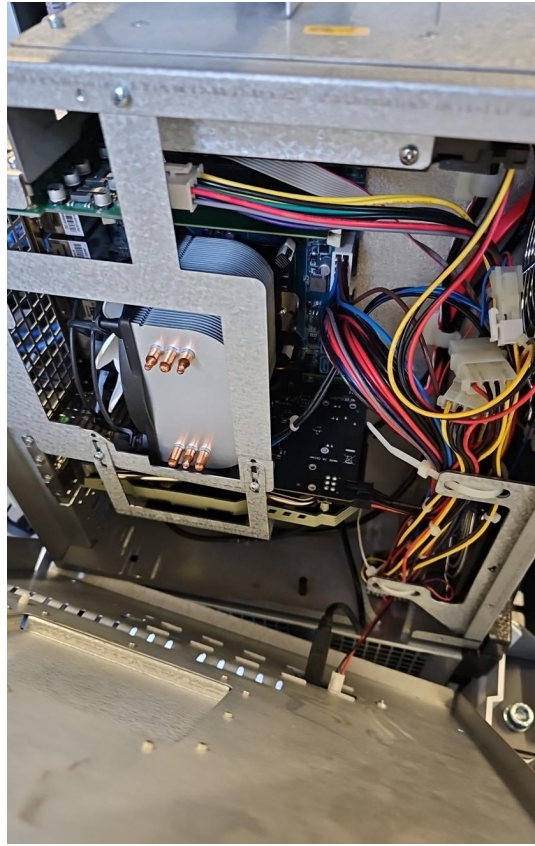


Photo 38

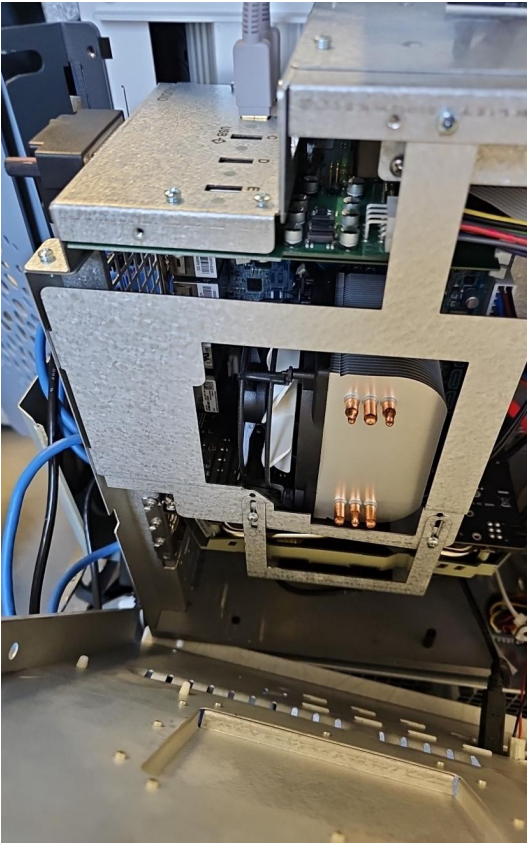


Photo 39

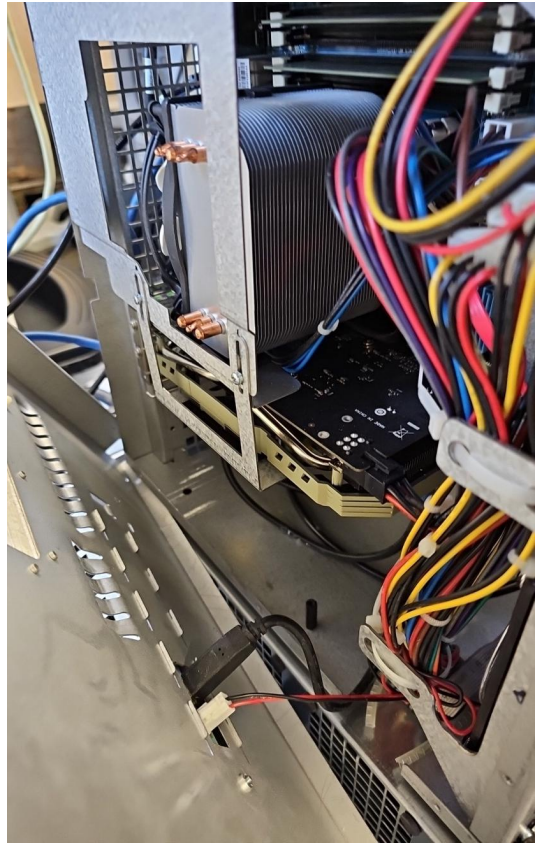


Photo 40



Photo 41



Photo 42

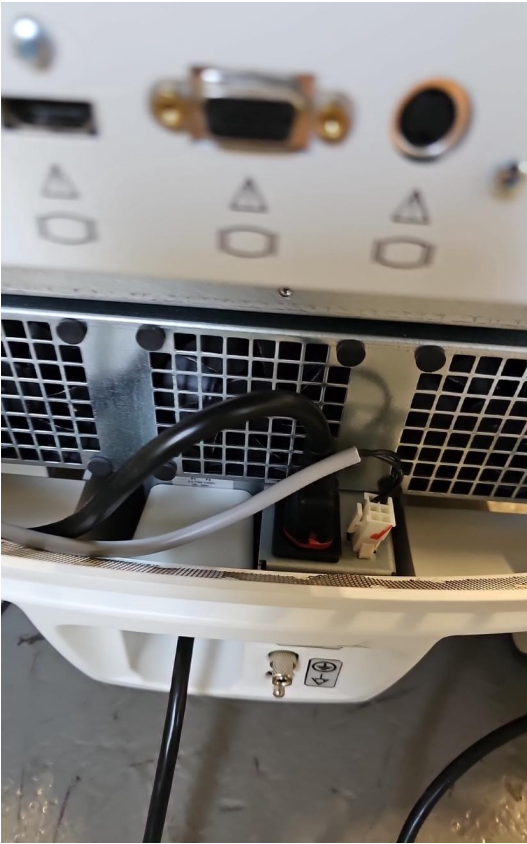


Photo 43



Photo 44



Photo 45



Photo 46

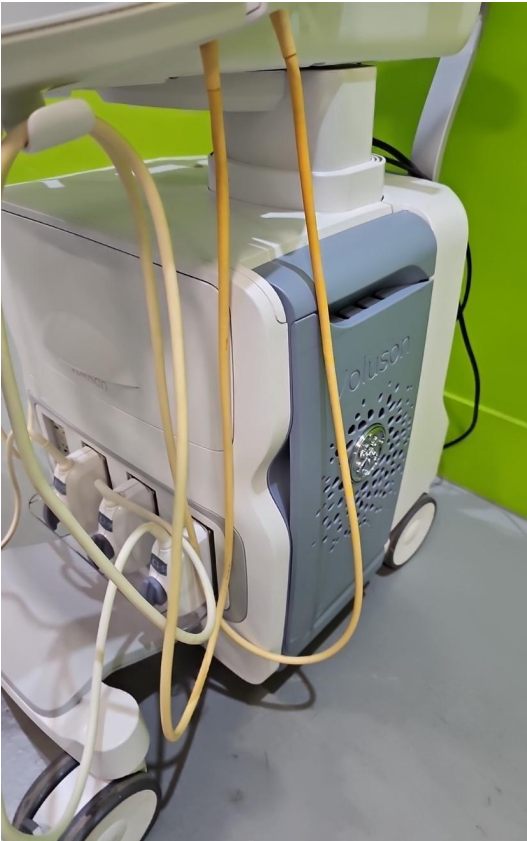


Photo 47



Photo 48



Photo 49

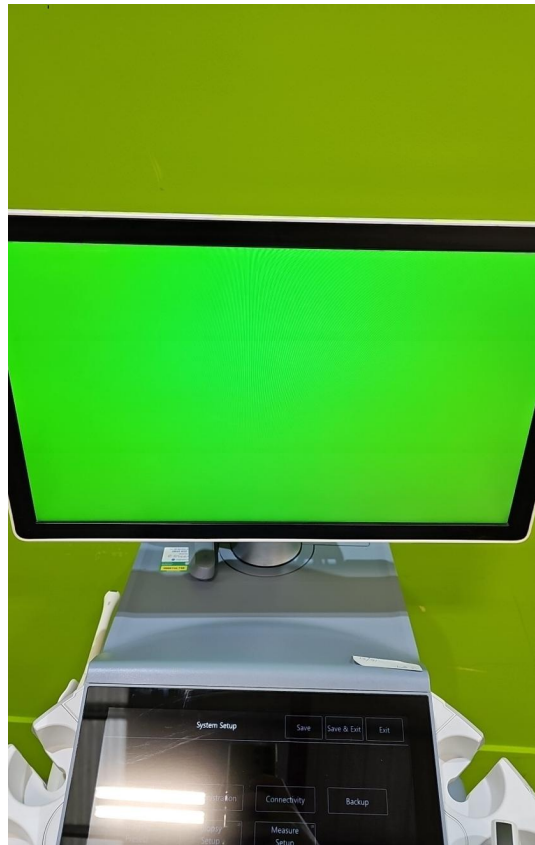


Photo 50



Photo 51



Photo 52



Photo 53



Photo 54



Photo 55

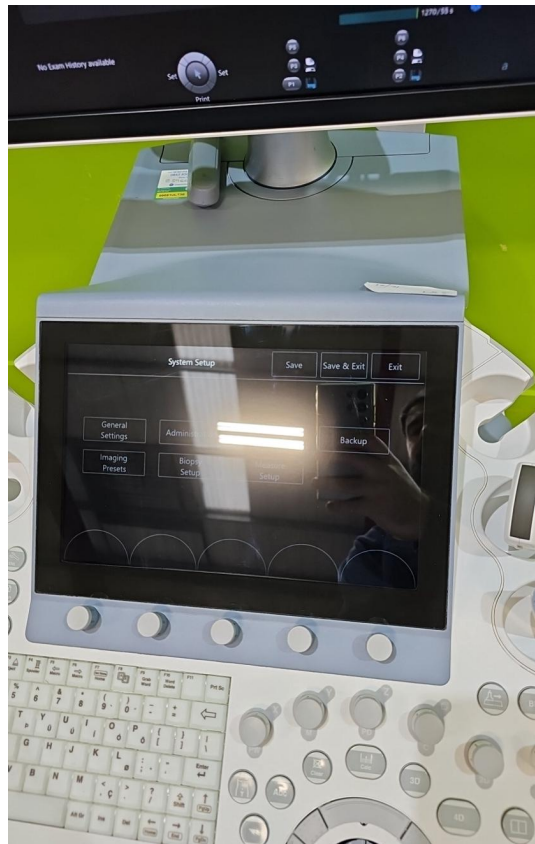


Photo 56

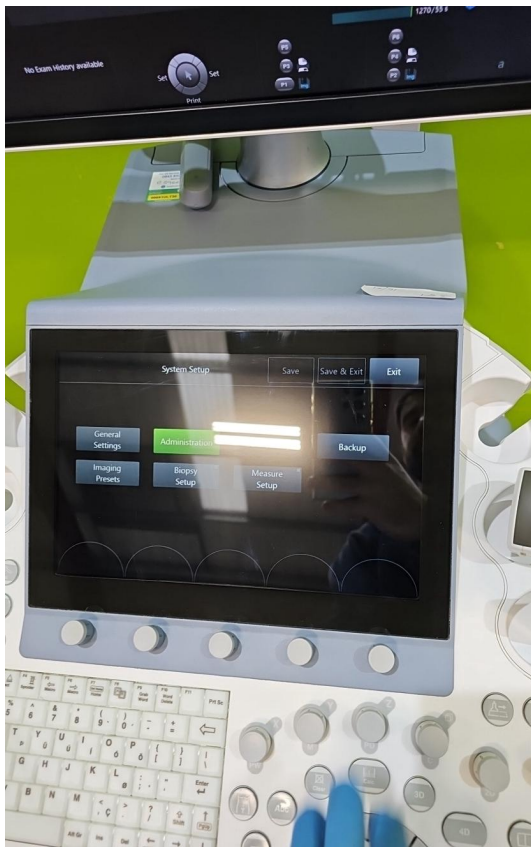


Photo 57

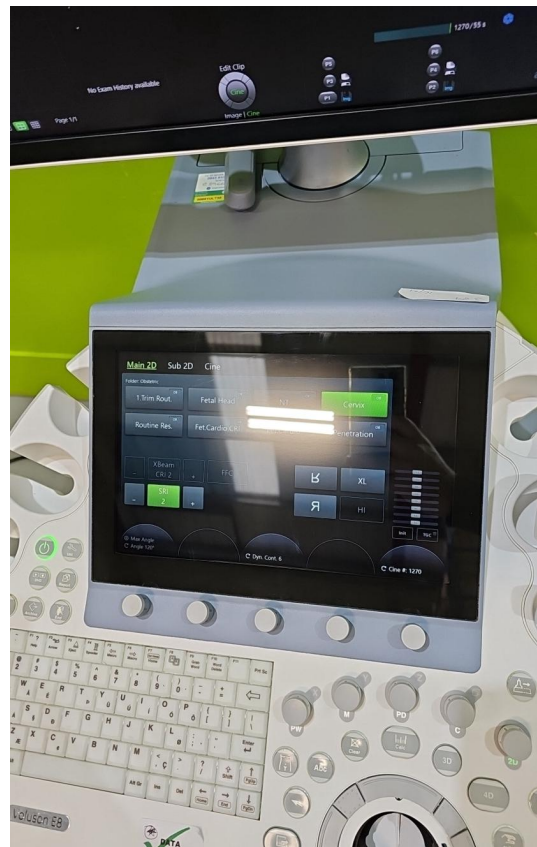


Photo 58



Photo 59



Photo 60

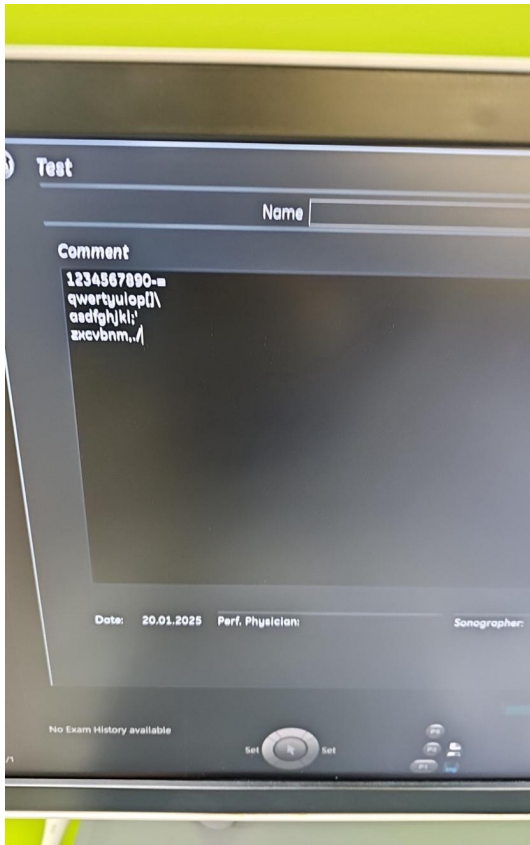


Photo 61

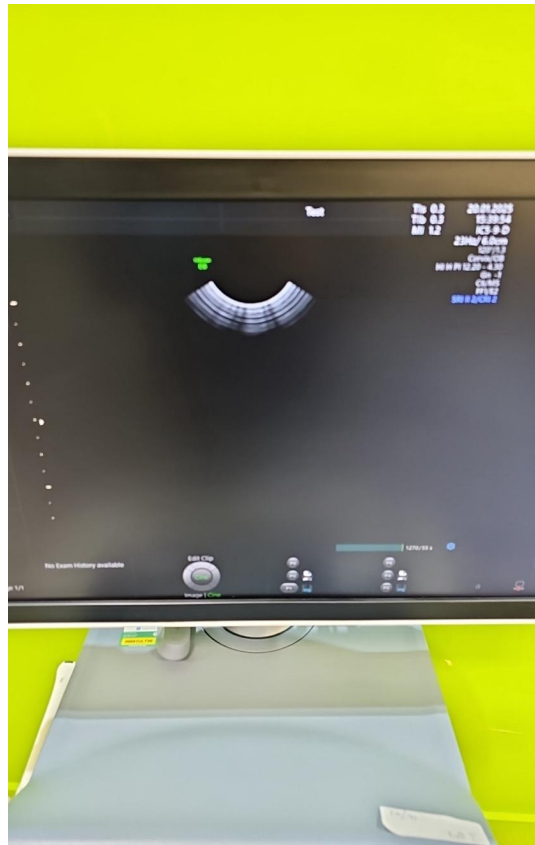


Photo 62



Photo 63

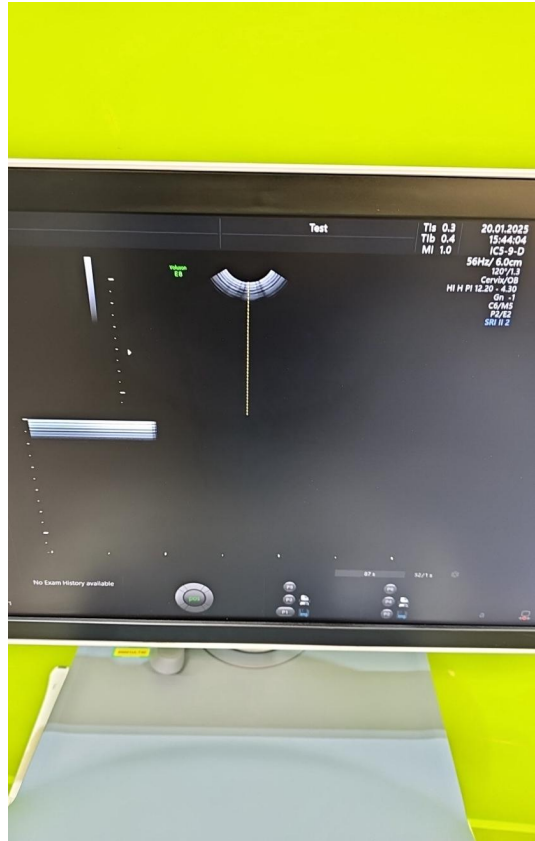


Photo 64

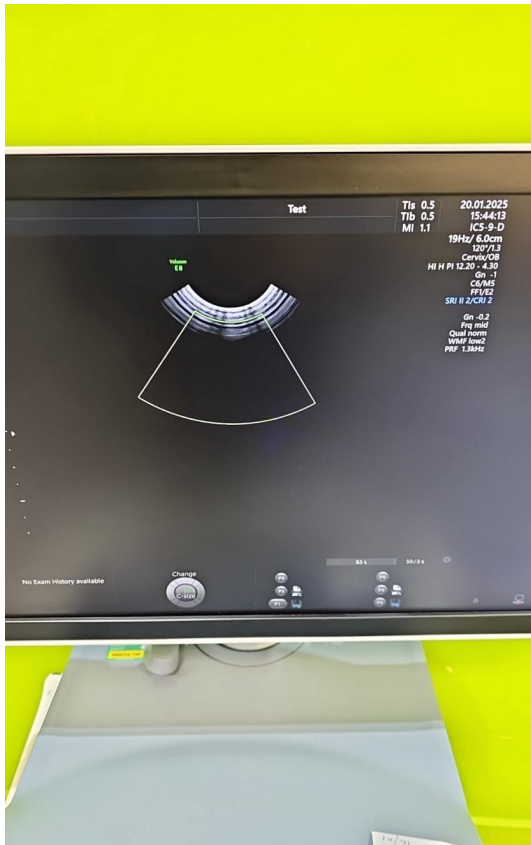


Photo 65

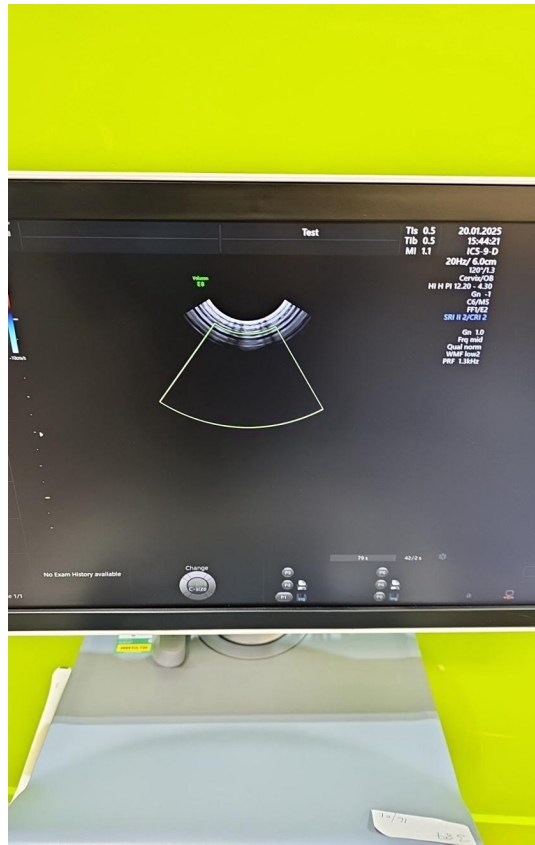


Photo 66



Photo 67

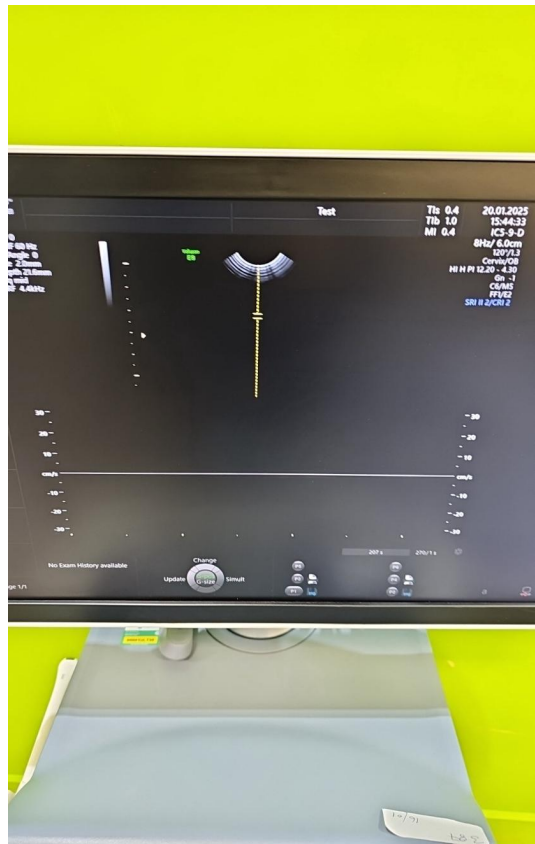


Photo 68



Photo 69



Photo 70



Photo 71

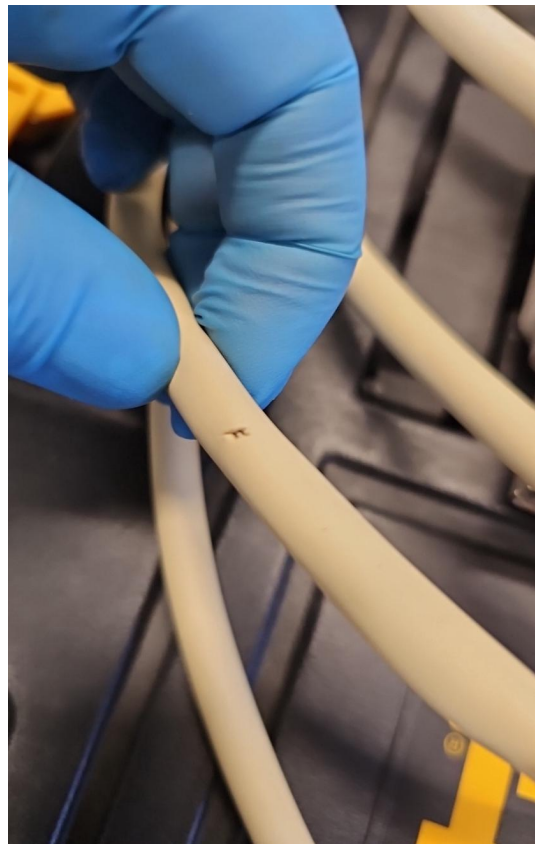


Photo 72



Photo 73

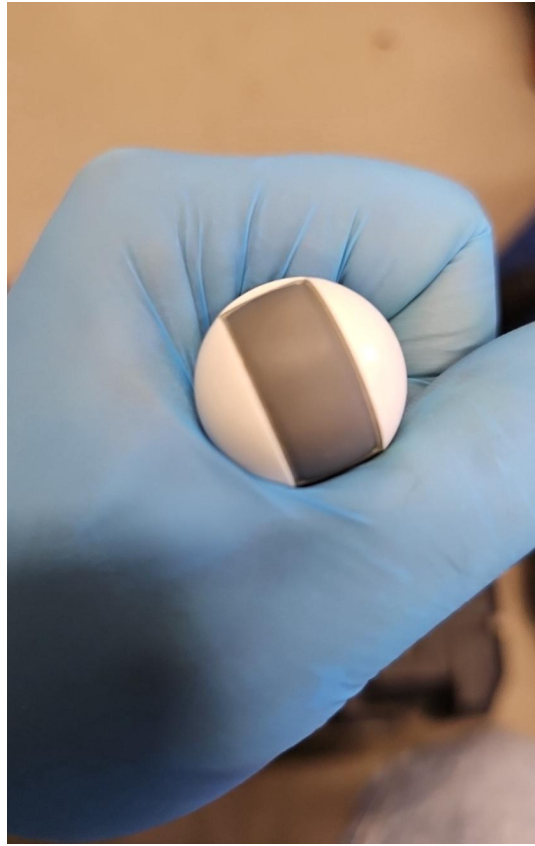


Photo 74

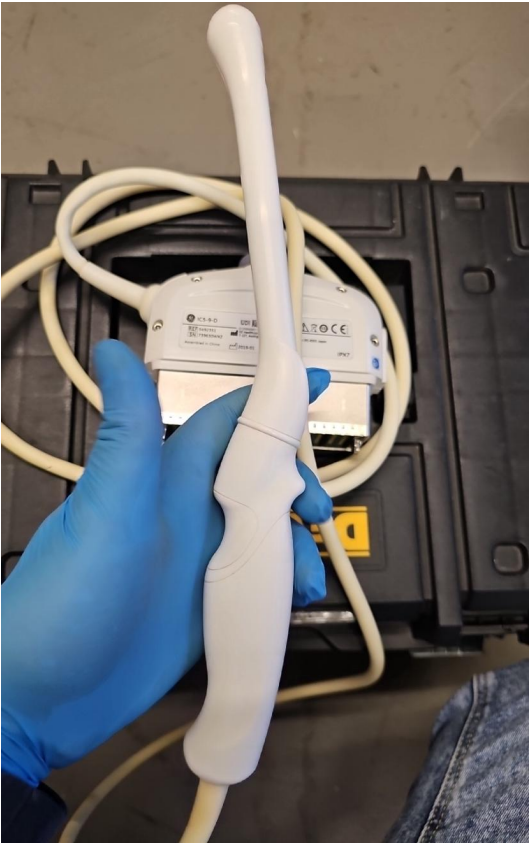


Photo 75

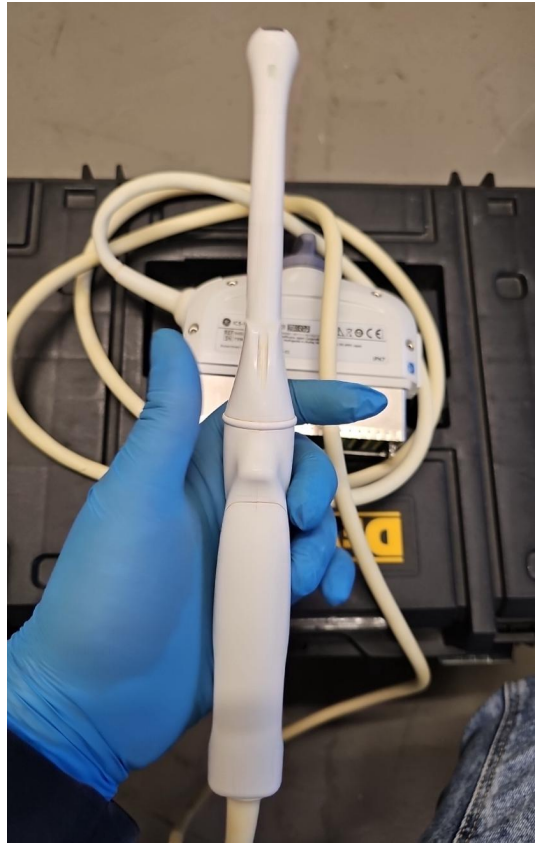


Photo 76

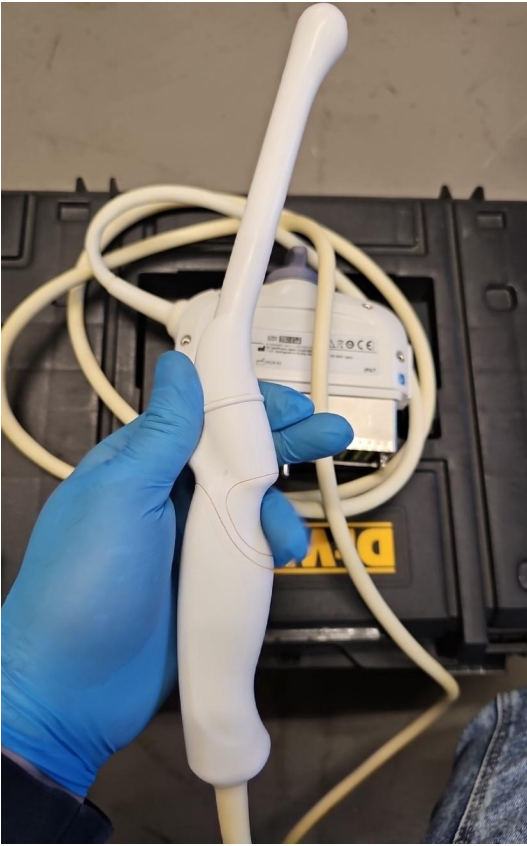


Photo 77

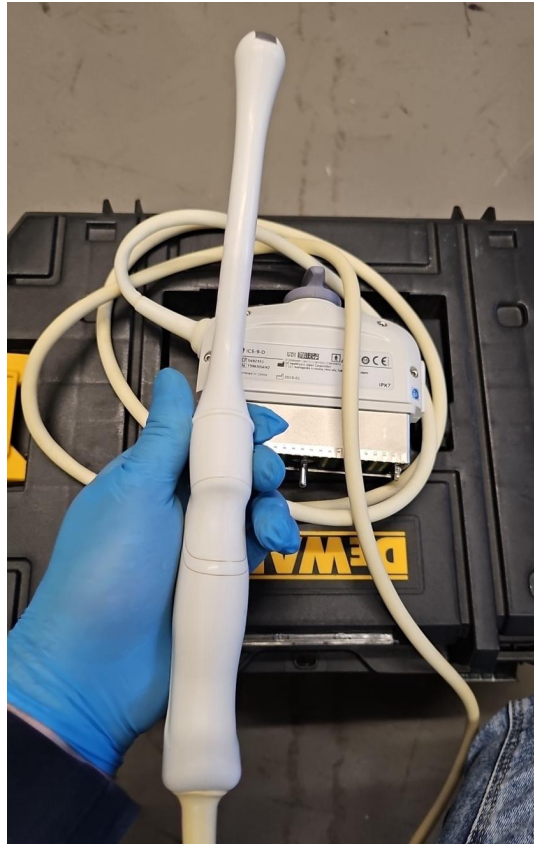


Photo 78



Photo 79

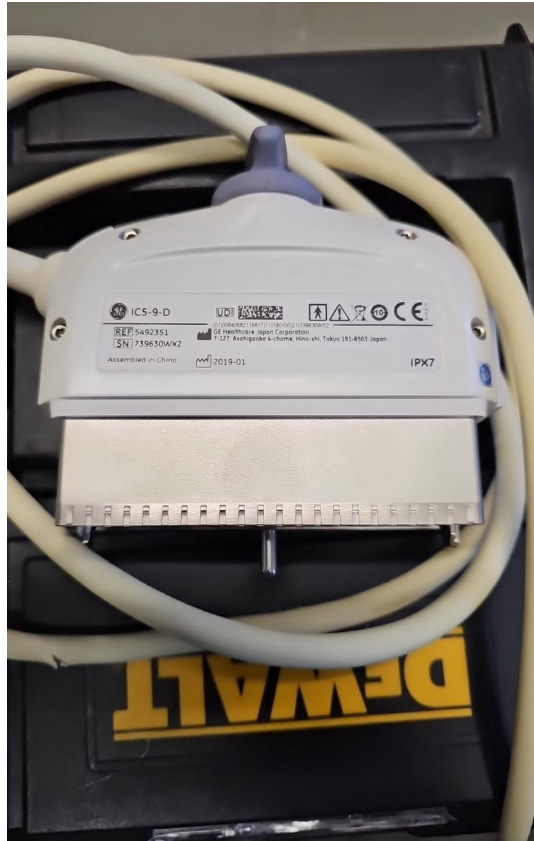


Photo 80



Photo 81



Photo 82

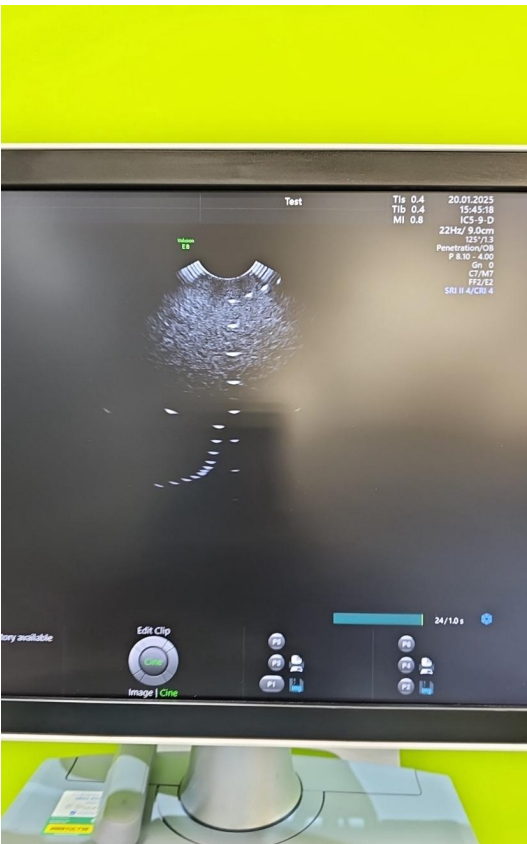


Photo 83

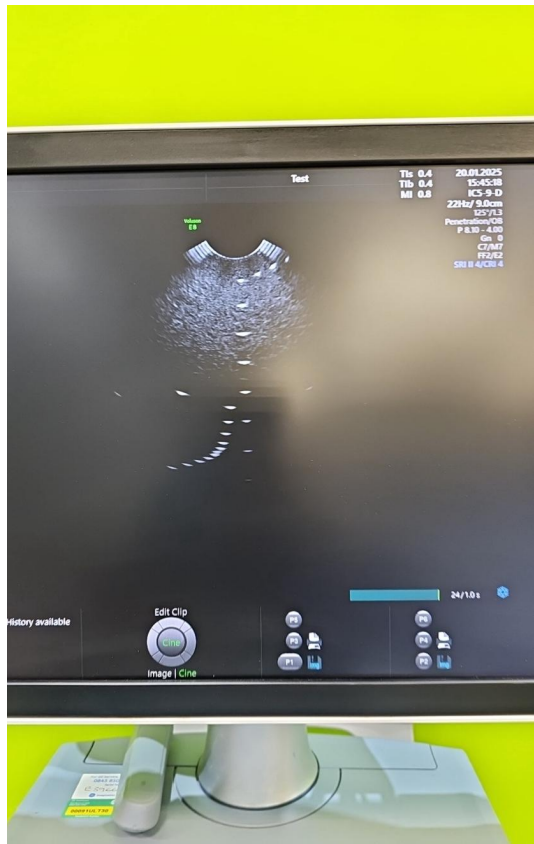


Photo 84

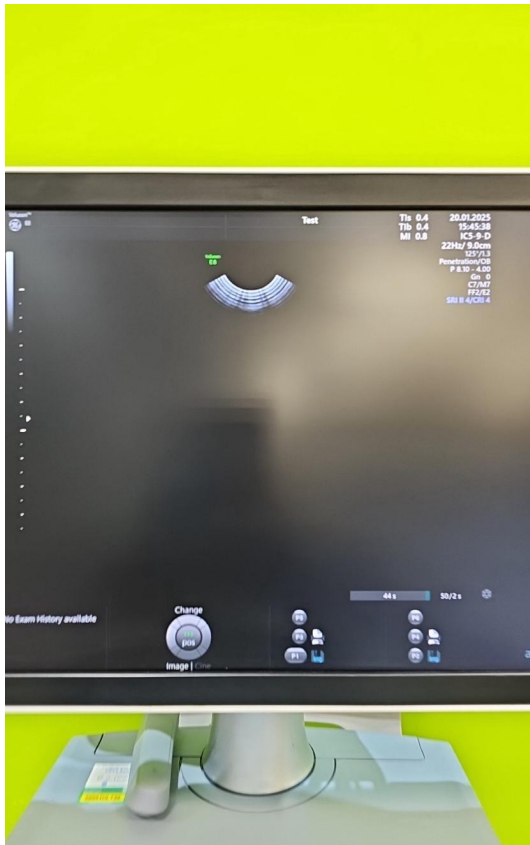


Photo 85

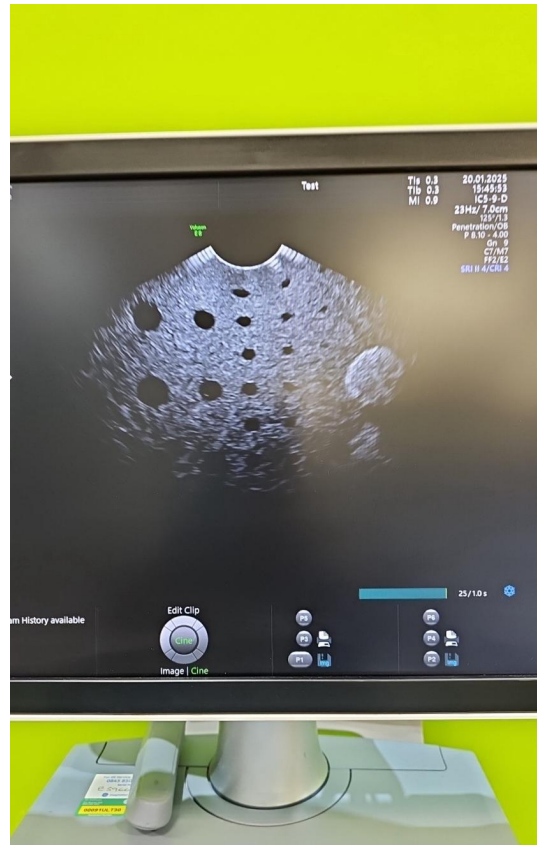


Photo 86

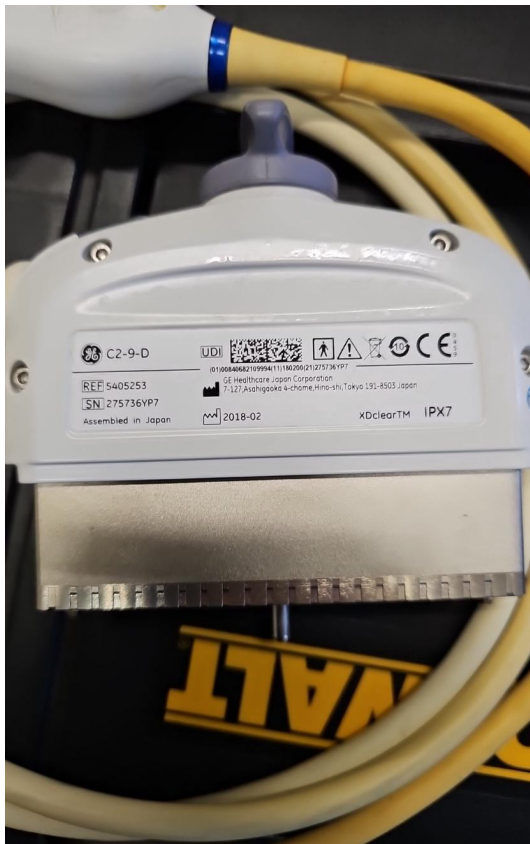


Photo 87



Photo 88



Photo 89



Photo 90



Photo 91



Photo 92



Photo 93



Photo 94



Photo 95

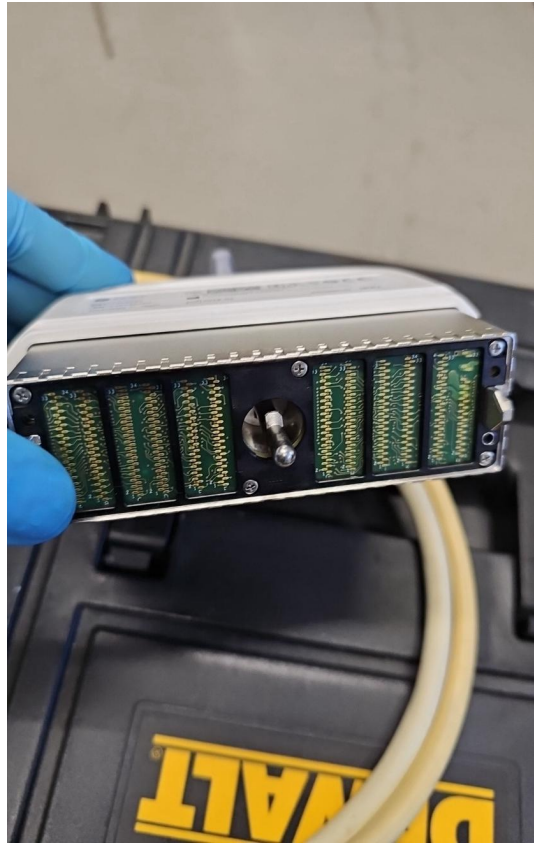


Photo 96



Photo 97

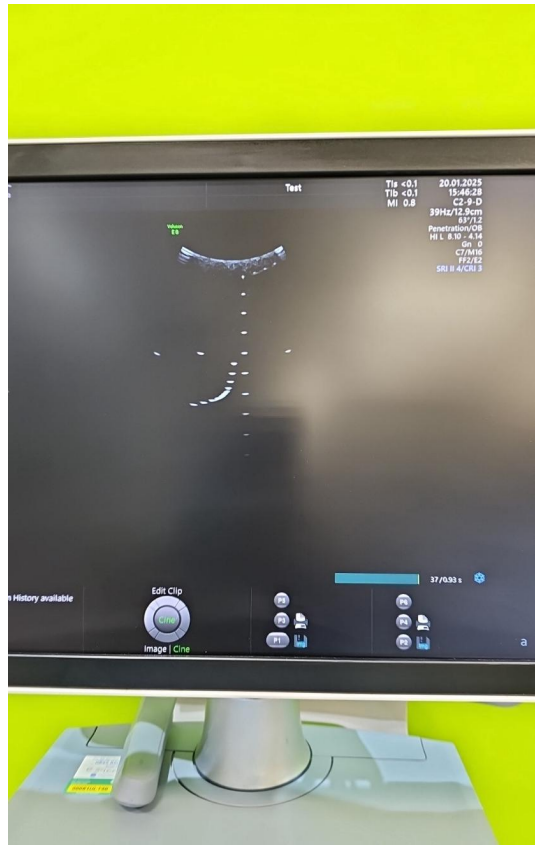


Photo 98

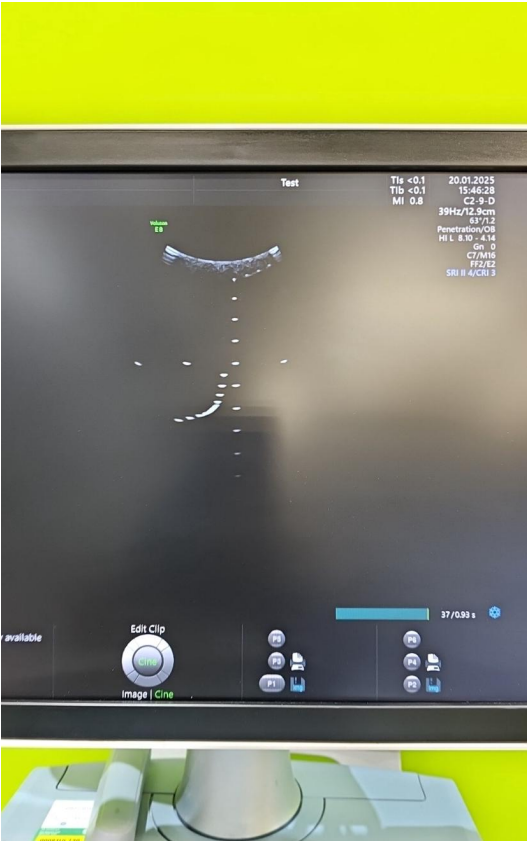


Photo 99

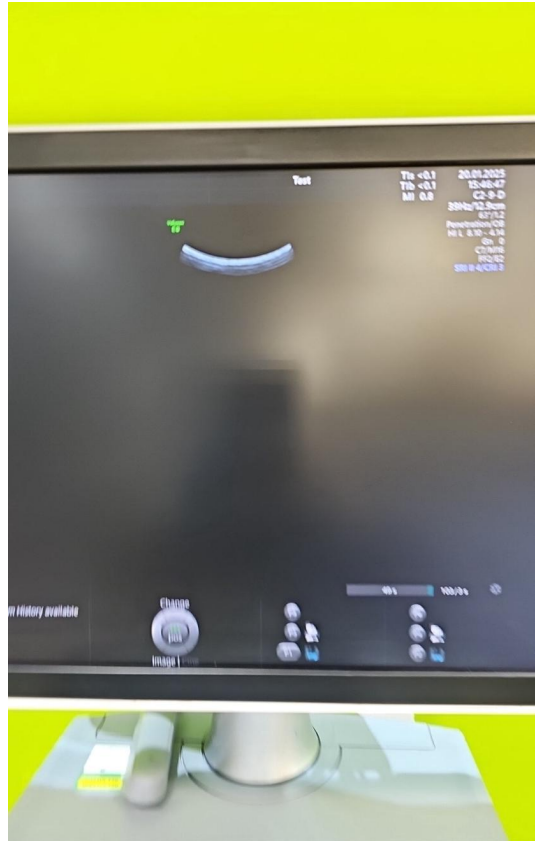


Photo 100

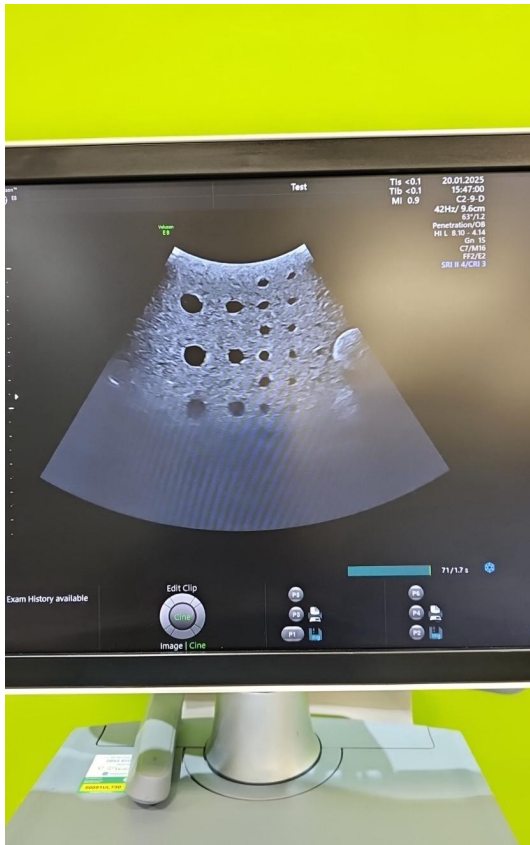


Photo 101



Photo 102



Photo 103



Photo 104



Photo 105



Photo 106



Photo 107



Photo 108

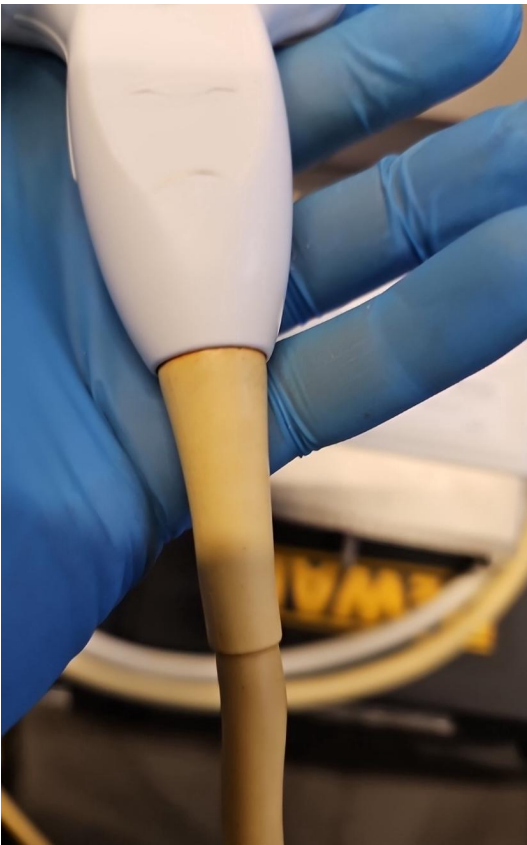


Photo 109



Photo 110

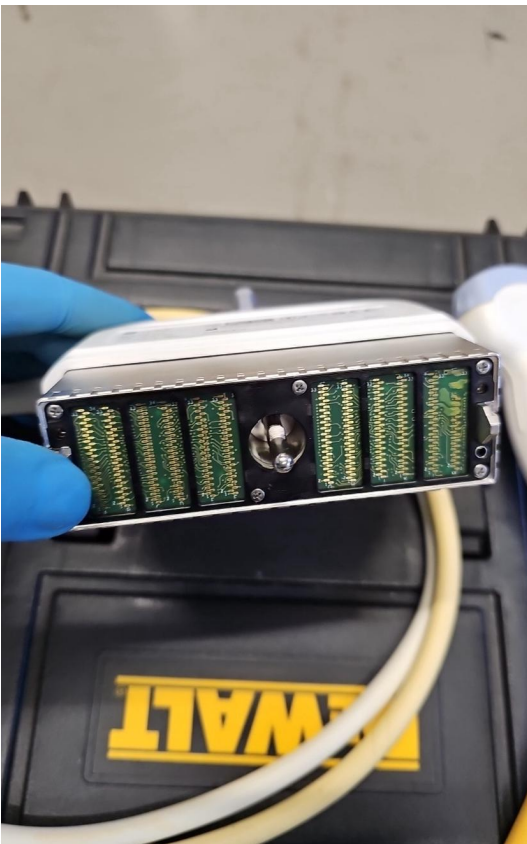


Photo 111



Photo 112

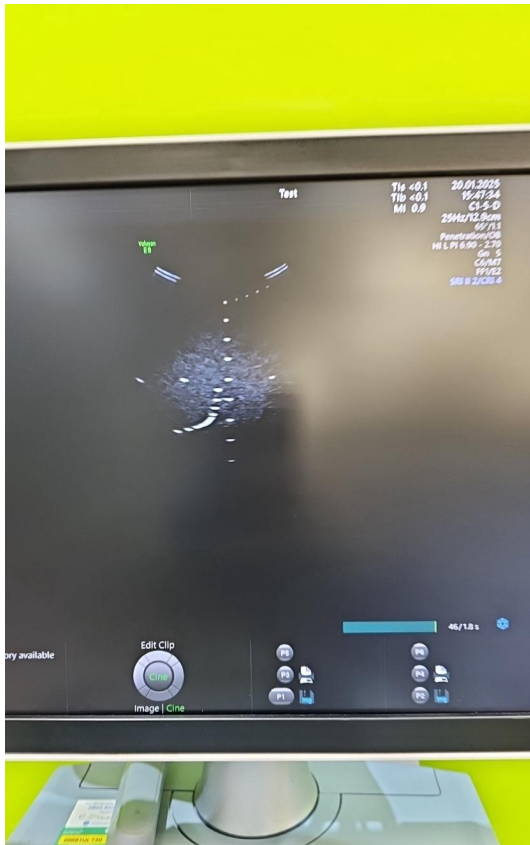


Photo 113



Photo 114

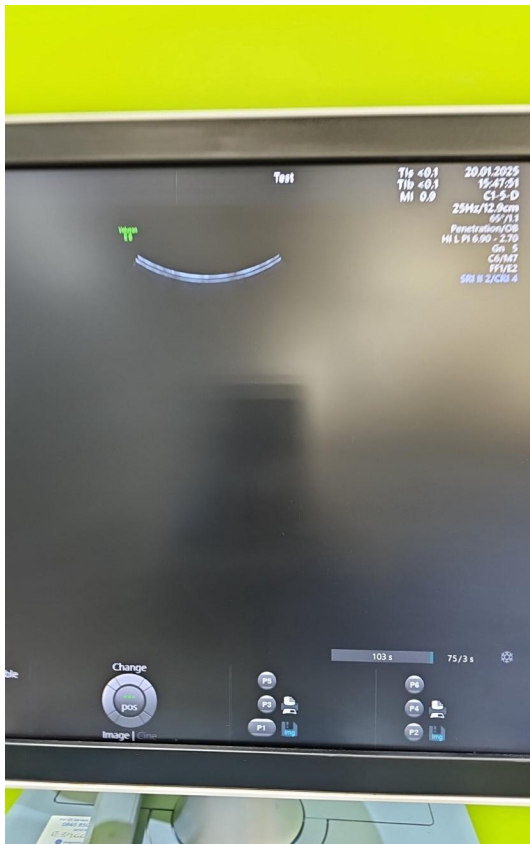


Photo 115



Photo 116



Photo 117



Photo 118



Photo 119



Photo 120



Photo 121